

bebionic[®] (small)

Distal Finger Palp Replacement Guide



Quality for life

The following step-by-step guide is for replacement of the finger palps on small bebionic by Ottobock hands. Please note – this is only possible with size small models. Medium bebionic hands require service through Ottobock.






Please ensure to follow appropriate safety measures including donning of gloves and protective eyewear

bebionic (small): Finger Palp replacement article numbers

Article #	Detailed Updated Description
9S398	Finger Palp Kit – includes palps for all 5 fingers, bebionic small only, both left and right sides
9S318	Finger Palp 3 and 4 (middle and index finger)
9S319	Finger Palp 2 (ring finger)
9S320	Finger Palp 1 (pinkie finger)
9S337	Finger Palp – Thumb

bebionic (small): Replacement of the palp

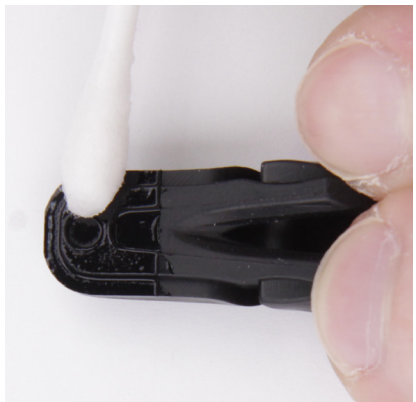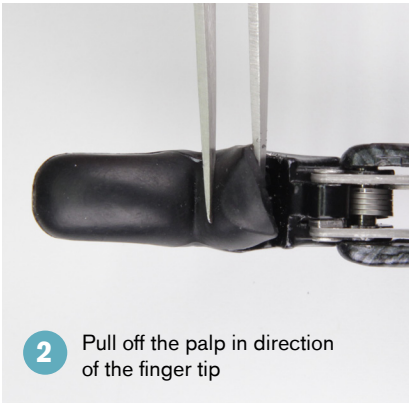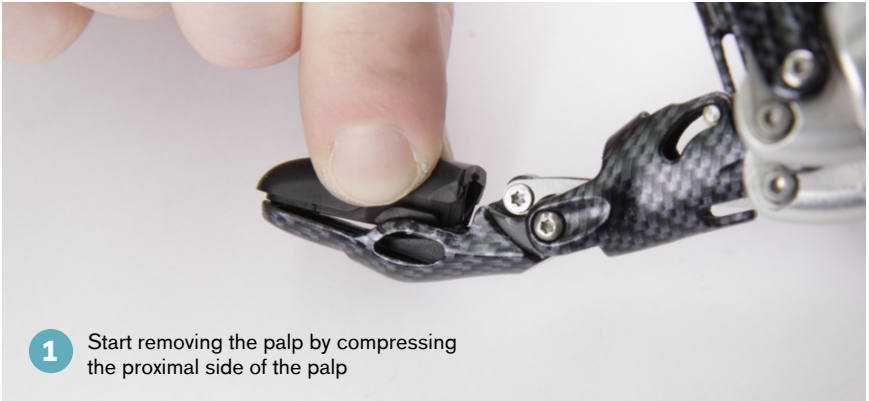

bebionic (small): Replacement of the palp



DISCLAIMER: This guide does not replace the Instructions for Use. The manufacturer will not assume liability for damage caused by disregarding the information in this document or associated Instructions for Use, particularly due to improper use or unauthorized modification of the product.

Ottobock US · P 800 328 4058 · F 800 655 4963 · professionals.ottobockus.com
Ottobock Canada · P 800 665 3327 · F 800 463 3659 · professionals.ottobock.ca