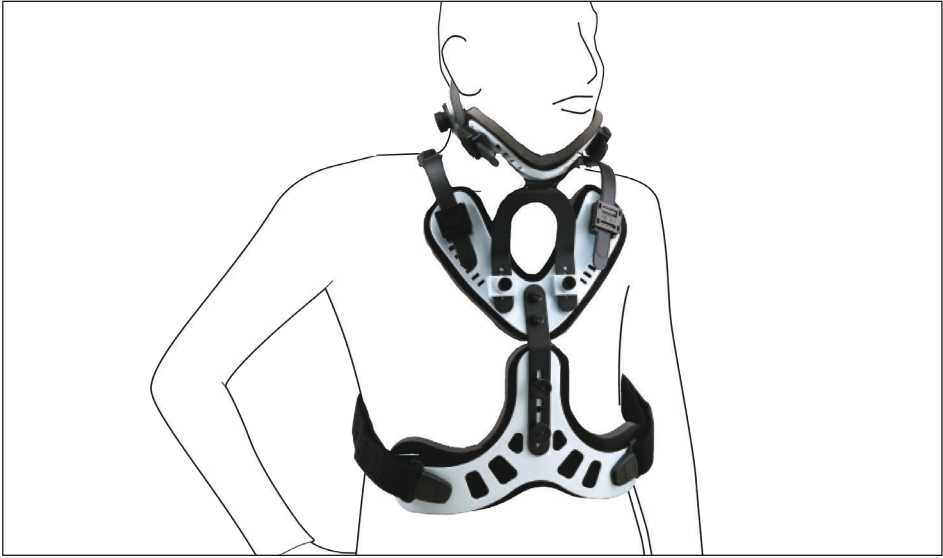
ottobock.

50C71=* Ottobock CTO

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50C71=* Ottobock CTO

Thank you for using an Ottobock device. The following information provides information about the fit and use of the device. Please read the entire document carefully before use and ask questions or clarification.

Clinical fitting instructions are provided on page 3.

Indications for Use

- Post operative non-invasive restriction and immobilization of the cervical and upper thoracic spine
- Jefferson's fracture without transverse ligament injury
- Light wedge fractures following traction or occipital avulsion fractures.
- Post trauma, arthritis, acute pain, spinal stenosis

User Instructions

CAUTION

Usage Restrictions

This device is intended for single patient-use only. Reuse of the device is prohibited. The daily use of the device depends on the medical indication and the prescribing physician orders.

- Carefully follow all instructions provided by the practitioner and your prescribing physician.
- Your device should only be removed to wash your face and neck and to change/clean the liners, unless otherwise instructed by your practitioner or physician.

How to Apply

1. Face a mirror, if possible, to make it easier to reapply the device.
2. To reapply, place the front part of the device with the chin portion on your neck.
3. While securely holding the front portion to your neck, carefully position the back portion around the back of your neck, pull the straps forward until the device is snug, and insert the straps into buckles and secure.
4. Secure the straps and buckles around the waist.

How to Remove

1. To remove the device, release the buckles and slide the straps out on one side of the device, carefully remove the device and set it aside.
2. Keeping your neck still and straight, gently wash and thoroughly dry your face and neck.
3. If necessary, replace liners with clean, dry liners.

Cleaning and Care

CAUTION

It is important to ensure the device is kept clean and dry. Do NOT use a washing machine, dryer, bleach, or harsh chemicals to clean the device.

1. Remove the liners that are attached to the plastic shell with hook and loop.
2. Hand wash the liners with mild soap and water.

3. Rinse thoroughly and gently squeeze to remove excess water. Allow to air dry thoroughly, approximately 6-8 hours.
4. The plastic shell can be cleaned with mild soap and water, if needed. Towel dry thoroughly.

CAUTION

Do not place the device near or on a heat source or open flame. DO NOT USE HEAT TO DRY THE DEVICE.

6. Reapply the back liner by folding the liner in half vertically and placing in the center of the shell. Unfold and press into the shell to secure. The widest part of liner should be at the bottom edge and should extend past the edge at least $\frac{3}{8}$ " all the way around.
7. Reapply the lower front liner and the chin pad liner in the same manner as explained above.

Practitioner Fitting Instructions

1. Measure for the Correct Size
2. Select appropriate size from chart below.

Adult Sizing Guide

Device Size	Chin to Shoulder	Mid-neck circumference	MODEL #
Small / Medium	2 ½" - 4"	11 - 15"	50C71=S/M
Large / XL	3 ¼" - 5 ½"	15 - 20"	50C71=L/XL

Practitioner Application Instructions

1. Apply device to patient's neck, making sure that the mandible is securely in the recess of the chin section and the lower edge is flush with the thoracic region. If the patient is in a recumbent position, apply posterior section of device first. Utilize the black tool that is included to loosen the section.
2. The diffusion tabs at the lower edge of device can be flared away from patient if there are any apparent undue pressure areas.
3. The posterior occipital component has an aluminum spine element that can be shaped for total customization and conformity of the cervical arch.
4. Center occipital component and evenly pull hook and loop tabs around to anterior section. It works best if the posterior ends tuck under the anterior portion.

Manufacturer's Limited Warranty

Otto Bock HealthCare, LP (Ottobock) warrants all of its devices, to the original purchaser, to be free from defects in materials and workmanship. This warranty applies, subject to normal wear and tear, when the devices are used as intended, without unapproved modifications, following all Ottobock instructions and requirements; and when they are fitted by or under the direct supervision of certified/licensed practitioners. This Limited Warranty does not cover device damage caused by accidents, neglect, misuse or operation beyond capacity, parts damaged by improper installation, substitution of parts not approved by Ottobock, or any alteration or repair by others that, in Ottobock's judgment, materially or adversely affect the device.

The duration of this Limited Warranty varies by product types and is effective from the date of delivery to the end-user. Please refer to www.ottobockus.com or call 800 328 4058 for questions. Ottobock's sole obligation under this Limited Warranty shall be to

repair, replace, refabricate the device at no charge, or refund the cost of the device to the original purchaser, at Ottobock's sole discretion.

THE EXPRESS WARRANTIES SET FORTH ABOVE ARE IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND ALL SUCH OTHER WARRANTIES ARE HEREBY DISCLAIMED AND EXCLUDED BY OTTOBOCK. IN NO EVENT SHALL OTTOBOCK'S LIABILITY OF ANY KIND INCLUDE ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF OTTOBOCK SHALL HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH POTENTIAL LOSS OR DAMAGE.

Some states do not allow the exclusion of incidental or consequential damages, thereby rendering the aforementioned limitation in applicable to certain original purchasers.

Special Instructions

As the patient, if you have any questions or problems:

- Call your prescribing physician with any physical or medical problems.
- Call your practitioner for any problems associated with the device.

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