50C80=* Minerva Orthosis

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50C80=* Minerva Orthosis

Thank you for using this Ottobock device. The following information will help you fit and care for this device. Please review it before using this device and ask your physician, your healthcare provider, or your practitioner for clarification if needed.

Indications for Use:

Indicated for Jefferson's fracture without transverse ligament injury, light wedge fractures following traction or occipital post-op fractures, post-trauma, arthritis, acute pain, and spinal stenosis.

Warning

This device does not prevent injury and is not intended to reduce or eliminate the risk of injury.

Warning

Always consult with your physician or healthcare provider before making changes to the device.

Warning

Carefully read all instructions for use before using or fitting this device.

Warning

Proper rehabilitation and activity modification are also a part of a safe treatment program.

Warning

If you experience pain, swelling, or any unusual reaction, contact your physician or healthcare provider immediately.

CAUTION

This device is offered under order of a physician or other qualified healthcare provider.

CAUTION

This device is intended for single patient use and is not intended to be reused on a second patient.

CAUTION

Any attempt at moving a patient in an acute and/or postoperative condition should be facilitated with help from the support staff and all necessary precautions should be taken.

CAUTION

This device should only be removed by the patient with consent from a physician or other qualified healthcare provider.

Application Instructions

Please follow all prescribing physician's orders for frequency and duration of use

1. Logroll patient to one side assuring that the head and neck position is maintained.
2. Place posterior thoracic and occipital component into position. Loosen adjustment knobs then position components to desired location and retighten.
3. Uprights may be shaped by bending over bed rail if necessary. Posterior thoracic component: Fit to clear inferior angle of scapula. Occipital component: Should be placed for maximum total contact by shaping aluminum spine element.
4. Logroll patient back to supine. Place anterior thoracic/chin piece component in place. Loosen knobs and position components to desired location and retighten. Engage cervical straps and adjust as necessary and tighten knobs. Apply strap stops.
5. Adjust thoracic belt circumference, remove alligator hook and loop fastener clip and trim excess. This may be facilitated with quick release buckle in place. Attach forehead strap with inferior edge placed approximately 1 cm superior to eyebrows.
6. Adjust upright as necessary and tighten knobs. Anterior thoracic component:
7. Superior edge flairs should be fit at sub-costal margin.

NOTE: After all adjustments have been made, it is recommended that the brace is removed and Loctite 414 applied using all precautions necessary.

Cleaning and Care

Remove the rigid components. The rigid components can be wiped clean with a damp cloth. Softgoods can be hand washed in warm water (90°F - 110°F or 32°C - 43°C) with mild detergent. Triple rinse, squeeze and air dry. Avoid twisting the material to remove water.

Liability/Warranty

This device is warranted for a period of 90 days. The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use. Details of the warranty are also available at www.ottobockus.com.

Manufacturer's Limited Warranty

Otto Bock HealthCare, LP (OttoBock) warrants all of its devices, to the original purchaser, to be free from defects in materials and workmanship. This warranty applies, subject to normal wear and tear, when the devices are used as intended, without unapproved modifications, following all OttoBock instructions and requirements; and when they are fitted by or under the direct supervision of certified/licensed practitioners. This Limited Warranty does not cover device damage caused by accidents, neglect, misuse or operation beyond capacity, parts damaged by improper installation, substitution of parts not approved by OttoBock, or any alteration or repair by others that, in OttoBock's judgment, materially or adversely affect the device.

The duration of this Limited Warranty varies by product types and is effective from the date of delivery to the end-user. Please refer to www.ottobockus.com or call 800 328 4058 for questions. OttoBock's sole obligation under this Limited Warranty shall be to repair, replace, refurbish the device at no charge, or refund the cost of the device to the original purchaser, at OttoBock's sole discretion.

THE EXPRESS WARRANTIES SET FORTH ABOVE ARE IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND ALL SUCH OTHER WARRANTIES ARE HEREBY DISCLAIMED AND EXCLUDED BY OTTOBOCK. IN NO EVENT SHALL OTTOBOCK'S LIABILITY OF ANY KIND INCLUDE ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF OTTOBOCK SHALL HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH POTENTIAL LOSS OR DAMAGE.

Some states do not allow the exclusion of incidental or consequential damages, thereby rendering the aforementioned limitation in applicable to certain original purchasers.

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