Your child's doctor will determine a daily wearing schedule for the hip abduction orthosis. Since this orthosis is waterproof, it may remain in place during bathing and other activities.

Highly stable shoes with good traction are suggested to help your child stand safely when wearing the orthosis. In addition, the outer edge of the shoes may be wedged by 3 mm to make it easier to stand. The limited ball joints in the orthosis are designed to minimize the torsion of the thigh supports while also providing optimal leg abduction during walking (fig. 5).

Regular physician examinations are an important factor in the success of the orthosis. During these exams, your child's doctor will examine his/her hip joints as well as the fit and function of the orthosis. Adjustments will be made in cooperation with the orthotist.

Once the child is to be weaned from the orthosis, please follow the doctor's instructions concerning modified wearing conditions.

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designed by John und Korn

Information for the Parents

When treating hip dysplasia, it is necessary to slightly abduct (spread) your child's legs and to hold them in this position. This 28L20 Hip Abduction Orthosis was designed for children ages 6 to 18 months. Its function is to control your child's leg movements as much as possible (fig. 1). The orthosis positions the hip joint head to promote development of the acetabulum (hip socket). This particular type of treatment (devised by Hoffman and Daimler) has been used effectively for decades.

Your child's hip abduction orthosis will have been initially provided by a physician in cooperation with an orthotist. Treatment will include adjusting abduction via the abduction unit as well as examining for fit and function (fig. 2). Because your cooperation as parents is vital to the successful treatment of your child we ask that you review the information that follows about the use of this orthosis.

To apply the orthosis, place the supports on the front of the child's thighs (fig. 3). Only the hook and loop closures are needed to apply or remove the orthosis (fig. 4). Before applying the thigh supports, place your thumb and index finger on the child's knee to locate the best position for the hollow in the support. The child's legs may be either extended or slightly flexed. The orthosis may be applied over bare skin and under loose trousers or over cotton stockings (available at medical specialty stores) or over thin, close-fitting trousers or leggings. The enclosed condyle pads may be glued in place by the orthotist.

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Otto Bock HealthCare Deutschland GmbH