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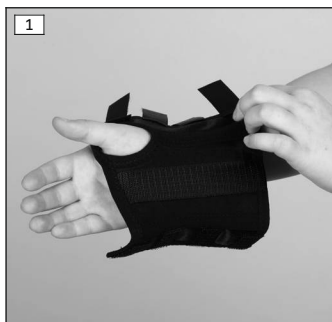
## INFORMATION

Last update: 2017-06-06

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

These instructions for use provide important information on the fitting and application of the 4067 Wrist Support Child (black) and the 4068 Wrist Support Child.

## 2 Intended use

### 2.1 Indications for use

The support is intended **exclusively** for treating the upper limbs and **exclusively** for contact with intact skin.

The support must be used according to the indication.

### 2.2 Indications

- Irritation of the carpometacarpal and the metacarpophalangeal joint
- Sprains
- Subluxation/dislocation
- Tendovaginitis in the hand
- Juvenile chronic polyarthritis

The indication must be determined by the physician.

### 2.3 Contraindications

#### 2.3.1 Absolute Contraindications

None known.

#### 2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent, swollen scars, reddening and hyperthermia in the relevant area; pronounced varicose veins, especially with impaired blood return, lymph drainage disorders and unclear soft tissue swelling distal to the product; sensory and circulatory disorders in the area of the forearm and hand, e.g. associated with diabetic neuropathy.

## 2.4 Mechanism of Action

The support stabilizes the wrist.

## 3 Safety

### 3.1 Explanation of warning symbols

**CAUTION** Warning regarding possible risks of accident or injury.

**NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

**CAUTION**

#### Use of a product with neoprene

Allergic reactions due to neoprene or heat allergies

- ▶ Do not wear products with neoprene for more than 3 – 4 hours without interruption (unless explicitly instructed otherwise by the doctor).
- ▶ Do **not** use the product in case of a known neoprene or heat allergy.

**CAUTION**

#### Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

**CAUTION**

#### Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

**NOTICE**

#### Improper use and changes

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

#### NOTICE

#### Contact with ointments, lotions, or other products that contain oils or acids

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to ointments, lotions, or other products that contain oils or acids.

## 4 Handling

#### INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

### 4.1 Selecting the Size

- 1) Measure the wrist circumference.
- 2) Determine the size of the support (see size chart).

### 4.2 Adaptation

- ▶ Adjust the splints individually.

### 4.3 Application

#### ⚠ CAUTION

#### Incorrect or excessively tight application

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

- > Open all hook-and-loop closures on the support.
- 1) Place the support on the wrist (see fig. 1).
- 2) Close the hook-and-loop closures and fit to the thumb (see fig. 2).
- The support fits snugly without pinching.

## 4.4 Cleaning

### NOTICE

#### Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- Only clean the product with the approved cleaning agents.

- 1) Remove all metal splints.
- 2) Fasten all hook-and-loop closures.
- 3) Recommendation: Use a laundry bag or net.
- 4) Wash the support in warm water at **40 °C** with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 5) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).
- 6) Reinsert the metal splints.

## 5 Disposal

Dispose of the product in accordance with national regulations.

## 6 Legal Information

All legal conditions are subject to the respective national law of the country of use and may vary accordingly.

### 6.1 Liability

The manufacturer shall be liable in the event that the product is used in accordance with the descriptions and instructions in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorized modification of the product.

### 6.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The manufacturer therefore drew up the declaration of conformity on its own responsibility in accordance with Annex VII of the directive.

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Otto Bock HealthCare GmbH