
## 4R74=AL

<b>DE</b> Gebrauchsanweisung .....	3	<b>HU</b> Használati utasítás .....	49
<b>EN</b> Instructions for use .....	6	<b>CS</b> Návod k použití.....	53
<b>FR</b> Instructions d'utilisation.....	10	<b>RO</b> Instrucțiuni de utilizare.....	56
<b>IT</b> Istruzioni per l'uso .....	14	<b>HR</b> Upute za uporabu .....	60
<b>ES</b> Instrucciones de uso .....	18	<b>SL</b> Navodila za uporabo .....	64
<b>PT</b> Manual de utilização.....	22	<b>SK</b> Návod na používanie .....	68
<b>NL</b> Gebruiksaanwijzing.....	26	<b>BG</b> Инструкция за употреба.....	72
<b>SV</b> Bruksanvisning .....	30	<b>TR</b> Kullanma talimatı .....	76
<b>DA</b> Brugsanvisning .....	34	<b>EL</b> Οδηγίες χρήσης .....	79
<b>NO</b> Bruksanvisning.....	37	<b>RU</b> Руководство по применению .....	83
<b>FI</b> Käyttöohje.....	41	<b>ZH</b> 使用说明书.....	88
<b>PL</b> Instrukcja użytkowania.....	45	<b>KO</b> 사용 설명서.....	91

1

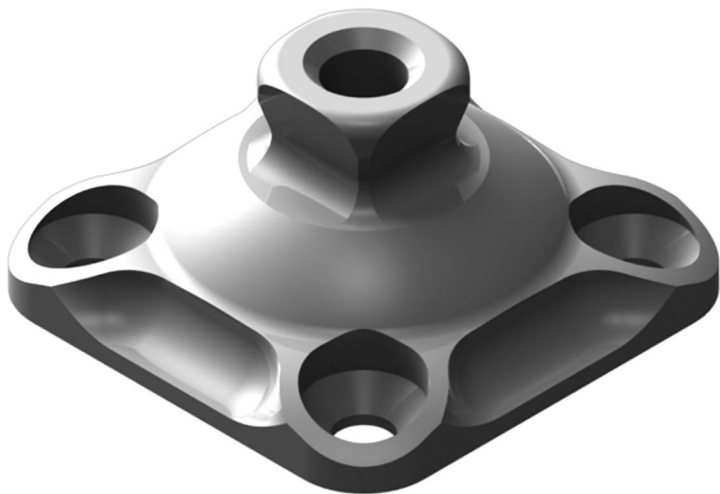# 1 Produktbeschreibung

Deutsch

## INFORMATION

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### **INFORMATION**

Last update: 2015-04-22

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

## 1.1 Construction and Function

Socket adapters are used as components in modular lower limb prostheses. They connect the socket connector of a transfemoral or transtibial prosthesis to the distal components of the prosthesis. These instructions for use apply to the following socket adapters:

Reference number	Distal connection
4R74=AL	Pyramid adapter with bore for shuttle lock

## 1.2 Combination Possibilities

This prosthetic component is part of the Ottobock modular system. As a result of its design, it may be combined with other prosthetic components of the modular system. You can find possible restrictions in this section. Contact the manufacturer with questions.

## 2 Application

### 2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

### 2.2 Area of Application

Approved for a body weight of up to **136 kg (300 lbs)**.

### 2.3 Environmental Conditions

Allowable environmental conditions
Temperature range for use: -10 °C to +60°C
Allowable relative humidity 0 % to 90 %, non-condensing

Unallowable environmental conditions
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids
Dust, sand, highly hygroscopic particles (e.g. talcum)

### 2.4 Service Life

This prosthetic component has been tested by the manufacturer for 3 million load cycles according to ISO 10328. Depending on the

patient's activity level, this corresponds to a service life of 3 to 5 years.

## 3 Safety

### 3.1 Explanation of Warning Symbols

**CAUTION** Warning regarding possible risks of accident or injury.

**NOTICE** Warning regarding possible technical damage.

### 3.2 General Safety Instructions

**CAUTION**

#### Unallowable combination of prosthesis components

Fall due to breakage or deformation of the product

- ▶ Only combine the product with prosthesis components that are approved according to the section "Combination Possibilities".
- ▶ Based on the instructions for use of the prosthesis components, verify that they may be combined with each other.

**CAUTION**

#### Excessive strain on the product

Fall due to breakage of load-bearing components

- ▶ Use the prosthetic components according to the classification (see section "Area of Application").

**CAUTION**

#### Exceeding the service life

Fall due to change in or loss of functionality and damage to the product

- ▶ Ensure that the approved service life is not exceeded (see section "Service life").

**CAUTION****Use under unallowable environmental conditions**

Fall due to damaged product

- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

**CAUTION****Mechanical damage to the product**

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

**NOTICE****Mechanical overload**

Impaired functionality due to mechanical damage

- ▶ Check the product for damage prior to each use.
- ▶ Do not use the product if its functionality has been impaired.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

**Signs of changes in or loss of functionality during use**

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

**4 Scope of Delivery**

The following single components and accessories are included in the scope of delivery in the specified quantities and are available for reordering as single components (■), single components with minimum order quantities (▲) or single-component packs (●):

**4R74=AL socket adapter**

Fig.	Item	Quantity	Designation	Reference number
-	-	1 (■)	Instructions for use	-
1	-	1	Socket adapter	-
-	-	-(▲)	Countersunk head screw	501S41=M6x12
-	-	-(▲)	Countersunk head screw	501S41=M6x25
-	-	-(▲)	Countersunk head screw	501S41=M6x30

**5 Preparation for Use****CAUTION****Incorrect alignment or assembly**

Injuries due to fall resulting from damaged prosthesis components

- ▶ Observe the alignment and assembly instructions.

**CAUTION****Improper assembly of the screw connections**

Fall due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.

- ▶ Apply the specified installation torque values.
- ▶ Observe the instructions for securing the screw connections and the use of the correct length.

## 5.1 Mounting the adapter

### > Recommended tools:

710D4 torque wrench, 636K13 Loctite®

- 1) **If the pyramid receiver or the pyramid adapter is rotatable:**  
Place the pressure plate on the adapter.
- 2) Place the adapter on the connecting component.
- 3) **Optional:** Align the pyramid receiver or the pyramid adapter.
- 4) Secure the screws with Loctite®.
- 5) Screw in the 2 posterior countersunk head screws and tighten them (mounting tightening torque: **12 Nm**).
- 6) Screw in the 2 anterior countersunk head screws and tighten them (mounting tightening torque: **12 Nm**).

## 5.2 Connecting the pyramid adapter and pyramid receiver

The pyramid adapter is fixed with the set screws of the pyramid receiver.

### > Required tools and materials:

710D4 torque wrench, 636K13 Loctite®

- 1) **Trial fitting:**  
Screw in the set screws.  
Use the torque wrench to tighten the set screws (**10 Nm**).
- 2) **Definitive mounting:**  
Use Loctite® to secure the set screws.  
Screw in the set screws.  
Pre-tighten the set screws with the torque wrench (**10 Nm**) and then tighten them (**15 Nm**).
- 3) Replace any set screws that are protruding or recessed too much with suitable ones (see selection table).

**Selection table for set screws**

Reference number	Length (mm)
506G3=M8X12-V	12
506G3=M8X14	14
506G3=M8X16	16

## Alignment

The set screws in the pyramid receiver can be used to make static adjustments during alignment, trial fittings and after the prosthesis is finished.

## Replacement and disassembly

The set position of the prosthetic component can be maintained during replacement or disassembly. In order to do this, unscrew the two set screws that are screwed in the furthest and located next to each other.

## 6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

## 7 Maintenance

### CAUTION

#### **Failure to follow the maintenance instructions**

Injuries due to changes in or loss of functionality and damage to the product

- ▶ Observe the maintenance instructions.

- ▶ The prosthetic components should be inspected after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

## 8 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

## 9 Legal Information

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

### 9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

## 10 Technical data

<b>Reference number</b>	<b>4R74=AL</b>
<b>Weight [g]</b>	70

<b>Reference number</b>	<b>4R74=AL</b>
<b>System height [mm]</b>	-7
<b>Material</b>	Aluminium
<b>Max. body weight [kg]</b>	136

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Otto Bock HealthCare GmbH