1E57 Lo Rider

DE Gebrauchsanweisung	3
EN Instructions for use	11
FR Instructions d'utilisation	18
IT Istruzioni per l'uso	27
ES Instrucciones de uso	35
PT Manual de utilização	43
NL Gebruiksaanwijzing	51
SV Bruksanvisning	59
DA Brugsanvisning	67
NO Bruksanvisning	75
FI Käyttöohje	82
PL Instrukcja użytkowania	90
HU Használati utasítás	98
CS Návod k použití	106
RO Instrucțiuni de utilizare	113
HR Upute za uporabu	121
SL Navodila za uporabo	129
SK Návod na používanie	137
BG Инструкция за употреба	145
TR Kullanma talimatı	153
EL Οδηγίες χρήσης	161
RU Руководство по применению	169
JA 取扱説明書	178
ZH 使用说明书	186
KO 사용 설명서	192

1

INFORMATION

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INFORMATION

Date of last update: 2020-07-07

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

1.1 Construction and Function

The 1E57 Lo Rider prosthetic foot is suitable for walking on various surfaces and for a wide range of speeds.

The special combination of carbon and polyurethane absorbs heel strike. The prosthetic foot permits perceptible plantar flexion at heel strike and a natural rollover movement. Stored energy is released by the spring elements.

1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

Limited combination options for Ottobock components

- High ankle moments occur on the prosthetic foot. Only use structural components below the knee joint/prosthetic knee joint which are approved for a higher body weight. (**Example:** body weight = 90 kg, structural components approved up to 125 kg)

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Area of application

Our components perform optimally when paired with appropriate components based upon weight and mobility grades identifiable by our MOBIS classification information and which have appropriate modular connectors.

The product is recommended for mobility grade 3 (unrestricted outdoor walker) and mobility grade 4 (unrestricted outdoor walker with particularly high demands).

- The maximum approved body weight is specified in the technical data (see Page 18).

2.3 Environmental conditions

Allowable environmental conditions
Temperature range for use: -10 °C to +60°C
Allowable relative humidity 0 % to 90 %, non-condensing

Unallowable environmental conditions
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids
Dust, sand, highly hygroscopic particles (e.g. talcum)

2.4 Service life

Depending on the patient's level of activity, the service life of the product is 2 to 3 years.

3 Safety

3.1 Explanation of warning symbols

 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

3.2 General safety instructions

 CAUTION
Exceeding the service life and reuse on another patient Risk of injury due to loss of functionality as well as damage to the product <ul style="list-style-type: none">▶ Ensure that the approved service life is not exceeded.▶ Only use the product on a single patient.
 CAUTION
Excessive strain on the product Risk of injury due to breakage of load-bearing components <ul style="list-style-type: none">▶ Use the product according to the specified area of application (see Page 11).

⚠ CAUTION

Unallowable combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved for that purpose.
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

⚠ CAUTION

Using the prosthetic foot without footshell

Risk of injury due to lack of surface grip and loss of functionality of the prosthetic foot

- ▶ Always use the prosthetic foot with a suitable footshell.

NOTICE

Mechanical damage to the product

Change in or loss of functionality due to damage

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

NOTICE

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

Signs of changes in or loss of functionality during use

Decreased forefoot resistance or changes in roll-over behaviour are noticeable indications of loss of functionality.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Prosthetic foot	–
1	Protective sock	SL=SPECTRA-SOCK

Spare parts/accessories (not included in the scope of delivery)

Designation	Reference number
Foot shell with connection cap	2C5*
Connection cap	2C10*
Changing device	2C100

5 Preparing the product for use

CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- Observe the alignment and assembly instructions.

CAUTION

Use of talcum

Risk of injury, damage to the product due to lack of lubrication

- Do not use talcum on the product or other prosthesis components.

NOTICE

Grinding the prosthetic foot or footshell

Premature wear resulting from damage to the product

- Do not grind the prosthetic foot or footshell.

5.1 Alignment

INFORMATION

There is a plastic adapter cover on the proximal connection of the prosthetic foot. This protects the connecting section from scratches during the alignment and trial fitting.

- ▶ Remove the adapter cover before the patient leaves the workshop/fitting area.

5.1.1 Applying/removing the footshell

- > **Recommended tools:** 2C100 changing device
- 1) Slide the prosthetic foot into the footshell.
- 2) Press the heel of the prosthetic foot into the footshell until it engages.
- > **Recommended tools:** 2C100 changing device
- 1) Push the lock of the footshell to the rear and pull up the heel of the prosthetic foot.
- 2) Remove the prosthetic foot from the footshell.

5.1.2 Bench Alignment

TT bench alignment

Bench alignment process	
Required materials: 662M4 goniometer, 743S12 heel height measuring apparatus, 743A80 50:50 gauge, alignment apparatus (e.g. 743L200 L.A.S.A.R. Assembly or 743A200 PROS.A. Assembly)	
Perform the assembly and alignment of the prosthetic components in the alignment apparatus according to the following specifications:	
Sagittal plane	
①	Heel height: Effective heel height (shoe heel height – sole thickness of fore-foot) + 5 mm
②	Exterior foot rotation: approx. 5°
③	a–p positioning, middle of the prosthetic foot to the alignment reference line: positioning according to the table below
④	Connect the prosthetic foot and prosthetic socket using the chosen adapters. The instructions for use of the adapters must be observed.
⑤	Determine the centre of the prosthetic socket with the 50:50 gauge. Align the prosthetic socket centrally to the alignment reference line. Socket flexion: Individual residual limb flexion + 5°
Frontal plane	
⑥	Alignment reference line of prosthetic foot: between big toe and second toe Alignment reference line of prosthetic socket: along the lateral patella edge
⑦	Observe the abduction or adduction position.
a–p positioning, middle of the prosthetic foot	
Foot size [cm]	Mid-foot anterior to the alignment reference line [mm]
22 to 25	10 to 15
26 to 28	15

a-p positioning, middle of the prosthetic foot	
Foot size [cm]	Mid-foot anterior to the alignment reference line [mm]
29 to 31	15 to 20

TF bench alignment

- Observe the information in the prosthetic knee joint instructions for use.

5.1.3 Static Alignment

- Ottobock recommends checking the alignment of the prosthesis using the L.A.S.A.R. Posture and adapting it as needed.
- If necessary, the alignment recommendations (TF modular leg prostheses: **646F219***, TT modular leg prostheses: **646F336***) may be requested from Ottobock.

5.1.4 Dynamic Trial Fitting

- Adapt the alignment of the prosthesis in the frontal plane and the sagittal plane (e.g. by making angle or slide adjustments) to ensure an optimum gait pattern.
- **TT fittings:** Make sure that physiological knee movement in the sagittal and frontal plane is achieved when the leg begins to bear weight after the heel strike. Avoid medial movement of the knee joint. If the knee joint moves in the medial direction in the first half of the stance phase, move the prosthetic foot in the medial direction. If the medial movement occurs in the second half of the stance phase, reduce the exterior rotation of the prosthetic foot.
- Remove the adapter cover from the prosthetic foot after completion of the dynamic trial fitting and gait training exercises.

5.2 Optional: Installing the foam cover

The foam cover sits between the prosthetic socket and prosthetic foot. It is cut longer in order to compensate for the movements of the prosthetic foot and prosthetic knee joint. During flexion of the prosthetic knee joint, the foam cover undergoes posterior compression and anterior elongation. The foam cover should be stretched as little as possible in order to increase its service life. There is a connecting element (such as a connection plate, connection cap or connection cover) on the prosthetic foot.

- > **Required materials:** degreasing cleaner (e.g. 634A58 isopropyl alcohol), 636N9 contact adhesive or 636W17 plastic adhesive
- 1) Measure the length of the foam cover on the prosthesis and add the length allowance.
TT prostheses: Distal allowance for movement of the prosthetic foot.
TF prostheses: Allowance proximal of the knee rotation point for flexion of the prosthetic knee joint and distal allowance for movement of the prosthetic foot.
 - 2) Cut the pre-shaped foam cover to length and fit it in the proximal area on the prosthetic socket.
 - 3) Pull the foam cover over the prosthesis.
 - 4) Set the connecting element onto the footshell or prosthetic foot. Depending on the version, the connecting element engages in the edge or rests on the foot adapter.
 - 5) Install the prosthetic foot on the prosthesis.
 - 6) Mark the outer contour of the connecting element on the distal face of the foam cover.
 - 7) Disassemble the prosthetic foot and remove the connecting element.
 - 8) Clean the connecting element using a degreasing cleaner.
 - 9) Glue the connecting element onto the distal face of the foam cover according to the marked outer contour.
 - 10) Let the glue dry (approx. **10 minutes**).
 - 11) Install the prosthetic foot and adapt the exterior cosmetic shape. Take into account compression caused by cosmetic stockings or SuperSkin.

6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

7 Maintenance

- ▶ A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible

authorities in your country regarding return, collection and disposal procedures.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

10 Technical data

Size [cm]	22	23	24	25	26	27	28	29	30	31
Heel height [mm]	9 ± 5									
System height [mm]	18									
Product weight [g]	240	255	270	280	290	305	320	330	345	355
Max. body weight [kg]	Mobility grade 3: 136 kg (300 lbs), mobility grade 4: 100 kg (220 lbs)									
Mobility grade	3 and 4									

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