
4R98

DE Gebrauchsanweisung	3	CS Návod k použití.....	43
EN Instructions for use	6	RO Instrucțiuni de utilizare.....	46
FR Instructions d'utilisation.....	9	HR Upute za uporabu	50
IT Istruzioni per l'uso	12	SL Navodila za uporabo	53
ES Instrucciones de uso	15	SK Návod na používanie	55
PT Manual de utilização.....	18	BG Инструкция за употреба	59
NL Gebruiksaanwijzing.....	22	TR Kullanma talimatı	62
SV Bruksanvisning	25	EL Οδηγίες χρήσης	65
DA Brugsanvisning	28	RU Руководство по применению	68
NO Bruksanvisning	31	JA 取扱説明書.....	72
FI Käyttöohje.....	34	ZH 使用说明书.....	75
PL Instrukcja użytkownika.....	37	KO 사용 설명서.....	77
HU Használati utasítás	40		

1

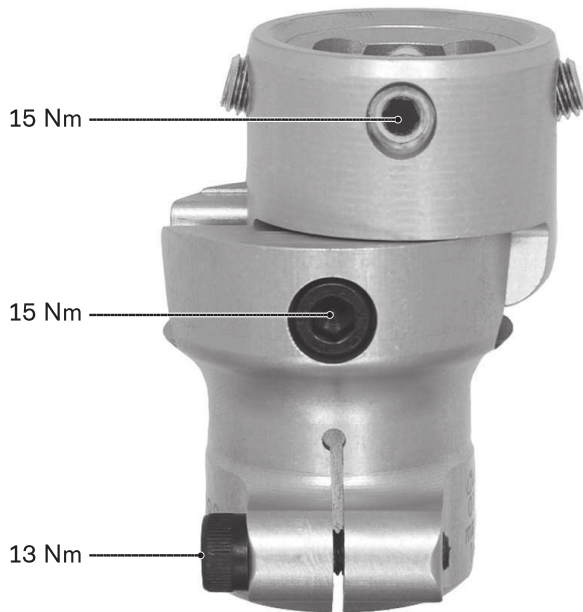1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Date of last update: 2021-06-23

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

1.1 Construction and Function

The 4R98 Sliding Tube Clamp Adapter permits translation adjustments at the proximal end of a tube adapter. It therefore permits the distal components of the modular prosthesis to be shifted in parallel, regardless of the angle adjustment by the pyramid adapters. The adjustments can be made either in the frontal plane (medial or lateral)

or in the sagittal plane (anterior or posterior). This is also possible while wearing the prosthesis.

1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Area of application

Approved for a body weight of up to **75 kg**.

Ottobock recommends the use of structural components with a diameter of 34 mm when elevated loads are expected (e.g. for a transtibial prosthesis).

2.3 Environmental conditions

Storage and transport

Temperature range -20 °C to +60 °C (-4 °F to +140 °F), relative humidity 20 % to 90 %, no mechanical vibrations or impacts

Allowable environmental conditions

Temperature range: -10 °C to +45 °C

Moisture: relative humidity: 20% to 90%, non-condensing

Unacceptable environmental conditions

Chemicals/liquids: fresh water, salt water, perspiration, urine, acids, soapsuds, chlorine water

Solids: dust, sand, highly hygroscopic particles (e. g. talcum)

2.4 Lifetime

This product was tested by the manufacturer with 3 million load cycles. Depending on the user's activity level, this corresponds to a maximum lifetime of 5 years.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION!

Risk of injury and risk of product damage

- ▶ Comply with the product's field of application and do not expose it to excessive strain (see page 6).
- ▶ Note the combination possibilities/comboination exclusions in the instructions for use of the products.
- ▶ Do not expose the product to prohibited environmental conditions.
- ▶ Check the product for damage if it has been exposed to prohibited environmental conditions.
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).
- ▶ Observe the maximum lifetime of the product.
- ▶ To prevent mechanical damage, use caution when working with the product.
- ▶ If you suspect the product is damaged, check it for proper function and readiness for use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for Use	–
1	Sliding Tube Clamp Adapter	–
1	Cap Screw	501Z2=M6x25
1	Cap Screw	501Z2=M6x35
4	Set Screw	506G3=M8x16

5 Preparing the product for use

CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

CAUTION

Improper assembly of the screw connections

Risk of injury due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.
- ▶ Apply the specified torque values.
- ▶ Follow the instructions regarding the length of the screws and about how to secure the screws.

CAUTION

Incorrect mounting of the tube

Risk of injury due to breakage of load-bearing components

- Slide the tube all the way to the stop in the intended prosthetic component when mounting.

5.1 Mounting the adapter

- > **Required materials:** torque wrench (e.g. 710D20), degreasing cleaner (e.g. acetone), 636K13 Loctite 241
- 1) Loosen the tube clamp cap screw by **two turns**.
 - 2) **For final assembly:** Degrease the insertion range of the tube clamp adapter and tube adapter.
 - 3) Insert the tube adapter fully into the tube clamp adapter up to the stop. Align the clamping slot:
Tube clamp adapter: **anterior**
Tube clamp adapter, sliding: **anterior** or **medial**
 - 4) **4WR95=3 only:** Apply Loctite to the tube clamp cap screw.
 - 5) Tighten the cap screw (tightening torque according to thread diameter: M4: **5 Nm**, M5: **10 Nm**, M6: **13 Nm**).

5.2 Installation in the modular prosthesis

- > **Required materials:** torque wrench (e.g. 710D20), 636K13 Loctite 241
- Install the other prosthetic components as described in the respective instructions for use.

5.3 Adjusting adapter

- > **Required tools:**
710D20 Torque Wrench
- > No load on the prosthesis, or the prosthesis is not being worn.
- 1) Loosen the cap screw of the dovetail guide (see fig. 1).
 - 2) Move the proximal section of the sliding tube clamp adapter in the dovetail guide. Note the maximum displacement.
 - 3) Tighten the cap screw (assembly torque: **15 Nm**).

6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

7 Maintenance

- A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- Inspect the entire prosthesis for wear during normal consultations.
- Conduct annual safety inspections.

8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

10 Technical data

Reference number	4R98
Weight [g]	150
System height [mm]	57
Material	Aluminium
Diameter [mm]	30
Max. displacement [mm]	±9
Max. body weight [kg]	75

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