## VS5 Restore

<b>DE</b>	Gebrauchsanweisung .....	5
<b>EN</b>	Instructions for use .....	12
<b>FR</b>	Instructions d'utilisation .....	19
<b>IT</b>	Istruzioni per l'uso .....	27
<b>ES</b>	Instrucciones de uso .....	35
<b>PT</b>	Manual de utilização .....	42
<b>NL</b>	Gebruiksaanwijzing .....	50
<b>SV</b>	Bruksanvisning .....	57
<b>DA</b>	Brugsanvisning .....	64
<b>NO</b>	Bruksanvisning .....	71
<b>FI</b>	Käyttöohje .....	78
<b>PL</b>	Instrukcja użytkowania .....	85
<b>HU</b>	Használati utasítás .....	93
<b>CS</b>	Návod k použití .....	100
<b>RO</b>	Instrucțiuni de utilizare .....	107
<b>HR</b>	Upute za uporabu .....	115
<b>SL</b>	Navodila za uporabo .....	122
<b>SK</b>	Návod na používanie .....	129
<b>BG</b>	Инструкция за употреба .....	136
<b>TR</b>	Kullanma talimatı .....	143
<b>EL</b>	Οδηγίες χρήσης .....	150
<b>RU</b>	Руководство по применению .....	158
<b>JA</b>	取扱説明書 .....	166
<b>ZH</b>	使用说明书 .....	173
<b>KO</b>	사용 설명서 .....	178

## ISO 10328 - P n - m kg\*)


\*) Body mass limit not to be exceeded!

Stiffness	Weight (kg)	Label text
1	59	ISO 10328-P3-59-kg
2	81	ISO 10328-P3-81-kg
3	109	ISO 10328-P4-109-kg
4	136	ISO 10328-P5-136-kg


1

2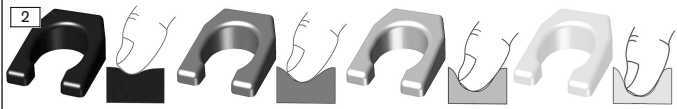

3

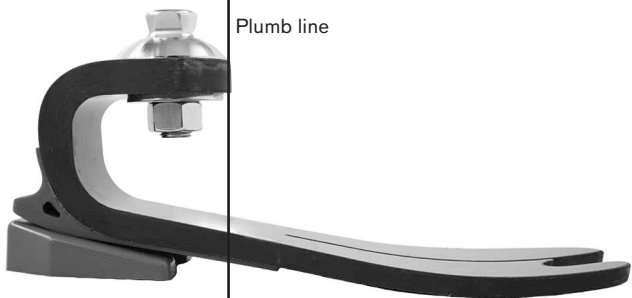Plumb line

## INFORMATION

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### INFORMATION

Date of last update: 2022-06-07

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

### 1.1 Construction and Function

This document applies to the following products: VS5 Restore

The prosthetic foot has fibreglass spring elements. The heel stiffness can be increased with heel wedges.

## 1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

### Not compatible Ottobock components

- 4R21\*, 4R52\*

## 2 Intended use

### 2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

### 2.2 Area of application

The MOBIS classification describes the mobility grade and body weight, and makes it easy to identify compatible components.

The product is recommended for mobility grade 1 (indoor walker) and mobility grade 2 (restricted outdoor walker).

The table below shows the appropriate spring stiffness of the prosthetic foot for the patient's body weight.

Spring stiffness relative to body weight	
Body weight [kg]	Spring stiffness
45 to 59	1
60 to 81	2
82 to 109	3
110 to 136	4

### 2.3 Environmental conditions

Storage and transport
Temperature range $-20\text{ }^{\circ}\text{C}$ to $+60\text{ }^{\circ}\text{C}$ ( $-4\text{ }^{\circ}\text{F}$ to $+140\text{ }^{\circ}\text{F}$ ), relative humidity 20 % to 90 %, no mechanical vibrations or impacts
Allowable environmental conditions
<b>Temperature range:</b> $-10\text{ }^{\circ}\text{C}$ to $+45\text{ }^{\circ}\text{C}$ ( $14\text{ }^{\circ}\text{F}$ to $113\text{ }^{\circ}\text{F}$ )
<b>Chemicals/liquids:</b> fresh water, salt water, perspiration, urine, soapsuds, chlorine water
<b>Moisture:</b> Submersion: max. 1 h in 3 m depth, relative humidity: no restrictions
<b>Solids:</b> Dust, occasional contact with sand
<b>Clean the product after contact with humidity/chemicals/solids, in order to avoid increased wear and damage</b> (see page 18).

<b>Unallowable environmental conditions</b>
<b>Solids:</b> Highly hygroscopic particles (e.g. talcum), continuous contact with sand
<b>Chemicals/liquids:</b> Acids, continuous use in liquid media

## 2.4 Lifetime

### Prosthetic foot

Depending on the patient's activity level, the maximum lifetime of the product is 3 years.

### Footshell, protective sock

The product is a wear part, which means it is subject to normal wear and tear.

## 3 General safety instructions

### Risk of injury and risk of product damage

- ▶ Note the combination possibilities/comboination exclusions in the instructions for use of the products.
- ▶ Comply with the product's field of application and do not expose it to excessive strain (see page 13).
- ▶ To avoid the risk of injury and product damage, do not use the product beyond the tested lifetime.
- ▶ To avoid the risk of injury and product damage, only use the product for a single patient.
- ▶ To prevent mechanical damage, use caution when working with the product.
- ▶ If you suspect the product is damaged, check it for proper function and readiness for use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

## A rectangular box with a black border containing the word "NOTICE!" in bold black capital letters.

### Risk of product damage and limited functionality

- ▶ Check the product for damage and readiness for use prior to each use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

- ▶ Do not expose the product to prohibited environmental conditions.
- ▶ Check the product for damage if it has been exposed to prohibited environmental conditions.
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

### Signs of changes in or loss of functionality during use

Reduced spring effect (e.g. decreased forefoot resistance or changed rollover behaviour) or delamination of the spring are indications of loss of functionality. Unusual noises can indicate a loss of functionality.

## 4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	-
1	Prosthetic foot	-
1	Protective sock (black)	SL=Spectra-Sock-7
1	Heel wedge set	2F13=*

Additional accessories/spare parts (not included in the scope of delivery)	
Designation	Reference number
Footshell (without cap)	FTC-2B-1*
Tool to change the footshell	ACC-00-10300-00

## 5 Preparing the product for use

### CAUTION

#### Incorrect alignment, assembly or adjustment

Injury due to incorrectly installed or adjusted as well as damaged prosthetic components

- ▶ Observe the alignment, assembly and adjustment instructions.

### NOTICE

#### Altering the prosthetic foot or footshell

Premature wear resulting from damage to the product

- ▶ Do not alter the prosthetic foot or footshell.

### INFORMATION

There is a plastic protector on the product's pyramid. It protects the connecting section from scratches during the alignment and fitting of the prosthesis.

- ▶ Remove the protector before the patient leaves the workshop/fitting area.

### INFORMATION

**Use the prosthetic foot only with an installed heel wedge.**

## 5.1 Installing the heel wedge

The behaviour of the prosthetic foot at heel strike and during heel contact in the mid-stance phase can be adapted by replacing the heel wedge. Heel wedges of various hardness are included in the scope of delivery.

**Heel wedge hardness:** The colour of the heel wedge indicates the hardness (see fig. 2). Ottobock recommends starting with the softest heel wedge.

- 1) **If already installed:** Remove the heel wedge from the footshell.
- 2) Insert the desired heel wedge into the footshell. No tools or adhesive are required.

## 5.2 Applying/removing the footshell

### INFORMATION

- ▶ Pull the protective sock over the prosthetic foot to prevent noises in the footshell.
  - ▶ Always use the prosthetic foot with the footshell.
- ▶ Apply or remove the footshell as described in the footshell instructions for use.

## 5.3 Bench Alignment

### TT bench alignment

#### Bench alignment process

**Required materials:** 662M4 goniometer, 743S12 heel height measuring device, 743A80 50:50 gauge, alignment tool (e.g. 743L200 L.A.S.A.R. Assembly or 743A200 PROS.A. Assembly)

Assemble and align the prosthetic components in the alignment apparatus according to the following specifications:

1

Bench alignment process	
	<b>Dorsiflexion/plantar flexion:</b> Adjust the prosthetic foot to the heel height of the shoe. (The prosthetic foot is designed for shoes with a heel height of <b>10 mm.</b> )
②	<b>Adduction/abduction:</b> Adjust the angle of the prosthetic socket in the frontal plane.
③	<b>Flexion/extension:</b> Adjust the angle of the prosthetic socket in the sagittal plane.
④	<b>Linear shifting of the prosthetic socket:</b> Shift the prosthetic socket so that the plumb line runs along the front edge of the adapter on the prosthetic foot (see fig. 3).

## TF bench alignment

- Observe the information in the prosthetic knee joint instructions for use.

### 5.4 Static Alignment

- Ottobock recommends checking the alignment of the prosthesis using the L.A.S.A.R. Posture and adapting it as needed.

### 5.5 Dynamic Trial Fitting

- Adapt the alignment of the prosthesis in the frontal plane and the sagittal plane (e.g. by making angle or slide adjustments) to ensure an optimum gait pattern.
- **TT fittings:** Make sure that physiological knee movement in the sagittal and frontal plane is achieved when the leg begins to bear weight after the heel strike. Avoid medial movement of the knee joint. If the knee joint moves in the medial direction in the first half of the stance phase, move the prosthetic foot in the medial direction. If the medial movement occurs in the second half of the stance phase, reduce the exterior rotation of the prosthetic foot.
- Remove the plastic protector from the pyramid after completing the dynamic fitting and the walking exercises.

Heel too soft	
Symptoms	Possible solutions
<ul style="list-style-type: none"> <li>• Foot makes full ground contact too quickly</li> <li>• Forefoot feels too stiff</li> <li>• Knee goes into hyperextension</li> </ul>	<ul style="list-style-type: none"> <li>• Shift prosthetic socket forward relative to the foot</li> <li>• Install harder heel wedge</li> </ul>

Heel too hard	
Symptoms	Possible solutions
<ul style="list-style-type: none"> <li>• Fast knee flexion, low stability</li> </ul>	<ul style="list-style-type: none"> <li>• Shift prosthetic socket back relative to the foot</li> </ul>

Heel too hard	
<ul style="list-style-type: none"> <li>• Transition from heel strike to toe-off too fast</li> <li>• Energy return feels low</li> </ul>	<ul style="list-style-type: none"> <li>• Install softer heel wedge</li> </ul>

Prosthetic foot too stiff	
Symptoms	Possible solutions
<ul style="list-style-type: none"> <li>• Minimal rollover movement of the prosthetic foot at low walking speed (long full ground contact)</li> </ul>	<ul style="list-style-type: none"> <li>• Select prosthetic foot in lower stiffness</li> </ul>

Prosthetic foot too soft	
Symptoms	Possible solutions
<ul style="list-style-type: none"> <li>• Very pronounced deformation of the forefoot at high activity levels</li> </ul>	<ul style="list-style-type: none"> <li>• Select prosthetic foot in higher stiffness</li> </ul>

## 6 Cleaning

- > **Allowable cleaning agent:** pH neutral soap (e.g. 453H10 Derma Clean)
- 1) **NOTICE! To avoid product damage, only use the allowable cleaning agents.**  
Clean the product with clear fresh water and a pH neutral soap.
  - 2) **If present:** Remove dirt from water drainage contours using a toothpick and rinse.
  - 3) Rinse the soap away with clear fresh water. In doing so, rinse the foot-shell until all dirt has been removed.
  - 4) Dry the product with a soft cloth.
  - 5) Allow to air dry in order to remove residual moisture.

## 7 Maintenance

- ▶ A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Inspect the product for visible damage every six months. Make additional inspection appointments as needed (e.g. for highly active or heavy users).

## 8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible

authorities in your country regarding return, collection and disposal procedures.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

## 10 Technical data

VS5 Restore									
Sizes [cm]	22	23	24	25	26	27	28	29	30
Heel height [mm]	10								
System height [mm]	71			71			81		
Build height [mm]	89			89			99		
Average weight with foot-shell [g]	562			674			801		
Max. body weight [kg]	81	109	136						
Mobility grade	1, 2								

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