

2R67, 4R39, 4R40, 4R85, 4R86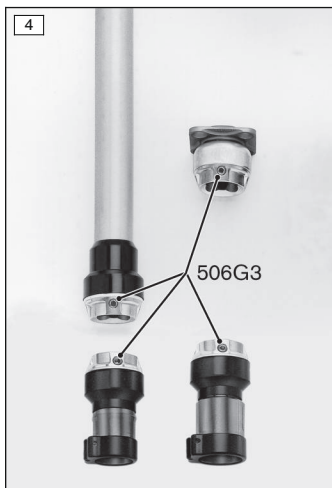

DE Gebrauchsanweisung (Fachpersonal)	3
EN Instructions for use (qualified personnel)	13
FR Instructions d'utilisation (Personnel spécialisé)	22
IT Istruzioni per l'uso (Personale tecnico specializzato)	33
ES Instrucciones de uso (Personal técnico especializado)	43
PT Manual de utilização (Pessoal técnico)	53
NL Gebruiksaanwijzing (Vakmensen)	64
SV Bruksanvisning (Fackpersonal)	74
DA Brugsanvisning (Faguddannet personale)	83
NO Bruksanvisning (Fagpersonell)	93
FI Käyttöohje (Ammattihenkilöstö)	103
PL Instrukcja użytkowania (Personel fachowy)	113
HU Használati utasítás (szakszemélyzet)	123
CS Návod k použití (Odborný personál)	133
SK Návod na používanie (Odborný personál)	142
TR Kullanma talimatı (Uzman personel)	152

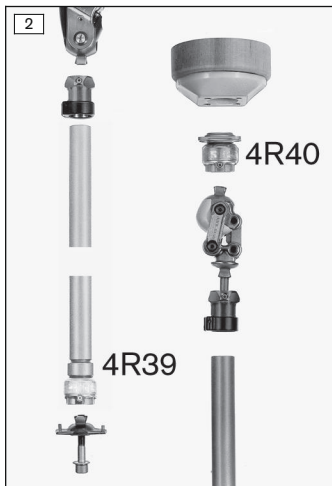INFORMATION

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INFORMATION

Last update: 2015-02-10

- ▶ Please read this document carefully.
- ▶ Follow the safety instructions and the precautions specified in this accompanying document.
- ▶ If assistance is required during the start-up, use or maintenance of the product, as well as in the event of unexpected operating behaviour or circumstances, contact the manufacturer (see manufacturer's address on the back of this accompanying document).

In the following, the torsion adapter is called the product/torsion adapter. These instructions for use provide you with important information on the use, adaptation and handling of the product. Instruct the patient in the proper use and care of the product. The product may not be transferred to the patient without prior instruction. Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product Description

2.1 Construction and Function

The product is used as a component for modular lower limb prostheses. The torsion adapters optimise the function of the modular lower limb prostheses through elastic rotation ($\pm 20^\circ$). Five different versions are available, with the same functionality and housing dimensions, the only difference being the connection:

2R67 torsion adapter with tube, 34 mm Ø

4R39 torsion adapter with tube, 30 mm Ø

4R40 torsion adapter with 4-hole connection

4R85 torsion adapter with tube clamp adapter, 30 mm Ø

4R86 torsion adapter with tube clamp adapter, 34 mm Ø

The torsion movement of the adapters occurs in the horizontal plane. In the spring housing, these movements are transferred to the vertical by a deflection system.

2.2 Combination Possibilities

The product can only be combined with the Ottobock modular lower limb prosthetic components.

3 Application

3.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

3.2 Area of Application

Area of application according to the MOBIS mobility system:

4R85

Recommended for mobility grades **1, 2, 3 and 4** (indoor walker, restricted outdoor walker, unrestricted outdoor walker and unrestricted outdoor walker with particularly high demands).
Approved for a body weight of **up to 100 kg (220 lbs)**.

4R86

Recommended for mobility grades **1, 2, 3 and 4** (indoor walker, restricted outdoor walker, unrestricted outdoor walker and unrestricted outdoor walker with particularly high demands).
Approved for a body weight of **up to 110 kg (242 lbs)**.

2R67, 4R39, 4R40

Recommended for mobility grades **1, 2, 3 and 4** (indoor walker, restricted outdoor walker, unrestricted outdoor walker and unrestricted outdoor walker with particularly high demands).
Approved for a body weight of **up to 125 kg (275 lbs)**.

3.3 Conditions of use

The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, extreme sports (free climbing, parachuting, paragliding, etc.).

4 Safety

4.1 Explanation of Warning Symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

4.2 Structure of the safety instructions

CAUTION

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in case of failure to observe the hazard
- > E.g.: Consequence 2 in case of failure to observe the hazard

- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General Safety Instructions

CAUTION

Non-observance of safety instructions

Falling due to damage or malfunction of the product.

- ▶ Follow the safety instructions and the precautions specified in this accompanying document.

CAUTION

Incorrectly secured screws

Falling due to breakage of load-bearing components caused by screw connections coming loose.

- ▶ After completing all settings, the set screws in the tube adapter must be secured with adhesive before they are tightened to the specified torque.
- ▶ The screws in the clamp bracket must not be secured but only tightened to the specified torque.

CAUTION

Overloading due to unusual activities

Falling due to breakage of load-bearing components.

- ▶ The product was developed for everyday use and must not be used for unusual activities. These unusual activities include, for example, extreme sports (free climbing, paragliding, etc.).
- ▶ Careful handling of the product and its components not only increases their service life but, above all, ensures the patient's personal safety!
- ▶ If the product and its components have been subjected to extreme loads (e.g. due to a fall, etc.), then the product must be inspected for damage immediately. If necessary, forward the product to an authorised Ottobock Service Centre.

CAUTION

Remaining in areas outside the allowable temperature range

Falling due to malfunction or the breakage of load-bearing product components.

- ▶ Avoid remaining in areas with temperatures outside of the permissible range (see Page 22).

⚠ CAUTION

Use of unsuitable prosthetic components

Falling due to unexpected behaviour of the product or breakage of load-bearing components.

- ▶ Use the product only in combination with components approved by Ottobock (see instructions for use and catalogues).

⚠ CAUTION

Improper use

Injury due to faulty operation or malfunction of the product.

- ▶ Instruct the patient in the proper use of the product.

⚠ CAUTION

Penetration of dirt and humidity into the product

- > Falling due to unexpected product behaviour as the result of malfunction.
- > Falling due to breakage of load-bearing components.
- ▶ Ensure that neither solid particles, foreign objects nor liquids penetrate into the product.
- ▶ Should the product come into contact with liquids, allow the product to dry. The product must be inspected by an authorised Ottobock Service Centre.
- ▶ Should the product come into contact with salt water, immediately clean it with a cloth moistened with fresh water and let it dry. The product must be inspected by an authorised Ottobock Service Centre.

⚠ CAUTION

Mechanical overload

Falling due to breakage of load-bearing components.

- ▶ Do not subject the system components to mechanical vibration or impacts.
- ▶ Check the product for visible damage before each use.

⚠ CAUTION

Incorrect setting of the torsion moments in the torsion unit

Falling due to unexpected behaviour of the product.

- ▶ The marking on the Allen head screw may not be turned as far as the red area or beyond the red area (see fig. 1).

5 Scope of Delivery

- 1 pc. 647G23 instructions for use (qualified personnel)
- 1 pc. 2R67/4R39/4R40/4R85/4R86 torsion adapter

6 Preparation for Use

⚠ CAUTION

Incorrect alignment or assembly

Falling due to damage to the prosthesis components.

- ▶ Observe the alignment and assembly instructions.

⚠ CAUTION

Improper assembly of the screw connections

Falling due to breakage or loosening of the screw connections.

- ▶ Clean the threads before every installation.
- ▶ Apply the specified installation torque values.
- ▶ Observe the instructions for securing the screw connections and the use of the correct length.

6.1 Required Tools

- 719R2 tube cutter
- 718R1 tube deburrer
- 710D4 torque wrench (up to 30 Nm)
- 710D12 torque wrench (up to 12 Nm)
- 636K13 adhesive (Loctite® 241)

6.2 Adapting the 2R67, 4R39 torsion adapters

⚠ CAUTION

Improper processing of the tube

Falling due to damage to the tube.

- ▶ Do not clamp the tube into a vice.
- ▶ Only shorten the tube with the prescribed tools.

- 1) Shorten the tube with the tube cutter if needed.
INFORMATION: Make sure that the cut edge is at right angles to the longitudinal axis of the tube.
- 2) Deburr the inside and outside of the cut edge with the tube deburrer.

6.3 Mounting the adapter

6.3.1 Mounting the torsion adapter (2R67, 4R39)

- Fully insert the 2R67, 4R39 tube adapter into the tube clamp adapter up to the stop.

6.3.2 Mounting the torsion adapter (4R40)

- 1) Remove the screws on the socket attachment block or modular adapter.
- 2) Screw the torsion adapter to the socket attachment block or modular adapter.
- 3) Tighten the screws.

6.3.3 Mounting the torsion adapter (4R85/4R86)

INFORMATION

When using the 3R15 and 3R49 knee joints with friction brake, the supplied longer extension assist cable has to be installed.

- 1) On the tube clamp adapter, loosen the tube clamp cap screw by 2 turns.
- 2) Fully insert the tube adapter into the torsion adapter (4R85/4R86) up to the stop.
- 3) Tighten the tube clamp cap screw (installation torque: 5 Nm).

6.4 Connecting the pyramid adapter and pyramid receiver

The pyramid adapter is fixed with the set screws of the pyramid receiver.

Trial fitting:

- Use the torque wrench to tighten the set screws to **10 Nm**.

Definitive mounting:

- 1) Secure the set screws with the adhesive.
- 2) Pre-tighten the set screws with the torque wrench to **10 Nm** and then tighten to **15 Nm**.

6.5 Adjusting the torsion moment

CAUTION

Incorrect setting of the torsion moments in the torsion unit

Falling due to unexpected behaviour of the product.

- The marking on the Allen head screw may not be turned as far as the red area or beyond the red area (see fig. 1).

The torsion moment can be adjusted with the Allen head screw in the centre of the adapter (see fig. 1).

Increasing the torsion moment:

- ▶ Turn the mark in the centre of the torsion unit clockwise.

Decreasing the torsion moment:

- ▶ Turn the mark in the centre of the torsion unit counterclockwise.

7 Handling

7.1 Adjusting the adapter

The set screws in the pyramid receiver can be used to make static adjustments during alignment and trial fittings as well as after the prosthesis is finished.

7.2 Replacement and disassembly

The set position of the prosthetic component can be maintained during replacement or disassembly. In order to do this, unscrew the two set screws that are screwed in the furthest but are not opposite each other.

Replacing set screws

⚠ CAUTION

Incorrectly secured screws

Falling due to breakage of load-bearing components caused by screw connections coming loose.

- ▶ After completing all settings, the set screws in the tube adapter must be secured with adhesive before they are tightened to the specified torque.
- ▶ The screws in the clamp bracket must not be secured but only tightened to the specified torque.

Replace any set screws that are protruding or recessed too much with suitable ones (see selection table).

Selection table for set screws	
Length [mm]	Article number
12 mm	506G3=M8x12-V
14 mm	506G3=M8x14
16 mm	506G3=M8x16

8 Cleaning

- 1) Clean the product with a damp, soft cloth.

- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

9 Maintenance

- ▶ The prosthetic components should be inspected after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

10 Legal Information

10.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

10.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

10.3 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

11 Appendices

11.1 Symbols Used

Declaration of conformity according to the applicable European directives

11.2 Technical data

Environmental conditions	
Storage and transport in original packaging	-10°C/14°F to 60°C/140°F
Storage and transport without packaging	-10°C/14°F to 60°C/140°F Max. 90% relative humidity, non-condensing
Operation	-10°C/14°F to 60°C/140°F Max. 90% relative humidity, non-condensing

Article number	2R67	4R39	4R40	4R85	4R86
Weight [g]	520	500	340	350	340
System height [mm]	Min.: 117 Max.: 322	Min.: 113 Max.: 476	58	68	68
Material	Aluminium/titanium	Stainless steel	Stainless steel	Stainless steel	Titanium
Max. body weight [kg]	125	125	125	100	110
Service life	3 million load cycles (3 – 5 years)				

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Otto Bock Healthcare Products GmbH