757L13, 757L14

DE Gebrauchsanweisung	3
EN Instructions for use	8
FR Instructions d'utilisation	14
IT Istruzioni per l'uso	20
ES Instrucciones de uso	25
PT Manual de utilização	31
NL Gebruiksaanwijzing	37
DA Brugsanvisning	43
PL Instrukcja użytkowania	48
HU Használati utasítás	54
RO Instrucțiuni de utilizare	60
HR Upute za uporabu	66
TR Kullanma talimatı	71
EL Οδηγίες χρήσης	77
RU Руководство по применению	83
JA 取扱説明書	89

1

INFORMATION

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INFORMATION

Date of last update: 2017-03-09

- ▶ Please read this document carefully before using the product.
- ▶ Have yourself instructed by qualified personnel in the proper and safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). The contact information for the manufacturer is found on the back page.
- ▶ Please keep this document in a safe place.

The product "757L13/757L14 pulse charger" is referred to simply as the charger/product below. These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product description

2.1 Function

The product is intended for charging the batteries.

Charging is performed by inserting the batteries into the charging slot. The batteries are charged with a pulse-shaped current. It is reduced to a trickle charge once the voltage reaches a certain upper limit.

During the charging process, the current battery charge level and operating status are indicated by the LEDs on the battery charger (Status signals).

3 Intended use

3.1 Indications for use

The product is to be used **exclusively** for charging batteries.

3.2 Area of application

The product is intended **exclusively** for use with the 757B13 interchangeable battery and 757B15 X-ChangePack.

4 Safety

4.1 Explanation of warning symbols

 WARNING	Warning regarding possible serious risks of accident or injury.
 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

4.2 Structure of the safety instructions

 CAUTION
The heading describes the source and/or the type of hazard
The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:
> E.g.: Consequence 1 in case of failure to observe the hazard
> E.g.: Consequence 2 in case of failure to observe the hazard
▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General safety instructions

 WARNING
Non-observance of safety instructions
Injury due to damage or unexpected product behaviour.
▶ Follow the safety instructions and the precautions specified in this accompanying document.
 WARNING
Use of damaged power supply unit, adapter plug or battery charger
Risk of electric shock due to contact with exposed, live components.

- ▶ Do not open the power supply unit, adapter plug or battery charger.
- ▶ Do not expose the power supply unit, adapter plug or battery charger to extreme loading conditions.
- ▶ Immediately replace damaged power supply units, adapter plugs or battery chargers.

⚠ WARNING

Operating the product near active implanted systems

Interference with active implantable systems (e.g., pacemaker, defibrillator, etc.) due to electromagnetic interference of the product.

- ▶ When operating the product in the immediate vicinity of active implantable systems, ensure that the minimum distances stipulated by the manufacturer of the implant are observed.
- ▶ Make sure to observe any operating conditions and safety instructions stipulated by the manufacturer of the implant.

NOTICE

Independent changes or modifications carried out to the battery charger

Lack of proper charging functionality due to malfunction.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.
- ▶ The product and any damaged components may only be opened and repaired by authorised, qualified Ottobock personnel.

NOTICE

Penetration of dirt and humidity into the product

Lack of proper charging functionality due to malfunction.

- ▶ Ensure that neither solid particles nor liquids can penetrate into the product.

NOTICE

Mechanical stress on the power supply/battery charger

Lack of proper charging functionality due to malfunction.

- ▶ Do not subject the power supply/battery charger to mechanical vibrations or impacts.
- ▶ Check the power supply/battery charger for visible damage before each use.

NOTICE

Operating near sources of strong magnetic or electrical interference (e.g. theft prevention systems, metal detectors)

Lack of proper charging functionality due to malfunction.

- ▶ Avoid operating in the vicinity of visible or concealed theft prevention systems at the entrance/exit of shops, metal detectors/body scanners for persons (e.g. in airports) or other sources of strong magnetic and electrical interference (e.g. high-voltage lines, transmitters, transformer stations, computer tomographs, magnetic resonance tomographs, etc.).

NOTICE

Improper product care

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Clean the product with a damp cloth and mild soap only.

NOTICE

Using the product with soiled or damaged contacts

Lack of proper charging functionality.

- ▶ Make sure that the contacts are always clean and free of grease.
- ▶ Clean the electrical contacts regularly using cotton swabs and a mild soap solution.
- ▶ Take care to avoid damaging the contacts with pointed or sharp objects.

NOTICE

Operating the product outside of the permissible temperature range

Lack of proper charging functionality due to malfunction.

- ▶ Avoid operating in areas outside the permissible temperature range. The section "Technical Data" contains information on the permissible temperature range.

NOTICE

Distance to HF communication devices is too short (e.g. mobile phones, Bluetooth devices, WiFi devices)

Lack of proper charging functionality due to malfunction.

- ▶ Therefore, keeping the following minimum distances to these HF communication devices is recommended:
 - Mobile phone GSM 850 / GSM 900: 0.99 m
 - Mobile phone GSM 1800 / GSM 1900 / UMTS: 0.7 m
 - DECT cordless phones incl. base station: 0.35 m
 - WiFi (routers, access points ...): 0.22 m
 - Bluetooth devices (third-party products not approved by Ottobock): 0.22 m

5 Scope of delivery

- 1 pc. 757L13/757L14 pulse charger
- 1 pc. 757L16* power supply
- 1 pc. 647G260 instructions for use

6 Preparation for use

- 1) Slide the country-specific plug adapter onto the power supply until it locks into place (see fig. 1).
- 2) Connect the round plug of the power supply to the receptacle on the battery charger so that the plug locks into place.

INFORMATION: Ensure correct polarity (guide lug). Do not use force when connecting the cable plug to the battery charger.

- 3) Plug the power supply unit into the wall socket.
→ The green LED on the back of the power supply lights up.

7 Handling

- ▶ Insert one or two batteries into the charging slots on the product.
→ The LED of the respective charging slot lights up.
→ The charging process starts.

After charging the batteries can be taken out of the charging slots. If the battery remains in the charging slot, the battery charger will switch to trickle charging.

INFORMATION

Continuous charging of the batteries for several days adversely affects the life expectancy of the batteries.

8 Disposal

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

9 Legal information

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

9.3 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

The product meets the requirements under the RoHS Directive 2011/65/EU of the European Parliament and of the Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

10 Appendices

10.1 Symbols Used

Declaration of conformity according to the applicable European directives

LOT:PPPP YYYY WW

Lot number

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

Legal manufacturer

For indoor use only

10.2 Operating states/error signals

10.2.1 Status signals

Battery charge level

LED on pulse charger	
	Battery is charging. The on time of the LED indicates the current battery charge level. The on time of the LED gets shorter as the charge level increases (after 4 or 8 hours). It stays on continuously at the start of the charging process and only flashes briefly at the end.
	The battery is empty and being charged.

10.2.2 Warnings/error signals

Error during use

LED on pulse charger	Date	Error	Solution
	Immediately after the battery is inserted in the charging slot	Defective battery	The battery must be inspected by an authorised Ottobock Service Centre.
	After 4 or 8 hours of charging	Defective battery	The battery must be inspected by an authorised Ottobock Service Centre.
	Immediately after the battery is inserted in the charging slot	Defective charging slot/pulse charger or power supply	The pulse charger and power supply must be inspected by an authorised Ottobock Service Centre.

10.3 Technical data

Battery charger	
Storage and transport in original packaging	-20 °C/-4 °F to +70 °C/+158 °F
Storage and transport without packaging	-20 °C/-4 °F to +70 °C/+158 °F Max. 93 % relative humidity, non-condensing
Operation	0 °C/+32 °F to +40 °C/+104 °F Max. 93 % relative humidity, non-condensing

General information	757L13	757L14
Product service life	5 years	

General information	757L13	757L14
Max. charging current	50 mA	70 mA
Charging times (battery fully discharged)	5 h	10 h
Power supply operating voltage	100 V - 240 V	
Power supply operating frequency	50 Hz - 60 Hz	
Weight	220 g	
Dimensions (L x W x H)	130 x 70 x 45 mm	

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Otto Bock Healthcare Products GmbH