

17F31*, 17F32*

DE	Gebrauchsanweisung	4
EN	Instructions for use	11
FR	Instructions d'utilisation	18
IT	Istruzioni per l'uso	25
ES	Instrucciones de uso	33
PT	Manual de utilização	41
NL	Gebruiksaanwijzing	48
SV	Bruksanvisning	56
DA	Brugsanvisning	63
NO	Bruksanvisning	70
PL	Instrukcja użytkowania	76
CS	Návod k použití	84
TR	Kullanma talimatı	91
EL	Οδηγίες χρήσης	98
RU	Руководство по применению	105
ZH	使用说明书	114

1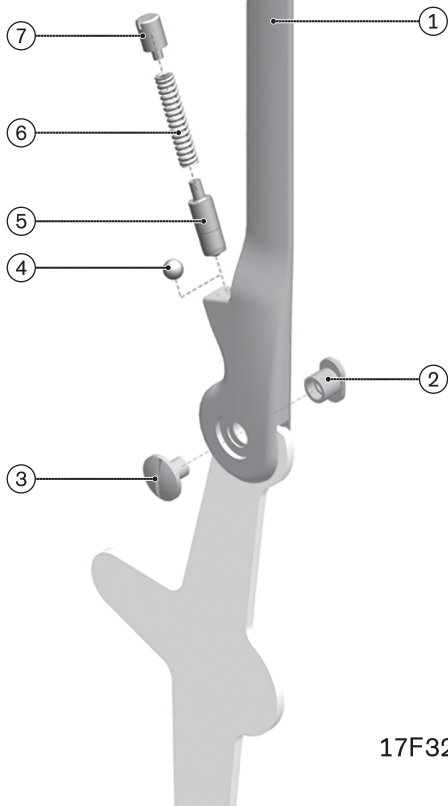

2

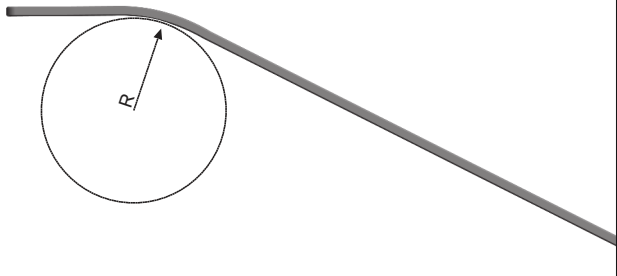INFORMATION

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INFORMATION

Date of last update: 2021-05-07

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the use of the 17F31* and 17F32* orthotic ankle joints.

2 Product description

2.1 Available sizes

Article number	Side	Bar width	Joint head Diameter	Slot width	Material
17F31=L26X2.5	Left	15 mm	26 mm	2.5 mm	Stainless steel
17F31=R26X2.5	Right				
17F31=L28X3	Left	16 mm	28 mm	3 mm	Stainless steel
17F31=R28X3	Right				
17F31=L30X3	Left	17 mm	30 mm	3 mm	Stainless steel
17F31=R30X3	Right				
17F32=L26X2.5	Left	15 mm	26 mm	2.5 mm	Aluminium
17F32=R26X2.5	Right				

Article number	Side	Bar width	Joint head Diameter	Slot width	Material
17F32=L28X3	Left	16 mm	28 mm	3 mm	Aluminium
17F32=R28X3	Right				
17F32=L30X3	Left	17 mm	30 mm	3 mm	Aluminium
17F32=R30X3	Right				

2.2 Components/design

Scope of delivery (see fig. 1) 17F31=L/R26X2.5, 17F32=L/R26X2.5			
Item	Quantity [pcs.]	Component	Reference number
1	1	Joint bar	
2	1	Bearing nut	17Y93*
3	1	Joint screw	501S32*
4	1	Ball	509Y2=3/16"
6	1	Spring	513D18*
7	1	Set screw	17Y18*

Scope of delivery (see fig. 1) 17F31=L/R28X3, 17F31=L/R30X3, 17F32=L/R28X3, 17F32=L/R30X3			
Item	Quantity [pcs.]	Component	Reference number
1	1	Joint bar	
2	1	Bearing nut	17Y93*
3	1	Joint screw	501S32*
5	1	Thrust piece with ball	17Y80
6	1	Spring	513D18*
7	1	Set screw	17Y18*

Accessories

Not included in scope of delivery			
Item	Quantity [pcs.]	Component	Reference number
	1	Foot stirrup	17B58*

Not included in scope of delivery			
Item	Quantity [pcs.]	Component	Reference number
not illustrated	1	Shoe stirrup, double-sided	17F33*
	1	System shoe stirrup	17B65*
	1	System shoe plate	17F35*

3 Intended use

3.1 Indications for use

The orthosis joint is intended **exclusively** for orthotic fittings of the lower limbs on one patient.

3.2 Indications

- Partial or total paralysis of the leg muscles
 - Orthopaedic diseases of the lower limbs
- Indications must be determined by the physician.

3.3 Lifetime

The product is designed for a lifetime of **3 years** when used as intended and assembled professionally.

3.4 Qualification

Patients may be fitted with the product only by trained qualified personnel. The qualified personnel must be familiar with the handling of the various techniques, materials, machines and tools.

4 Safety

4.1 Explanation of warning symbols

 CAUTION	Warning regarding possible risks of accident or injury.
 NOTICE	Warning regarding possible technical damage.

4.2 Safety instructions

 CAUTION
Excessive strain due to use on more than one patient
Risk of injury and loss of functionality as well as damage to the product

- ▶ Use the product on only one patient.
- ▶ Observe the maintenance recommendations.

⚠ CAUTION

Excessive strain on load-bearing components

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.
- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

⚠ CAUTION

Mechanical damage to the product

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

⚠ CAUTION

Use under restricted environmental conditions

Risk of injury due to damage to the product

- ▶ Do not expose the product to restricted environmental conditions.
- ▶ If the product has been exposed to restricted environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

NOTICE

Exposure of the product to unsuitable environmental conditions

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below -10 °C (14 °F) or above +60 °C (140 °F) (e.g. sauna, excessive sunlight, drying on a radiator).

NOTICE

Thermal overloading of the product

Damage due to improper thermal treatment

- ▶ Do not carry out any heat treatment at temperatures above **300 °C (570 °F)**.
- ▶ Prior to thermal treatment, remove all temperature-critical components (such as plastic parts).

5 Preparing the product for use

⚠ CAUTION

Incorrect alignment or assembly

Injuries due to changes in or loss of functionality

- ▶ The product may only be installed by trained, qualified personnel.
- ▶ Observe the alignment and assembly instructions.

⚠ CAUTION

Repeated bending at the same position

Risk of injury due to breakage of the bar, change in or loss of functionality

- ▶ Avoid repeated bending at the same position.

INFORMATION

Parallel alignment of the orthotic joints

Use the 743R6 alignment fixture to position the orthotic joints on the plaster positive.

Shaping the bars

- > **Required tools:** Vice, 2x 711S* bending irons, grinding machine, screwdriver, Loctite 241
- 1) Use the 711S* bending irons for contouring the joint bar.
- 2) **INFORMATION: Do not use bending irons with sharp edges for the 17F32 aluminium joint bar; use only the 711S4 bending irons or bending irons with rounded edges.**
- 3) Contour the bar until the desired result is obtained.
- 4) Observe the minimum bending radius of **10 mm** (see fig. 2).
- 5) Remove grooves and burrs by grinding.
- 6) For final assembly, secure the joint screw with Loctite 241.

Bending the foot stirrups

- > **Required tools:** Vice, 711S* bending iron, grinding machine
- 1) Use the 711S* bending irons for contouring the foot stirrup.
 - 2) Clamp a bending iron in a vice and position the foot stirrup.
 - 3) **INFORMATION: To prevent the risk of breakage, do not bend or otherwise modify the insertion zone.** Adapt the foot stirrup with the bending iron.
 - 4) Remove grooves and burrs by grinding.

Mounting the foot stirrup

- > **Prerequisite:** The orthotic ankle joints are disassembled. To do so, remove the joint screw and the bearing nut from the joint.
- 1) Insert the contoured foot stirrup into the orthotic ankle joint.
 - 2) Assemble the foot stirrup and the orthotic joint with the bearing nut and the joint screw.
 - 3) For final assembly, secure the screw connection with Loctite 241.

Improving corrosion protection

Stainless steel

The 17F31 orthotic joint made of stainless steel has a shiny surface that is protected against corrosion. Sanding, polishing and sintering improves the protection against corrosion.

- ▶ **NOTICE! Do not use polishing pastes containing iron to polish stainless steel.** The micro-fine iron particles are sufficient to initiate corrosion.

Aluminium

The 17F32 orthotic joint is made of aluminium.

- ▶ **NOTICE! Do not carry out any heat treatment at temperatures above 200 °C.**
- ▶ Do not weld! Use a screw or rivet connection.

Ottobock recommends

For sanding:	649G9 belts, free of iron oxide, 280 – 320 grit
For polishing:	Polishing mop coated with Vienna chalk 649Z4 polishing paste, non-ferrous
For surface coating:	618T40 Sintering powder

- ▶ After sanding, polish the surfaces, paying special attention to the edges.

- ▶ Do not heat the material longer than **5 minutes** at max. **150 °C** for surface coating.

6 Cleaning

Promptly clean the product after contact with:

- Water containing salt, chlorine or soap, perspiration, urine or splashed water (e.g. after rain) and soiling.
 - 1) Rinse the product with clear fresh water.
 - 2) Dry the product with a cloth or allow it to air dry. Avoid exposure to direct heat (e.g. from an oven or radiator).

7 Maintenance

INFORMATION

The product may be exposed to increased loads by the patient.

- ▶ Shorten the maintenance intervals according to the expected loads.

The manufacturer requires at least a semi-annual inspection of the product to verify functionality and check for wear.

Only use 633F7 special lubricant.

Spare parts are listed in the section "Components/design".

8 Disposal

Dispose of the product in accordance with national regulations.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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