## 4R76, 4R78

<b>DE</b> Gebrauchsanweisung .....	3	<b>CS</b> Návod k použití.....	39
<b>EN</b> Instructions for use .....	5	<b>RO</b> Instrucțiuni de utilizare.....	41
<b>FR</b> Instructions d'utilisation.....	8	<b>HR</b> Upute za uporabu .....	44
<b>IT</b> Istruzioni per l'uso .....	11	<b>SL</b> Navodila za uporabo .....	47
<b>ES</b> Instrucciones de uso .....	14	<b>SK</b> Návod na používanie .....	49
<b>PT</b> Manual de utilização.....	17	<b>BG</b> Инструкция за употреба .....	52
<b>NL</b> Gebruiksaanwijzing.....	19	<b>TR</b> Kullanma talimatı .....	55
<b>SV</b> Bruksanvisning .....	22	<b>EL</b> Οδηγίες χρήσης .....	57
<b>DA</b> Brugsanvisning .....	25	<b>RU</b> Руководство по применению .....	60
<b>NO</b> Bruksanvisning .....	28	<b>JA</b> 取扱説明書.....	63
<b>FI</b> Käyttöohje.....	30	<b>ZH</b> 使用说明书.....	66
<b>PL</b> Instrukcja użytkowania.....	33	<b>KO</b> 사용 설명서.....	68
<b>HU</b> Használati utasítás .....	36		

1

4R76

2

4R78

## 1 Produktbeschreibung

Deutsch

### INFORMATION

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### INFORMATION

Date of last update: 2021-02-19

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.

► Please keep this document for your records.

### 1.1 Construction and Function

The 4R76, 4R78 double adapters are used to connect two pyramid receivers. The adapters allow for different adjustments due to the shape of the support surfaces.

Reference number	Adjustments
4R76	Horizontal shifting on the flat side, angle adjustment on the rounded side
4R78	Angle adjustment on both sides

### 1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

## 2 Intended use

### 2.1 Indications for use

The product is intended exclusively for lower limb exoprosthesis fittings.

### 2.2 Area of application

Approved for a body weight of **up to 150 kg**.

### 2.3 Environmental conditions

#### Storage and transport

Temperature range  $-20\text{ °C}$  to  $+60\text{ °C}$  ( $-4\text{ °F}$  to  $+140\text{ °F}$ ), relative humidity 20 % to 90 %, no mechanical vibrations or impacts

#### Allowable environmental conditions

**Temperature range:**  $-10\text{ °C}$  to  $+45\text{ °C}$

**Moisture:** relative humidity: 20% to 90%, non-condensing

**Chemicals/liquids:** fresh water as dripping water, occasional contact with salty air (e.g. near the ocean)

#### Allowable environmental conditions

**Solids:** dust

#### Prohibited environmental conditions

**Chemicals/moisture:** salt water, perspiration, urine, acids, soapsuds, chlorine water

**Solids:** dust in high concentrations (e.g. construction site), sand, highly hygroscopic particles (e.g. talcum)

### 2.4 Lifetime

This product was tested by the manufacturer with 3 million load cycles. Depending on the user's activity level, this corresponds to a maximum lifetime of 5 years.

## 3 Safety

### 3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

 **CAUTION!**

#### Risk of injury and risk of product damage

- Note the combination possibilities/combinations exclusions in the instructions for use of the products.
- Comply with the product's field of application and do not expose it to excessive strain (see page 6).
- Observe the maximum lifetime of the product.
- To prevent mechanical damage, use caution when working with the product.

- ▶ If you suspect the product is damaged, check it for proper function and readiness for use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

#### NOTICE

#### Risk of product damage and limited functionality

- ▶ Do not expose the product to prohibited environmental conditions.
- ▶ Check the product for damage if it has been exposed to prohibited environmental conditions.
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

#### Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

#### 4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Adapter	–

#### 5 Preparing the product for use

##### CAUTION

##### Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

#### 5.1 Installation in the modular prosthesis

##### CAUTION

##### Improper assembly of the screw connections

Risk of injury due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.
- ▶ Apply the specified torque values.
- ▶ Follow the instructions regarding the length of the screws and about how to secure the screws.

The pyramid adapter is fixed with the set screws of the pyramid receiver.

> **Required materials:** torque wrench (e.g. 710D20), 636K13 Loc-tite 241

- ▶ Install the other prosthetic components as described in the respective instructions for use.

#### 6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

#### 7 Maintenance

- ▶ The prosthetic components should be inspected after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

#### 8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the

responsible authorities in your country regarding return, collection and disposal procedures.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

### 9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

## 10 Technical data

Reference number	4R76	4R78
Weight [g]	95	115
System height [mm]	-32	-30
Build height [mm]	4	6
Material	Stainless steel	
Max. body weight [kg]	150	

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