## 317L20

<b>DE</b> Gebrauchsanleitung (Benutzer / Techniker) .....	2
<b>EN</b> Instructions for use (user / technician) .....	5
<b>FR</b> Mode d'emploi (Utilisateur / Technicien) .....	8
<b>IT</b> Istruzioni d'uso (utilizzatore / tecnici) .....	11
<b>ES</b> Instrucciones de uso (usuario / técnico) .....	14
<b>PT</b> Manual de instruções (Usuário / Técnico) .....	17
<b>NL</b> Gebruiksaanwijzing (Gebruiker/Technicus) .....	20
<b>SV</b> Bruksanvisning (användare/tekniker) .....	24
<b>DA</b> Brugsvejledning (bruger/tekniker) .....	27
<b>NO</b> Bruksanvisning (bruker/teknikker) .....	30
<b>CS</b> Návod k použití (pro uživatele / techniky) .....	33
<b>RU</b> Руководство по применению (пользователь / техник) .....	36

## INFORMATION

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### INFORMATION

Date of last update: 2022-08-30

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the use of the 317L20 battery charger.

## 2 Product description

### 2.1 Available sizes

The 317L20 battery charger is available in one size and version.

### 2.2 Components/design

Scope of delivery		
Quantity [Piece(s)]	Designation	Article number
1	Battery charger	
1	Power supply unit with EU and US plug	757L16-2
1	Instructions for use	

### 2.3 Technical data

317L20 Battery charger	
Operating voltage	100–240 V
Charging current	Pulsed medium direct charging current, approx. 250 mA, reduction to trickle charging
Weight	190 g
317L20 battery charging time	Approx. 8 to 10 hours when fully discharged
Operating temperature	0 – 40°C
Ambient temperature	-10 – +50°C

## 3 Intended use

### 3.1 Indications for use

The 317L20 battery charger is to be used **exclusively** to supply power to the 317B20 rechargeable battery.

### 3.2 Lifetime

The product is designed for a lifetime of **3 years** when used as intended and assembled professionally.

## 4 Safety

### 4.1 Explanation of warning symbols

### 4.2 Safety instructions

#### CAUTION

##### **Mechanical damage to the product**

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

#### CAUTION

##### **Incorrect alignment, assembly or adjustment**

Risk of injury due to change in or loss of functionality

- ▶ Assembly, adjustment and maintenance operations may only be completed by qualified personnel.
- ▶ Observe the alignment, assembly and adjustment instructions.

#### NOTICE

##### **Exposure of the product to unsuitable environmental conditions**

Damage to electronic components

- ▶ Use the product only in enclosed, dry rooms.
- ▶ Avoid the penetration of liquids and foreign objects.

## 5 Preparing the product for use

#### NOTICE

##### **Charging the battery with soiled or damaged contacts**

Loss of battery functionality due to insufficient charging functionality

- ▶ Ensure that the contacts are always clean and free of grease
- ▶ Clean all electrical contacts of the battery regularly with a lint-free cloth
- ▶ Take care to avoid damaging the contacts with pointed or sharp objects

#### NOTICE

##### **Continuous charging of the battery**

Damage to the battery

- ▶ Avoid uninterrupted charging for several days

### Putting into service

#### INFORMATION

##### **Battery charger malfunction**

The LEDs on the battery charger must not be illuminated prior to inserting the interchangeable battery.

- 1) Connect the power supply unit to the charging station and outlet.
- 2) Insert one or two rechargeable batteries into the charger. **INFORMATION: The LEDs light up to indicate the start of the charging process. The corresponding LED lights up for each battery. The charging time for a discharged battery is 8-10 hours.**

### Operating states

The LEDs indicates the charging and operating state of the rechargeable battery.

Meaning of the LED display		
Signal	Meaning	Recommendation
Continuously lit	The rechargeable battery is charging	Leave the rechargeable battery in the charging station
Even flashing (on/off)	Fast flashing Slow flashing	Continue charging process Rechargeable battery charged, remove it from the charging station

### Troubleshooting

If the rechargeable battery is not being charged, check the following causes of faults.

Meaning of the LED display		
Signal	Cause	Recommendation
Not lit up during putting into service	Battery charger/power supply unit not connected Defective battery charger	Connect power supply unit to battery charger and outlet Send battery charger/power supply unit to the manufacturer
Two rechargeable batteries inserted, only one LED illuminated	Switch the rechargeable batteries! a) The same LED is illuminated: Defective battery charger b) The other LED is illuminated: Defective rechargeable battery	Send battery charger to the manufacturer. Send rechargeable battery to the manufacturer
Flashing immediately after inserting a fully discharged rechargeable battery	Defective battery	Send rechargeable battery to the manufacturer
No flashing after charging more than 8 hours	Defective battery	Send rechargeable battery to the manufacturer
LED is illuminated even though the rechargeable battery has been removed	Defective battery charger	Send battery charger to the manufacturer

## 6 Repair

In the event of defects in the electronic components, the entire component must be replaced with a spare part.

## 7 Disposal

Dispose of the product in accordance with national regulations.

## 8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

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