
ottobock.

8K18, 8K19

DE	Gebrauchsanweisung	3
EN	Instructions for use	8
FR	Instructions d'utilisation	13
IT	Istruzioni per l'uso	19
ES	Instrucciones de uso	24
PT	Manual de utilização	30
NL	Gebruiksaanwijzing	35
SV	Bruksanvisning	41
DA	Brugsanvisning	46
NO	Bruksanvisning	51
PL	Instrukcja użytkowania	56
HU	Használati utasítás	61
CS	Návod k použití	67
RO	Instrucțiuni de utilizare	72
TR	Kullanma talimatı	77
EL	Οδηγίες χρήσης	82
RU	Руководство по применению	88
JA	取扱説明書	94
ZH	使用说明书	98

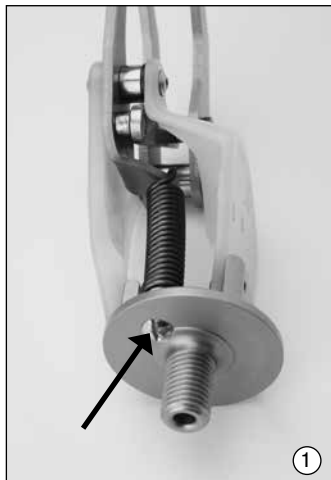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Explanation of symbols

English

⚠ WARNING	Warnings regarding possible risks of severe accident or injury.
------------------	---

⚠ CAUTION	Warnings regarding possible risks of accident or injury.
------------------	--

NOTICE	Warnings regarding possible technical damage.
---------------	---

INFORMATION	Additional information on the fitting / use.
--------------------	--

INFORMATION

Date of last update: 2021-02-18

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- Please keep this document for your records.

The scope of delivery is shown on the front page.

1 Single components

1.1 Single components

For available single components refer to the catalogue.

1.2 Accessories

Select the glove according to the hand size.

- 8S4=* Prosthetic Glove
- 8S5=* Prosthetic Glove
- 8S6=* Prosthetic Glove
- 8S4N=* MovoSkin Natural
- 8S5N=* MovoSkin Natural
- 8S6N=* MovoSkin Natural
- 640F12 Special Cleaner
- 640F13 Pump Sprayer for 640F12

1.3 Accessories for long below-elbow and through-wrist residual limb

- 9S96=40 Chassis (for hand size L/R6¾)
- 9S184=* Chassis (for hand sizes L/R7¼, L/R7¾, and L/R8)
- 506G4=M3x5 Set Screw (for hand size L/R6¾)
- 506G4=M4x5 Set Screw (for hand sizes L/R7¼, L/R7¾, and L/R8)
- 9E94=* Lamination Ring (for hand sizes L/R6¾, L/R7¼, L/R7¾, and L/R8)

The accessories are not included in the delivery. They must be ordered separately.

2 Description

2.1 Indications for use

The Ottobock System Hands –passive– are to be used **solely** for the exoprosthetic fitting of the upper extremities in combination with Ottobock arm components.

2.2 Field of application

The passive System Hands can be used for any residual limb length. The hands' simple functionality places low demands on the user.

2.3 Lifetime

Expected lifetime: 5 years

The design, manufacturing and requirements for the intended use of the product are based on the expected lifetime.

2.4 Safety instructions

Please make your patients aware of the following safety instructions.

WARNING

Risk of accident when driving a motor vehicle. The ability of prosthesis users to drive a vehicle is determined on a case-by-case basis. Factors include the type of fitting (amputation level, unilateral or bilateral, residual limb conditions, design of the prosthesis) and the amputee's abilities.

All persons are required to observe their country's national and state driving laws when operating motor vehicles. For insurance purposes, drivers should have their driving ability examined and approved by an authorized test centre.

For maximum safety and convenience, Ottobock recommends that, at the very least, a specialist evaluate the need for any adaptations to the car (such as by installing a steering fork, automatic shift). Risk-free driving should be ensured even when the prosthesis is not functioning.

CAUTION

Risk of injury as a result of excessive strain. The Ottobock prosthetic components described in these instructions for use have been developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

Careful handling of the prosthesis and its components not only increases their service life but, above all, ensures the patient's safety!

Should prosthetic components be subjected to unusual stresses (such as a fall), immediately contact your prosthetist and have the components inspected for any damage. If necessary, the responsible prosthetist will pass the prosthesis on to the Ottobock Service.

NOTICE

Risk of corrosion. Prosthetic components must not be exposed to corrosive elements such as fresh water, salt water, and acids.

Using this medical product in such environmental conditions will render all claims against Ottobock null and void.

NOTICE

Damage caused by inappropriate environmental conditions. Prosthetic components must not be subjected to intense smoke, dust, vibrations, shocks, or high temperatures. Do not allow debris or liquids to get into the prosthetic components.

Failure to follow these instructions can lead to malfunctioning and damage to the prosthesis.

NOTICE

Damage caused by improper cleaning agents. Do not use any aggressive cleaning agents since they could cause damage to bearings, seals, and plastic parts.

2.5 System Hands –passive– are available in the following designs:

- 8K18=L/R* with metric thread M12x1.5
- 8K19=L/R* with inch thread ½"-20

For the various sizes see table in section 3.

The Ottobock System Hands –passive– perform the three-finger grip with the index, middle finger, and thumb. The passive System Hand is opened with the sound hand at the thumb and closes under spring tension.

The grip force can be adjusted by turning the oval countersunk head screw at the bottom side of the chassis (Fig. 1):

Right → stronger

Left → weaker

3 Technical data

Size	6¾	7¼	7¾	8
Weight [g]	185	250	280	290
Total height [mm]	125	152	155	157
Thumb length [mm]	105	132	135	137

4 Handling

4.1 Assembly

The System Hands –passive– are to be mounted to an Ottobock wrist unit. For long below-elbow and through-wrist residual limbs, the chassis with threaded stud must be replaced by the chassis with 9E94=* Lamination Ring as follows:

- 9S96=40 for hand size L/R6¾
- 9S184=* for hand sizes L/R7¼, L/R7¾, and L/R8

These parts are attached with the 506G4=* Set Screw (Fig. 2).

For this purpose, the bore hole for the spring holder must be drilled into the new chassis (Fig. 1, see arrow).

The 640F18 Donning Spray facilitates the application of the inner hand on the System Hand.

To avoid any damage, use the 711M1 Hand Holding Tool for holding the System Hand while pulling on the glove. The 711M2 Mounting Plate is used for attaching the Hand Holding Tool to a workbench or wall (Fig. 3).

4.2 Combination possibilities

The System Hands –passive– can be combined with the adapters of the field of modular arm prostheses and with the Ottobock wrist units. It may be necessary to select a suitable adapter from the wrist unit accessories.

5 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

6 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

6.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard

of this document, particularly due to improper use or unauthorised modification of the product.

6.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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