10A3, 10A4, 10V15, 11S33

DE	Gebrauchsanweisung	3
EN	Instructions for use	7
FR	Instructions d'utilisation	11
IT	Istruzioni per l'uso	15
ES	Instrucciones de uso	19
PT	Manual de utilização	23
NL	Gebruiksaanwijzing	27
SV	Bruksanvisning	31
DA	Bruksanvisning	35
NO	Bruksanvisning	39
PL	Instrukcja użytkowania	42
HU	Használati utasítás	47
CS	Návod k použití	51
RO	Instrucțiuni de utilizare	55
TR	Kullanma talimatı	59
EL	Οδηγίες χρήσης	63
RU	Руководство по применению	67
JA	取扱説明書	71
ZH	使用说明书	74

INFORMATION

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INFORMATION

Last update: 2020-09-15

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- Please keep this document for your records.

Explanation of symbols

⚠ WARNING Warnings regarding possible risks of severe accident or injury.

⚠ CAUTION Warnings regarding possible risks of accident or injury.

NOTICE Warnings regarding possible technical damage.

INFORMATION Additional information on the fitting / use.

The scope of delivery is shown on the front page.

1 Single components

1.1 Single components

For available single components refer to the catalogue

2 Description

2.1 Indications for use

The 10A3, 10A4, Ottobock work tools, 10V15 Standard Connector and 11S33 Quick-disconnect Wrist Adapter with Standard Connector are to be used **solely** for the fitting of upper limb prostheses.

2.2 Field of application

The Ottobock work tools are suited for light to medium physical stress conditions.

2.3 Safety instructions

Please forward the following safety instructions to your patients:

WARNING

Risk of accident when driving a motor vehicle. The ability of prosthesis users to drive a vehicle is determined on a case-by-case basis. Factors include the type of fitting (amputation level, unilateral or bilateral, residual limb conditions, design of the prosthesis) and the amputee's abilities.

All persons are required to observe their country's national and state driving laws when operating motor vehicles. For insurance purposes, drivers should have their driving ability examined and approved by an authorized test center.

For maximum safety and convenience, Ottobock recommends that, at the very least, a specialist evaluate the need for any adaptations to the car (such as by installing a steering fork, automatic shift). Risk-free driving should be ensured even when the prosthesis is not functioning.

CAUTION

Risk of injury as a result of improper operation. If used incorrectly, the work tools can cause injuries to the user or others.

CAUTION

Risk of injury due to excessive load. Check the connector regularly if subject to high mechanical stress!

⚠ CAUTION

Risk of injury as a result of excessive strain. The Ottobock prosthetic components described in this manual have been developed for everyday life activities and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

Careful handling of the prosthesis and its components not only increases their service life but, above all, ensures the patient's safety!

Should prosthetic components be subjected to unusual stresses (such as a fall), immediately contact your prosthetist and have the components

Should prosthetic components be subjected to unusual stresses (such as a fall), immediately contact your prosthetist and have the components inspected for any damage. If necessary, the responsible prosthetist will pass the prosthesis on to the Ottobock Service.

NOTICE

Risk of corrosion. Please avoid exposing prosthetic components to surroundings that corrode metal parts, for example, freshwater, saltwater, acids and other liquids.

Using this medical product in such environmental conditions will render all claims against Ottobock HealthCare null and void.

NOTICE

Damage due to incorrect environmental conditions. Prosthetic components must not be subjected to intense smoke, dust, vibrations, shocks or high temperatures. Do not allow debris or liquids to get into the prosthesis and its components. Failure to follow these instructions can lead to malfunctioning and damage of the prosthesis.

NOTICE

Damage caused by improper cleaning agents. Do not use any aggressive cleaning agents since they could cause damage to bearings, seals and plastic parts.

2.4 Function

All work tools are provided with standard stud.

The 10V15 Standard Connector is required for the connection to the 10V8 Wrist Units.

The 11S33=* Quick-disconnect Wrist Adapter with Standard Connector is required for adapting to the Ottobock System Wrist Unit. The 11S33=* Wrist Adapter with Standard Connector is available with sizes of \varnothing 40 mm / \varnothing 44 mm and \varnothing 48 mm.

Turn the connector (Fig. 1) to lock the work tool in place by rotationally-stable connection.

3 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

4 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

4.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

4.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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