1E58

DE Gebrauchsanweisung	4
EN Instructions for use	10
FR Instructions d'utilisation	15
IT Istruzioni per l'uso	21
ES Instrucciones de uso	27
PT Manual de utilização	33
NL Gebruiksaanwijzing	39
SV Bruksanvisning	45
DA Brugsanvisning	50
HU Használati utasítás	56
CS Návod k použití	62
RO Instrucțiuni de utilizare	68
EL Οδηγίες χρήσης	74
RU Руководство по применению	80
ZH 使用说明书	86
JA 取扱説明書	91

■	●	▲
DE Einzelteile	Einzelteile-Pack	Mindestmenge
EN Single components	Single-Component Pack	Minimum order quantity
FR Pièces à l'unité	Kit de pièces de rechange	Quantité minimum
IT Singoli componenti	Imballo dei singoli componenti	Quantità minima
ES Componentes	Kit componentes	Cantidad mínima
PT Componentes	Conjunto de peças de reposo	Quantidade mínima
NL Onderdelen	Onderdelenpakket	Minimikvantitet
SV Separata delar	Service-Set	Minimum aantal
DA Komponenter	Komponentpakke	Mindste mængde
NO Enkle komponenter	Enkeltkomponentpakke	Minste ordremenge
PL Podzespoły	Zestaw naprawczy	Ilość minimalna
HU Alkatrészek	Alkatrész-csomag	Minimum mennyiség
CS Jednotlivé díly	Sada jednotlivých dílů	Minimální množství pro objednání
RO Componente individuale	Set componente individuale	Cantitate minimă
HR Pojedinačni dijelovi	Pakiranje pojedinačnih dijelova	Minimalna količina
TR Müfrit parçalar	Müfrit parça ambalajı	Minimum miktar
EL Μεμονωμένα εξαρτήματα	Σετ μεμονωμένων εξαρτημάτων	Ελάχιστη ποσότητα
RU Отдельные детали	Комплект деталей	Минимальное количество
JA 単一成分パック	単一成分パック	最低発注量
ZH 单个部件	维修组件	最低起订量

INFORMATION

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INFORMATION

Date of the last update: 2021-01-05

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- Please keep this document for your records.

Explanation of symbols

CAUTION Warning messages regarding possible risks of accident or injury.

NOTICE Warning messages regarding possible technical damages.

INFORMATION Additional information on the fitting/application.

1 Single components (Fig. 1; for reordering)**NOTICE**

The 1E58 Foot must be used in conjunction with the 2C5 Footshell as well as an Ottobock pylon foot adapter. Both units are purchased separately.

1.1 Single component ■

- (1) **SL=Spectra-Sock-7** protective sock
- (2) **2C100** Tool for Removing the Footshell

1.2 Accessories**INFORMATION**

The accessories are not included. They must be ordered separately.

- (1) **2C5= Footshell** (inclusive cosmesis connection cap)

Order examples: 2C5=R26-28 / 15

Sides	left (L), right (R)									
Sizes (cm)	22	23	24	25	26	27	28	29	30	31
Colours	beige (4), light brown (15)									

- (2) **2C10=Cosmesis Connection Cap** (individual spare part)

Order examples: 2C10=L21-22 / 4

Sides	left (L), right (R)			
Sizes (cm)	21-22	23-25	26-28	29-31
Colours	beige (4), light brown (15)			

(3) Adapter

Various adapters are available for proximal attachment to the modular system.
For available single components please refer to catalogue 646K2.

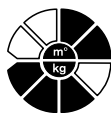
2 Description

2.1 Indications for use

The 1E58 Action DP prosthetic foot is **exclusively** used for lower limb prosthetic fittings.

2.2 Field of application

Our components perform optimally when paired with appropriate components based upon weight and mobility grades identifiable by our MOBIS classification information and which have appropriate modular connectors.

Recommended for amputees with **Mobility Grades 3 and 4** (unrestricted outdoor walkers and unrestricted outdoor walkers with particularly high demands).

Approved for a patient weight of up to 125 kg (275 lbs).

2.3 Safety instructions

Please inform your patients about these safety issues!

⚠ CAUTION

Avoid exposing prosthetic components to excessive impact loading, such as running or jumping activities (e.g. competitive sports) as it may weaken the spring elements. Excessive impact loading, such as running or jumping activities (e.g. competitive sports) may weaken the spring elements. To avoid the risk of accident and possible injury to the patient, the foot must no longer be used after a noticeable loss of function has occurred.

This noticeable function loss may be result of reduced spring effect or forefoot resistance, in some cases accompanied with abnormal noises.

Measure:

Have the prosthetic foot examined and, if necessary, replaced at a specialist workshop.

NOTICE

Please avoid exposing prosthetic components to surroundings that

- cause corrosion on metal parts (for example, freshwater, saltwater, and acids) or
- have an abrasive effect on the carbon fiber (such as sand).
- The footshell material is not UV- or water-resistant.
- Do not grind the surface of this prosthetic foot, as this can lead to premature wear.

Using this medical product under such conditions will render all claims against Otto Bock HealthCare null and void.

2.4 Function

This carbon fiber prosthetic foot has been designed for use in endoskeletal lower limb prostheses for moderately to highly active individuals, and offers high energy return to improve user mobility. This foot is ideal for walking at various speeds and also for use in recreational sports activities.

3 Technical data

Heel height	13 ± 5 mm									
Sizes (cm)	22	23	24	25	26	27	28	29	30	31
Weight* (~g)	357	376	381	389	424	432	443	453	463	473
Body weight up to	125 kg (275 lbs)									
Mobility Grade	3 + 4									
Min. system height A-Pylon	184 mm / 7.25 in									
Max. system height B-Pylon	200 mm / 7.87 in									
Max. system height	368 mm / 14.5 in									

4 Installation instructions

4.1 Initial Set-Up

INFORMATION

Make sure you carefully measure for length, accounting for all components, before cutting pylon length. Final assembly should be conducted only after patient acceptance.

The 1E58 should be used in conjunction with a footshell. However, during the initial fitting, the 1E58 can be used with or without a footshell. If fitting with a footshell, apply Spectra Sock to foot and insert into footshell, then lock heel securely into slot.

- If fitting without footshell, bond included crepe to plantar surface of prosthesis with 636W65=0.800 Contact Adhesive (do not use a Spectra Sock in this case).
- When under load, the Pylon at final heel height should have an anterior lean of 0-2°. Enclosed crepe may be used on plantar surface of heel or forefoot to adjust pylon vertical alignment. Insert into patient's shoe to compensate for heel height.

4.2 Adapter assembly

The Axtion DP is attached to prosthetic systems with either a set screw style adapter (2R183, Fig. 2 or 2R183=L) or side bolt style adapter (4R431=* or 4R432=*, Fig. 3). The Axtion DP is attached to the prosthesis either through an adapter with set screws (2R183, Fig. 2 or 2R183=L) or adapter screwed laterally (4R431=* or 4R432=*, Fig. 3). Please follow the appropriate directions below.

4.2.1 For set screw style adapters (Fig. 2)

Apply silicone grease inside pylon adapter. Next, install pylon adapter onto pylon until fully seated. Tighten the two 8 mm pylon adapter set screws (Fig.2, Pos. B) and torque to 80 in-lbs (9Nm). If using tube clamp adapters, slide tube over pylon adapter and torque the 5 mm tube clamp bolt (Fig.2, Pos. A) to 45 in-lbs (5 Nm).

4.2.2 For side bolt style adapters (Fig. 3)

Apply silicone grease inside pylon adapter. Next, install the 4-hole adapter onto pylon until fully seated. Tighten side bolts to 40 in-lbs (4.5 Nm).

- a. If using 4-hole adapter with threaded fitting (e.g. 4R431=1), install modular adapter (e.g. 4R51) to attachment fitting with four enclosed M6x20mm flat head screws. Torque to 90 in-lbs. (10 Nm).

b. If using 4-hole adapter with unthreaded fitting (e.g. 4R431=2), bolt directly to socket with four (4) enclosed M6x25mm cap screws. Torque screws to appropriate setting for component.

4.3 Footshell application / removal

NOTICE

The foot is designed to be used with footshell. Please Avoid frequent removal from, and insertion into, the footshell, as this can damage the carbon fiber elements.

NOTICE

Do not use screwdriver to insert the carbon foot into the footshell or to remove the footshell from the carbon foot as it can damage the carbon fiber elements and/or foot component.

NOTICE

The footshell must not be dusted by talcum powder as it would increase the abrasion. Use silicone spray if need be. Replace the footshell if it becomes damaged or worn.

4.3.1 To apply the footshell

- Apply supplied Spectra Sock to foot.
- Use the enclosed footshell insertion/removal tool to insert the carbon foot into the footshell. Make sure that the carbon heel of the foot locks in place under the footshell heel tabs.

4.3.2 To remove the footshell

- Insert the insertion/removal tool under the carbon fiber heel in the posterior part of the footshell and pry free from the heel tabs. The foot can now be removed from the footshell.

4.4 Cutting the pylon

Assemble entire prosthesis with all components and calculate amount of pylon to be removed. Remove all components. Square cut pylon with hacksaw to desired length.

4.5 Alignment

4.5.1 Bench alignment

Recommendations on bench alignment (carbon foot with footshell):

Footsize (cm)	Placement of mid-foot anterior to alignment line	Heel height
22 – 25	35–40 mm	13 mm ± 5 mm
26 – 28	40 mm	
29 – 31	40–45 mm	

Normally, walking barefoot during the trial is also tolerated by the construction.

4.5.2 Static alignment optimization

Align the prosthesis on the L.A.S.A.R. Posture, if available. Please use the Ottobock alignment recommendations for the alignment of transtibial and transfemoral amputees.

4.5.3 Dynamic alignment optimization

Align the prosthesis in the coronal (ML) and sagittal (AP) plane by making angular and/or slide changes to ensure the correct heel contact, a smooth rollover and optimal forefoot transfer of

weight onto the contralateral side. For transtibial amputees please make sure that there is physiological knee flexion during stance phase.

4.6 Final assembly

- Remove attachment fitting and wipe off silicone grease. Remove the footshell.
- Abrade top 2 inches (50 mm) of pylon and interior of adapter with sandpaper.
- Clean exterior of pylon and interior of abraded adapter with alcohol.
- Mix Part "A" and Part "B" polyurethane adhesive (included with adapter). Coat internal surface of abraded adapter with adhesive.
- Insert pylon into adapter until fully seated.
Tighten screws to appropriate torque specified below:
Pylon adapter: 8 mm set screws = 9 Nm, secure the screws by applying Loctite® (blue).
- 4-hole adapter: 6 mm side bolts = 4.5 Nm

INFORMATION

Do **not** secure the side bolts of the 4-hole adapter with Loctite®.

Allow polyurethane adhesive to cure for two hours at room temperature.

- Clean pylon, then apply Spectra Sock and insert into footshell.
- Apply Loctite® (blue) to the four 8 mm alignment set screws on the socket adapter and torque to 135 in-lbs (15 Nm).

4.7 Foam cover connection

The Cosmesis Connection Cap 2C10 serves as a detachable connection between the cosmetic foam cover and the footshell. It snaps into the proximal brim of the footshell.

Please observe the following recommendations:

- Cut the foam cover to length, allowing for distal compression about 0.4 in (10 mm) for transtibial and about 1.6 in (40 mm) for transfemoral covers.
- Pull foam cover over the prosthesis, reassemble the foot and mark the outer contours of the connection cap which is attached to the footshell.
- Remove the connection cap and clean with Isopropyl Alcohol 634A58.
- Apply 636N9 Ottobock Contact Adhesive or 636W17 Plastic Adhesive to the distal end of the foam cover and to the connection cap.
- Allow the bonding to dry (10 minutes) and cut out a space in the distal foam cover with a grinding and milling machine to allow for precise fit for the upper part of the adapter. Sufficient space should be available for the adapter to ensure proper function and eliminate noise during walking.
- Assemble the foam cover and finish off the outer shape of the foam cover in usual way with carbon foot installed. Any compression of the foam cover by cosmetic stockings or SuperSkin needs to be taken into account.

4.8 Maintenance instructions

NOTE

- Foot assembly should be inspected after first 30 days (4 weeks) of use.
- Inspect the entire prosthesis for wear during normal consultations.
- Footshell may require replacement if wear is excessive.

5 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

5.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

5.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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