## BionicLink PC 60X5

<b>DE</b> Gebrauchsanweisung (Fachpersonal) .....	3
<b>EN</b> Instructions for use (qualified personnel) .....	10
<b>FR</b> Instructions d'utilisation (Personnel spécialisé) .....	18
<b>IT</b> Istruzioni per l'uso (Personale tecnico specializzato) .....	26
<b>ES</b> Instrucciones de uso (Personal técnico especializado) .....	32
<b>PT</b> Manual de utilização (Pessoal técnico) .....	40
<b>NL</b> Gebruiksaanwijzing (Vakmensen) .....	47
<b>SV</b> Bruksanvisning (Fackpersonal) .....	54
<b>TR</b> Kullanma talimatı (Uzman personel) .....	61
<b>RU</b> Руководство по применению (Квалифицированный персонал) .....	67
<b>JA</b> 取扱説明書 (有資格担当者) .....	75
<b>ZH</b> 使用说明书 (专业人员) .....	81

## INFORMATION

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### INFORMATION

Date of last update: 2020-12-23

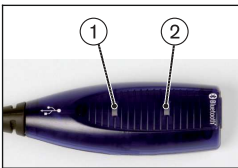
- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The product "60X5 BionicLink PC" is hereinafter referred to as the BionicLink PC or product. These instructions for use provide you with important information on the use, adaptation and handling of the product. Only put the product into use in accordance with the information contained in the accompanying documents supplied.

## 2 Product description

### 2.1 Design

The 60X5 BionicLink PC is equipped with the following light-emitting diodes (LEDs):

1. A green LED to indicate that the adapter is ready for operation
2. A blue LED to indicate an active Bluetooth connection to the Ottobock product

### 2.2 Function

The 60X5 BionicLink PC Bluetooth Adapter enables communication between Ottobock products with an integrated Bluetooth interface or an external Bluetooth interface (e.g., 60X3 BionicLink) and the USB port of a PC.

## 3 Intended use

### 3.1 Indications for use

The 60X5 BionicLink PC Bluetooth Adapter is to be used **solely** for establishing a data connection between the Bluetooth interface of an Ottobock product and the USB port of a PC.

### 3.2 Qualification

The product may only be used by persons who have previously been instructed in its proper use.

## 4 Scope of delivery

- 1 pc. 60X5 BionicLink PC
- 1 pc. Driver CD "USB Drivers"
- 1 pc. Instructions for use (qualified personnel)

## 5 Safety

### 5.1 Explanation of warning symbols

#### NOTICE

Warning regarding possible technical damage.

### 5.2 Structure of the safety instructions

#### NOTICE

#### The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in the event of failure to observe the hazard
- > E.g.: Consequence 2 in the event of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

### 5.3 General safety instructions

#### NOTICE

#### Non-observance of safety notices

Damage to the product due to usage of the product in specific situations.

- ▶ Observe the safety notices and the stated precautions in this accompanying document.

#### NOTICE

#### Connecting the product to a USB connection that is not freely accessible on the computer

In the case of a fault, the power supply of the product cannot be disconnected immediately.

- ▶ Connect the product only to USB connections that can be freely accessed on the computer.

#### NOTICE

#### Use of USB extensions

Limited function due to product malfunction as a result of reduced interference resistance.

- ▶ The use of USB extensions is not recommended since it can lead to interference.

#### NOTICE

#### Use of the product with a damaged USB cable or USB plug

Malfunction due to incorrect or interrupted data transfer.

- ▶ Avoid bending or squeezing the USB cable.
- ▶ Do not make any unauthorised changes to the USB cable or USB plug.

#### NOTICE

#### Independent user changes or modifications made to the product

Limited function due to product malfunction or damage.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.
- ▶ The product and any damaged components may only be opened and repaired by authorised, qualified Ottobock personnel.

#### NOTICE

##### **Penetration of dirt and humidity into the product**

Limited function due to product malfunction or damage.

- ▶ Ensure that neither solid particles nor liquids can penetrate into the product.

#### NOTICE

##### **Mechanical stress on the product**

Limited function due to product malfunction or damage.

- ▶ Do not subject the product to mechanical vibrations or impacts.
- ▶ Check the product for visible damage before each use.

#### NOTICE

##### **Operating the product outside of the permissible temperature range**

Limited function due to product malfunction or damage.

- ▶ Avoid operation in areas outside the permissible temperature range (see page 17).

#### NOTICE

##### **Improper product care**

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Only clean the product with a damp cloth and mild soap (e.g. 453H10=1-N Ottobock DermaClean).

#### NOTICE

##### **Too close to HF communication devices (e.g. mobile phones, Bluetooth devices, WiFi devices)**

Limited function due to product malfunction.

- ▶ Therefore, keeping the following minimum distances to these HF communication devices is recommended:
  - Mobile phone GSM 850 / GSM 900: 0.99 m
  - Mobile phone GSM 1800 / GSM 1900 / UMTS: 0.7 m
  - DECT cordless phones incl. base station: 0.35 m
  - WiFi (routers, access points ...): 0.22 m
  - Bluetooth devices (third-party products not approved by Ottobock): 0.22 m

#### NOTICE

##### **Operating the product very close to other electronic devices**

Limited function due to product malfunction.

- ▶ Do not stack the product during operation nor operate it in the immediate vicinity of other electronic devices in operation.
- ▶ If simultaneous operation cannot be avoided, monitor the product and verify proper use in the existing setup.

## NOTICE

### Use of a damaged product

Limited functionality due to product malfunction.

- ▶ Do not put the product into operation if the cable or housing is visibly damaged. If you notice any damage, please send the device to an authorised Ottobock service centre.
- ▶ Check the functions of the two light-emitting diodes on the product regularly. Should they fail, please send the product to an authorised Ottobock service centre.

## INFORMATION

### Cybersecurity

- ▶ Keep your operating system up to date and always install any available security updates.
- ▶ Protect your computer from unauthorised access (e.g., by using virus scans, password protection etc.).
- ▶ Do not use unsecured networks.
- ▶ Please contact the manufacturer if you suspect cybersecurity problems.

## 6 System Requirements

### 6.1 PC hardware requirements

## INFORMATION

Please also note the hardware requirements for the Ottobock software that is in use.

- At least 5 GB free hard drive space
- CD-ROM drive or DVD-ROM drive
- USB connection 2.0 or higher

### 6.2 Supported operating systems

## INFORMATION

Please also note the operating systems supported by the Ottobock software.

#### The following 32-bit operating systems are supported:

- Microsoft Windows 7, SP1 or higher, all editions except Microsoft Windows 7 Starter
- Microsoft Windows 8
- Microsoft Windows 10

#### The following 64-bit operating systems are supported:

- Microsoft Windows 7, SP1 or higher, all editions except Microsoft Windows 7 Starter
- Microsoft Windows 8
- Microsoft Windows 10

## INFORMATION

Please note that the latest USB driver version must always be installed for the Bluetooth connection via the 60X5 BionicLink PC. You can download the latest driver version from the following site: <http://ottobock.com/en/datastation>.

## 7 Driver software installation

### 7.1 Installation in Windows 7

Close all open applications.

- 1) Insert the enclosed CD into the CD or DVD drive.

- 2) In the dialogue window, select the option **Run Setup.exe**.
- 3) Follow the next instructions of the installation wizard.  
**INFORMATION: If the information window that references a driver software publisher that cannot be verified appears, the installation can still be continued regardless. To do so, click the corresponding option in the dialogue window.**
- 4) Follow the next instructions of the installation wizard.
- 5) When the dialogue window informing you that the driver has been installed successfully appears, click the **Finish** button.
- 6) Connect the BionicLink PC plug to the USB socket on the computer.  
 → The green LED on the 60X5 BionicLink PC is illuminated.

## 7.2 Installation in Windows 8

Close all open applications.

- 1) Insert the enclosed CD into the CD or DVD drive.
- 2) Open "Computer" with the key combination [Windows logo key +E].
- 3) Right-click the CD drive in the right-handwindow.
- 4) Left-click the menu item "**Install or run program**".  
 → Follow the next instructions of the installation wizard.  
**INFORMATION: When the dialogue window stating that "Otto Bock Healthcare Products GmbH" is the publisher of the software appears, continue the installation by clicking the corresponding button.**
- 1) Follow the next instructions of the installation wizard.
- 2) When the dialogue window informing you that the driver has been installed successfully appears, click the **Finish** button.
- 3) Connect the BionicLink PC plug to the USB socket on the computer.  
 → The green LED on the 60X5 BionicLink PC is illuminated.

## 7.3 Installation in Windows 10

Close all open applications.

- 1) Insert the enclosed CD into the CD or DVD drive.
- 2) Open "Computer" with the key combination [Windows logo key +E].
- 3) Right-click the CD drive in the leftwindow.
- 4) Left-click the menu item "**Install or run program**".  
 → Follow the next instructions of the installation wizard.  
**INFORMATION: When the dialogue window stating that "Otto Bock Healthcare Products GmbH" is the publisher of the software appears, continue the installation by clicking the corresponding button.**
- 1) Follow the next instructions of the installation wizard.
- 2) When the dialogue window informing you that the driver has been installed successfully appears, click the **Finish** button.
- 3) Connect the BionicLink PC plug to the USB socket on the computer.  
 → The green LED on the 60X5 BionicLink PC is illuminated.

## 8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for

damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

## 8.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

Microsoft, Outlook and Windows are registered trademarks of Microsoft Corporation.

Pentium, Intel and Intel Core are registered trademarks of Intel Corporation.

## 8.3 CE conformity

Otto Bock Healthcare Products GmbH hereby declares that the product is in compliance with applicable European ergänzen!European requirements for medical devices.

The product meets the requirements of the RoHS Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic devices.

This product meets the requirements of the 2014/53/EU directive.

The full text of the regulations and requirements is available at the following Internet address: <http://www.ottobock.com/conformity>

## 8.4 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use in this chapter.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This device may not cause harmful interference, and
- 2) This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/ TV technician for help.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

### **Caution: Exposure to Radio Frequency Radiation.**

This device must not be co-located or operating in conjunction with any other antenna or transmitter.

Responsible party:  
 Otto Bock Health Care, LP  
 3820 West Great Lakes Drive  
 Salt Lake City, Utah 84120-7205 USA  
 Phone + 1-801-956-2400  
 Fax + 1-801-956-2401

This device complies with RSS 210 of Industry Canada.

Operation is subject to the following two conditions:

- (1) this device may not cause interference, and
- (2) this device must accept any interference, including interference that may cause undesired operation of this device.

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**Caution: Exposure to Radio Frequency Radiation.**

The installer of this radio equipment must ensure that the antenna is located or pointed such that it does not emit RF field in excess of Health Canada limits for the general population; consult Safety Code 6, obtainable from Health Canada's website

<http://www.hc-sc.gc.ca/rpb>.

Responsible party:  
 Otto Bock Healthcare Canada Ltd.  
 5470 Harvester Road  
 L7L 5N5 Burlington, Ontario  
 Canada  
 Phone + 1-800-665-3327

---

**Caution: Federal law (USA) restricts this device to sale by or on the order of a practitioner licensed by law of the State in which he/she practices to use or order the use of the device.**

## 9 Troubleshooting

Verify the operational readiness of the 60X5 BionicLink PC using the table below. Eliminate any possible causes.

Event	Cause/required action
The green LED on the 60X5 BionicLink PC is not lit up	The 60X5 BionicLink PC is not operational <ul style="list-style-type: none"> <li>• Check the connection of the BionicLink on the PC.</li> <li>• Connect the BionicLink to a different USB port on the PC.</li> <li>• If a hub is being used, connect the BionicLink directly to the USB port on the PC.</li> </ul>
The blue LED on the 60X5 BionicLink PC is not lit up	Connection to component cannot be established or has been interrupted <ul style="list-style-type: none"> <li>• Reduce the distance of the component to the 60X5 BionicLink PC.</li> <li>• Charge level of the component.</li> </ul>

## 10 Technical data

<b>Ambient conditions</b>	
Storage and transport in original packaging	-25 °C/-13 °F to +70 °C/+158 °F
Storage and transport without packaging	-25 °C / -13°F to +70°C / +158°F max. 93% relative humidity, non-condensing
Operation	±0 °C/+32 °F to +60 °C/+140 °F max. 93% relative humidity, non-condensing

<b>General information</b>	
Reference number	60X5 BionicLink PC
Product service life	5 years
Input voltage (power supply via the USB connector)	5 V $\overline{=}$

<b>Data transfer</b>	
Wireless technology	Bluetooth v.2.1 + EDR
Bluetooth class	Class 1
Range	approx. 10 m / 32.8 ft
Frequency range	2402 MHz to 2480 MHz
Modulation	GFSK, $\pi/4$ DQPSK, 8DPSK
Data rate (over the air)	up to 3Mbps
Maximum output power (EIRP):	+19.9 dBm (~97.8 mW)

## 11 Appendix

### 11.1 Symbols Used

Declaration of conformity according to the applicable European directives

Compliance with the requirements according to "FCC Part 15" (USA)

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

Compliance with the requirements under the "Radiocommunications Act" (AUS)

Lot number (PPPP YYYY WW)  
PPPP – plant  
YYYY – year of manufacture  
WW – week of manufacture

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