ottobock.

CE 0681

60X6

DE Gebrauchsanweisung (Fachpersonal)	3
EN Instructions for use (qualified personnel)	12
FR Instructions d'utilisation (Personnel spécialisé)	23
IT Istruzioni per l'uso (Personale tecnico specializzato)	35
ES Instrucciones de uso (Personal técnico especializado) ..	44
PT Manual de utilização (Pessoal técnico)	54
NL Gebruiksaanwijzing (Vakmensen)	64
SV Bruksanvisning (Fackpersonal)	73

INFORMATION

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INFORMATION

Date of last update: 2021-03-09

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The product "60X6 MyolinoLink" is referred to as the product below.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product description

2.1 Function

The product allows wireless data communication between Ottobock products and a PC with Bluetooth connection.

Patient-specific adjustments are therefore possible without wire connections, and the patient has more freedom of movement during adjustment.

2.2 Combination possibilities

- 60X5 BionicLink PC

3 Intended use

3.1 Indications for use

The product is to be used exclusively for Bluetooth data communication for adjusting electronic components of upper limb prostheses.

3.2 Conditions of use

The fitting of a patient with the product may only be carried out by an orthopaedic technician.

3.3 Contraindications

- All conditions which contradict or go beyond the specifications listed in the section on "Safety" and "Indications for use".

3.4 Qualification

The product may only be used by persons who have previously been instructed in its proper use.

4 Safety

4.1 Explanation of warning symbols

Warning regarding possible serious risks of accident or injury.

Warning regarding possible technical damage.

4.2 Structure of the safety instructions

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in the event of failure to observe the hazard
- > E.g.: Consequence 2 in the event of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General safety instructions

Non-observance of safety notices

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety notices and the stated precautions in this accompanying document.

Fitting of children

Injury due to swallowing small parts.

- ▶ Never leave children unattended during the fitting.

NOTICE**Operating the product outside of the permissible temperature range**

Limited function due to product malfunction or damage.

- ▶ Avoid operation in areas outside the permissible temperature range (see page 21).

NOTICE**Independent user changes or modifications made to the product**

Limited function due to product malfunction or damage.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.
- ▶ The product and any damaged components may only be opened and repaired by authorised, qualified Ottobock personnel.

NOTICE**Penetration of dirt and humidity into the product**

Limited function due to product malfunction or damage.

- ▶ Ensure that neither solid particles nor liquids can penetrate into the product.

NOTICE**Mechanical stress on the product**

Limited function due to product malfunction or damage.

- ▶ Do not subject the product to mechanical vibrations or impacts.
- ▶ Check the product for visible damage before each use.

NOTICE**Using the product with soiled or damaged contacts**

Limited function due to product malfunction or damage.

- ▶ Make sure that the contacts are always clean and free of grease.
- ▶ Clean the electrical contacts regularly using cotton swabs and a mild soap solution.
- ▶ Take care to avoid damaging the contacts with pointed or sharp objects.

NOTICE

Product contact with magnetic data storage devices

Wiping of the data storage device.

- ▶ Do not place the product on credit cards, diskettes, audio or video cassettes.

NOTICE

Proximity to sources of strong magnetic or electrical interference (e.g. theft prevention systems, metal detectors)

Limited function due to product malfunction.

- ▶ Avoid remaining in the vicinity of sources of strong magnetic and electrical interference (e.g. high-voltage lines, transmitters, transformer stations, computer tomographs, magnetic resonance tomographs, etc.).
- ▶ Use the product only in orthopaedic workshops, or during in-patient stays in rehabilitation facilities.

NOTICE

Too close to HF communication devices (e.g. mobile phones, Bluetooth devices, WiFi devices)

Limited function due to product malfunction.

- ▶ Therefore, keeping the following minimum distances to these HF communication devices is recommended:
 - Mobile phone GSM 850 / GSM 900: 0.99 m
 - Mobile phone GSM 1800 / GSM 1900 / UMTS: 0.7 m
 - DECT cordless phones incl. base station: 0.35 m
 - WiFi (routers, access points ...): 0.22 m
 - Bluetooth devices (third-party products not approved by Ottobock): 0.22 m

INFORMATION

Cybersecurity

- ▶ Keep your operating system up to date and always install any available security updates.
- ▶ Protect your computer from unauthorised access (e.g., by using virus scans, password protection etc.).
- ▶ Do not use unsecured networks.
- ▶ Please contact the manufacturer if you suspect cybersecurity problems.

5 Scope of Delivery and Accessories

5.1 Scope of delivery

- 1 pc. 60X6 MyolinoLink
- 1 pc. Instructions for use (qualified personnel)

5.2 Accessories

- "560X3=V1.0 MyolinoSoft" or higher adjustment software
- 60X5 BionicLink PC
- 757B35=1 / 757B35=3 / 757B35=5 MyoEnergy Integral

6 Preparing the product for use

- 1) Connect the product to the charging receptacle of the "MyoEnergy Integral" battery.
 - The correct connection of the battery to the product is indicated by feedback (see page 23).

- 2) Establish the connection between the product and the PC.
INFORMATION: For details on establishing the connection, please see the instructions for use included with the 60X5 BionicLink PC.
- 3) Start the 560X3=* software.
- 4) Enter the Bluetooth ID in the software interface.
INFORMATION: The number is found on the face side of the housing (e.g.: BT D 2010014).
 - A correctly established connection is indicated by feedback (see page 23).

7 Use

INFORMATION

Secure data transfer is only guaranteed up to a distance of 5 m between the PC and the product.

- 1) The adjustments can be made.
 - The data transfer is indicated by feedback (see page 23).
- 2) After making adjustments, disconnect the product from the charging receptacle of the MyoEnergy Integral.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

8.3 CE conformity

Otto Bock Healthcare Products GmbH hereby declares that the product is in compliance with applicable European requirements for medical devices.

The product meets the requirements of the RoHS Directive 2011/65/EU on the restriction of the use of certain hazardous substances in electrical and electronic devices.

This product meets the requirements of the 2014/53/EU directive. The full text of the regulations and requirements is available at the following Internet address: <http://www.ottobock.com/conformity>

8.4 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use in this chapter.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This device may not cause harmful interference, and
- 2) This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no

guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/ TV technician for help.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Caution: Exposure to Radio Frequency Radiation.

This device must not be co-located or operating in conjunction with any other antenna or transmitter.

Responsible party:

Otto Bock Health Care, LP
3820 West Great Lakes Drive
Salt Lake City, Utah 84120-7205 USA
Phone + 1-801-956-2400
Fax + 1-801-956-2401

This device complies with RSS 210 of Industry Canada.

Operation is subject to the following two conditions:

- (1) this device may not cause interference, and
- (2) this device must accept any interference, including interference that may cause undesired operation of this device.

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Caution: Exposure to Radio Frequency Radiation.

The installer of this radio equipment must ensure that the antenna is located or pointed such that it does not emit RF field in excess of Health Canada limits for the general population; consult Safety Code 6, obtainable from Health Canada's website <http://www.hc-sc.gc.ca/rpb>.

Responsible party:

Otto Bock Healthcare Canada Ltd.

5470 Harvester Road

L7L 5N5 Burlington, Ontario

Canada

Phone + 1-800-665-3327

Caution: Federal law (USA) restricts this device to sale by or on the order of a practitioner licensed by law of the State in which he/she practices to use or order the use of the device.

9 Technical data

Ambient conditions	
Storage (with and without packaging)	+5 °C/+41 °F to +40 °C/+104 °F Max. 85% relative humidity, non-condensing
Transport (with and without packaging)	-20 °C/-4 °F to +60 °C/+140 °F Max. 90% relative humidity, non-condensing
Operation	-5 °C/+23 °F to +45 °C/+113 °F Max. 95% relative humidity, non-condensing

General	
Reference number	60X6 MyolinoLink
Product service life	5 years
Bluetooth connection range	approx. 5 m The actual range depends on the Ottobock product with which the Bluetooth connection is to be established.
Bluetooth class	Class 2
Power supply	757B35=* MyoEnergy Integral

10 Appendices

10.1 Symbols Used

Manufacturer

Lot number (PPPP YYYY WW)

PPPP – plant

YYYY – year of manufacture

WW – week of manufacture

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

Declaration of conformity according to the applicable European directives

10.2 Operating States

10.2.1 Status signals

LED on product	Event
 LED illuminated	Connection successfully established
 LED flashing	Data communication taking place

10.2.2 Beep signals

The following beep signals are set by default:

Beep signal	Event
2 x short	Connection to MyoEnergy Integral established

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This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

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The installer of this radio equipment must ensure that the antenna is located or pointed such that it does not emit RF field in excess of Health Canada limits for the general population; consult Safety Code 6, obtainable from Health Canada's website

<http://www.hc-sc.gc.ca/rpb>.

Responsible party:

Otto Bock Healthcare Canada Ltd.
5470 Harvester Road

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