8S500=\* AxonSkin Visual + Black,  
8S501=\*, 8S502=\* AxonSkin Natural

<b>DE</b> Gebrauchsanweisung (Fachpersonal) .....	2
<b>EN</b> Instructions for use (qualified personnel) .....	14
<b>FR</b> Instructions d'utilisation (Personnel spécialisé) .....	27
<b>IT</b> Istruzioni per l'uso (Personale tecnico specializzato) .....	39
<b>ES</b> Instrucciones de uso (Personal técnico especializado) .....	52
<b>PT</b> Manual de utilização (Pessoal técnico) .....	64
<b>NL</b> Gebruiksaanwijzing (Vakmensen) .....	76
<b>SV</b> Bruksanvisning (Fackpersonal) .....	89
<b>DA</b> Brugsanvisning (Faguddannet personale) .....	101
<b>NO</b> Bruksanvisning (Fagpersonell) .....	113
<b>FI</b> Käyttöohje (Ammattihenkilöstö) .....	125
<b>JA</b> 取扱説明書 (有資格担当者) .....	137

## INFORMATION

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### INFORMATION

Date of last update: 2021-05-25

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

### 1.1 Construction and Function

The AxonSkin prosthetic glove is used as a protective glove for a prosthetic hand. The PVC material is characterised by robustness and durability.

#### 8S501=\*, 8S502=\* AxonSkin Natural

The 8S501=\* AxonSkin Natural is intended for men, the 8S502=\* AxonSkin Natural is intended for women.

Its appearance is based on a real hand in order to achieve a high degree of naturalness.

The prosthetic glove is coloured by hand, giving rise to possible variations in appearance, which are not a product defect.

The fingernails can be painted with nail varnish.

### **8S500=\*-M0 AxonSkin Visual, 8S500=\*-M20 AxonSkin Black**

The 8S500=\*-M0 AxonSkin Visual is translucent, making the prosthetic hand visible. The 8S500=\*-M20 AxonSkin Black is completely black. Both have specific zones on the fingertips and palms, which improve grip.

## **1.2 Combination possibilities**

<b>Designation</b>	<b>Reference Number</b>
Prosthetic hand	8E500=*/8E550=* Michelangelo Hand

## **2 Intended use**

### **2.1 Indications for use**

The product may only be used as part of an upper limb prosthesis with Michelangelo Hand.

### **2.2 Area of application**

The product has been developed for use in everyday activities. It must not be used for activities where it would be exposed to unallowable environmental conditions or strains (e.g. extreme sports).

### **2.3 Environmental conditions**

<b>Allowable environmental conditions</b>
Allowable operating temperature: <b>-10 °C to +60 °C (+14 °F to +140 °F)</b>
Allowable storage temperature: <b>-20 °C to +40 °C (-4 °F to +104 °F)</b>
Allowable transport temperature: <b>-20 °C to +60 °C (-4 °F to +140 °F)</b>
Allowable relative humidity: <b>max. 80%</b>
Cleaning agents: warm water and soap, 640F12 Special Cleaner, nail varnish remover (acetone-free)

<b>Unallowable environmental conditions</b>
Mechanical vibrations or impacts, touching of sharp-edged or pointed objects
Contact with acids, unallowable cleaning agents, abrasive media (e.g. sand), damp soil, dust, dirt, highly staining materials (e.g. printing ink, new textiles), solvents (e.g. thinner, acetone, petrol), fire, embers and unallowable heat ( <b>&gt; +60 °C / &gt; +140 °F</b> )

### **2.4 Lifetime**

The product is a wear part, which means it is subject to natural wear and tear. The lifetime diminishes or increases depending on the individual level of use. Only when the instructions for use are observed it is possible to achieve the maximal lifetime, in dependence of the level of use.

## **3 Safety**

### **3.1 Explanation of warning symbols**

 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

### 3.2 General safety instructions

#### CAUTION

##### **Using the product without following the instructions for use**

Injuries caused by damage to the product or malfunction of the product

- ▶ Only use the product by following the instructions for use.
- ▶ Make the patient aware of all safety instructions marked with "**Inform the patient**".

#### CAUTION

##### **Contact with heat, embers or fire**

Risk of injury (e.g. burns) and damage to the product due to unallowable environmental conditions

- ▶ Keep the product away from open flames, embers and other sources of heat.

#### NOTICE

##### **Unallowable combination of prosthetic components**

Product damage due to malfunctions

- ▶ Only combine the product with prosthetic components that are approved according to the section "Combination Possibilities".
- ▶ Verify that the prosthetic components may be combined with each other by consulting their instructions for use.

#### NOTICE

##### **Non-observance of allowable environmental conditions**

Damage to product and soiling due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental conditions").
- ▶ If the product has been exposed to unallowable environmental conditions, check it for soiling and damage.
- ▶ Clean the product immediately if it gets dirty, following the instructions in the section "Cleaning", in order to prevent permanent soiling.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ **Inform the patient.**

#### NOTICE

##### **Careless use of the product**


Damage to the product due to contact with pointed or sharp-edged objects

- ▶ Do not allow the product to come into contact with the aforementioned objects.


### 4 Scope of Delivery and Accessories

The following single components and accessories are included in the scope of delivery in the specified quantities and are available for reordering as single components (■), single components with minimum order quantities (▲) or single-component packs (●):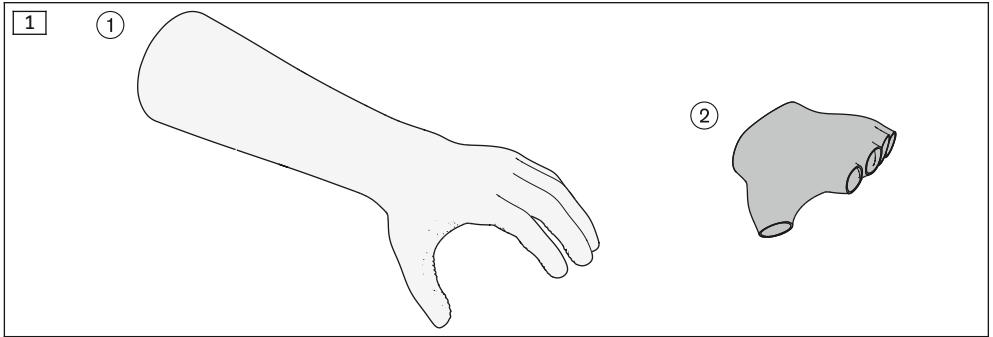

8S500=* AxonSkin Visual + Black, 8S501=*, 8S502=* AxonSkin Natural					
Fig.	Item no.		Quantity	Designation	Reference Number
-	-	■	1	Instructions for Use (Qualified Personnel)	647G596=all_INT
-	-	■	1	Instructions for Use (Patient)	647H898=all_INT
-	-	■	1	Sleeve AxonSkin PVC Gloves	647G993=all_INT
1	①	-	1	Prosthetic Glove	-
<b>The following parts are only for:</b>				<b>Prosthetic Glove</b>	<b>8S500=*-M20, 8S501=*, 8S502=*</b>
1	②	-	1	Textile Glove	8Y29

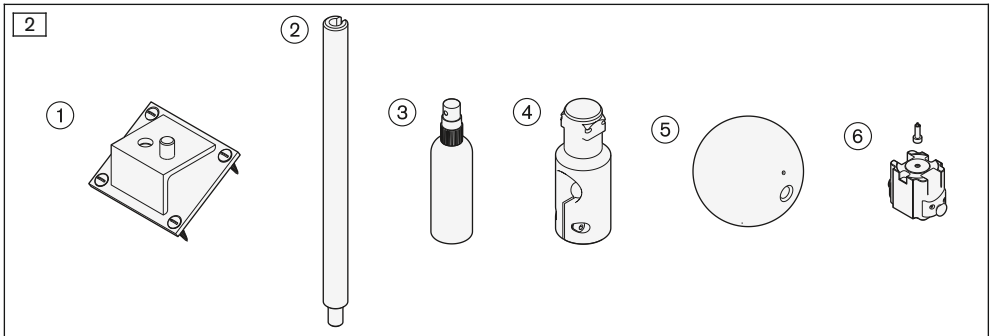Accessories					
Fig.	Item no.		Quantity	Designation	Reference number
2	①	■	0	Mounting plate	711M2
2	②	■	0	Mounting tool (alignment rod)	711M1
2	③	■	0	Donning spray	640F18
2	④	■	0	Mounting tool	711M64
2	⑤	■	0	Donning sphere	711M114
2	⑥	■	0	Mounting and test adapter (only for 8E550=*)	711M116
-	-	■	0	Special cleaner	640F12
-	-	■	0	Pump sprayer	640F13

## 5 Preparing the product for use

### NOTICE

#### Incorrect donning and removal of the prosthetic glove

Damage to the prosthetic hand and the prosthetic glove due to non-observance of instructions for use

- ▶ Observe all instructions for donning and removal. The prosthetic hand must always be mounted on the donning tool.
- ▶ Only use the components listed (equipment, tools, materials) and have them close to hand.
- ▶ Avoid damage that can be caused to the prosthetic hand and the prosthetic glove, e.g. from tools and excessive force (in particular on the thumb).

### INFORMATION

It is not possible to colour the prosthetic glove with the 635P15=\* Colour Pens.

## 5.1 Preparations for donning and removal

### NOTICE


#### Non-observance of instructions for the donning tool

Damage to the product due to unsuitable donning tool


- ▶ Only use the donning tool specified and follow the donning instructions.


- > **Tools required:**  
711M64 Mounting Tool, 711M1 Mounting Tool (Alignment Rod) and either 711M2 Mounting Plate or vice
- ▶ Install the mounting tool on the alignment rod.  
**INFORMATION: The alignment rod must be clamped in a vice, vertically for donning and horizontally for removal.**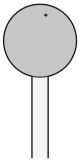

- > **Tools required:**  
711M114 Donning Sphere, 711M1 Mounting Tool (Alignment Rod) and either 711M2 Mounting Plate or vice
- ▶ Attach the donning sphere to the alignment rod.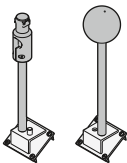

- ▶ **When using the mounting plate:**  
**INFORMATION: Install the mounting plate on a suitable surface and at optimal working height.**  
Attach the alignment rod to the mounting plate.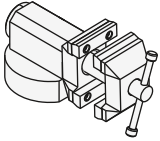

- ▶ **When using a vice:**  
Clamp the alignment rod into a vice at optimal working height.

## 5.2 Don the prosthetic glove on the prosthetic hand


- ▶ **INFORMATION:** There are differences in application between the 8E500 Michelangelo Hand and the 8E550 Michelangelo Hand Transcarpal. The headings indicate whether the instructions that follow them apply for one or both of the prosthetic hands.

Only follow the application instructions that apply for the prosthetic hand in question.

### 8E500: Install the prosthetic hand on the mounting tool

- ▶ Set the prosthetic hand to hand position "Neutral position" (Neutral Mode) and lock the wrist.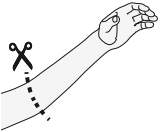

- ▶ Shorten the prosthetic glove at the point with the largest diameter (at the height of the elbow).


- ▶ Install the prosthetic hand on the mounting tool.


### 8E550: Install the prosthetic hand on the mounting tool


- ▶ Set the prosthetic hand to hand position "Neutral position" (Neutral Mode).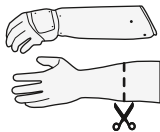


- ▶ Shorten the prosthetic glove to the correct length.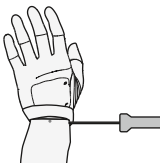

- ▶ Remove the prosthetic hand from the prosthetic socket by loosening the screws.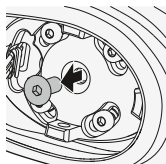

- ▶ Unscrew the screw from the prosthetic hand.

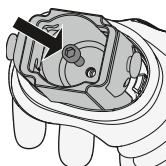- ▶ **NOTICE! Protect the cables and plugs against damage (for instance due to pinching).**
- ▶ Install the mounting and test adapter on the prosthetic hand.


- ▶ Install the prosthetic hand with mounting and test adapter on the mounting tool.

### 8E500/8E550: Slip on the prosthetic glove

#### INFORMATION

A crucial factor when donning the prosthetic glove is that it is warm during the stretching, donning and adjustment process.

- ▶ Work quickly, adhering to the sequence of the instructions provided.

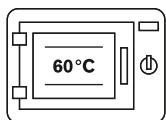- ▶ **When using a convection oven:**  
Preheat the oven (**60 °C**).


- ▶ **INFORMATION: A textile glove is not used with the AxonSkin Visual.**

**NOTICE! Do not overstretch the textile glove during the donning process.**

Roll up the textile glove, slide it over the fingers of the prosthetic hand and unroll it over the prosthetic hand.


- ▶ Observe the correct positioning:  
The cuff of the thumb is positioned in the last thumb groove.  
The cuff of the wrist ends at the last groove in the hand shell.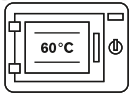

- ▶ **NOTICE! Do not use a hot air gun. Do not exceed the temperature or time.**


Place the prosthetic glove into the oven (**10 – 15 min / 60 °C**).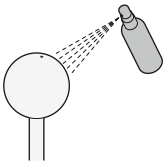


- > **Required materials:**

640F18 Donning Spray


- ▶ Spray the donning sphere with donning spray immediately prior to removal of the prosthetic glove from the oven.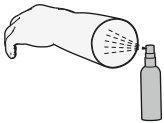


- ▶ Remove the prosthetic glove from the oven and spray the inside with donning spray.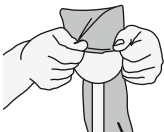


- ▶ Pull the prosthetic glove on to the donning sphere.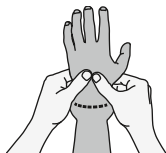


- ▶ **NOTICE! Do not exceed the maximum stretch time (20 s).** Place the prosthetic glove on to the donning sphere at the height of the wrist (see dashed line).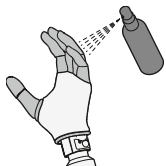

- ▶ Only spray the fingers with donning spray.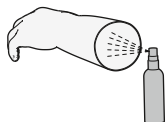

- ▶ Spray the inside of the prosthetic glove with donning spray.

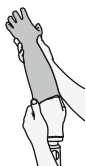
- ▶ **NOTICE! Avoid all excessive force on the fingers (especially the thumb) during the donning process.** Position the prosthetic glove on the prosthetic hand for donning.


- ▶ Hold the prosthetic glove at the sleeve end and pull it over the prosthetic hand in one movement.

### 8E550: Install the prosthetic hand on the prosthetic socket

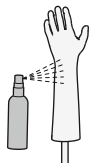
- ▶ Spray the outside of the prosthetic glove with donning spray.

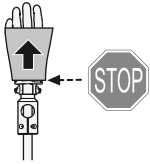
- ▶ Fold over the sleeve of the prosthetic glove in the distal direction so the threads are easily accessible for installation on the lamination ring.


- ▶ Remove the prosthetic hand from the mounting tool.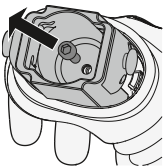


- ▶ Remove the mounting and test adapter from the prosthetic hand.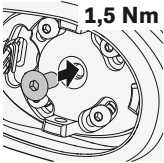


- ▶ Tighten the screw in the prosthetic hand (**1.5 Nm**).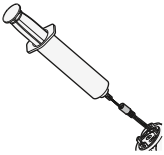


- ▶ Apply silicone grease to the connection plug.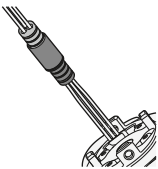

- ▶ Connect the connection plugs.

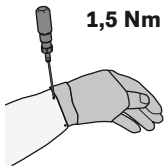
- ▶ **NOTICE!** When applying the prosthetic hand, protect the cables, connection plugs and prosthetic glove against damage (for instance due to pinching).
- ▶ Set the prosthetic hand onto the prosthetic socket and tighten the screws (**1.5 Nm**).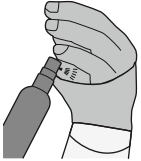


- ▶ Spray the prosthetic glove with donning spray and slip it over the prosthetic socket.

### 8E500: Install the prosthetic hand on the prosthetic socket

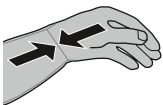- ▶ **NOTICE!** Protect the prosthetic glove against damage (for instance due to pinching).
- ▶ Slide the prosthetic hand onto the prosthetic socket until it engages audibly and cannot be pulled off again.

### 8E500/8E550: Smoothen and align the prosthetic glove

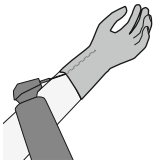
- ▶ Apply a very small amount of silicone spray under the prosthetic glove.
- ▶ Smoothen the prosthetic glove on the prosthetic socket.

- ▶ **NOTICE!** To prevent malfunctions, the prosthetic glove must not be twisted in the area of the fingers and especially the thumb.  
Adjust the prosthetic glove to fit the prosthetic hand properly.
- ▶ **CAUTION!** In order to prevent skin irritation, shorten the prosthetic glove so that it does not lie on the skin when wearing the prosthesis.


## 5.3 Removal from the prosthetic hand

### Removal with re-use of the prosthetic glove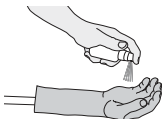


- ▶ Moisten the prosthetic glove with donning spray.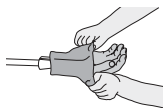

- ▶ **NOTICE! Avoid all excessive force on the fingers (especially the thumb) during the removal process.**  
Hold the prosthetic glove at the sleeve end and remove.

### Removal without re-use of the prosthetic glove

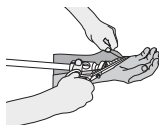- ▶ **NOTICE! Avoid damaging the prosthetic hand during cutting.**  
Cut open the prosthetic glove on the inside.

- ▶ **NOTICE! Avoid all excessive force on the fingers (especially the thumb) during the removal process.**  
Pull the sleeve as far apart as possible and remove the prosthetic glove.

## 6 Use

### 6.1 Instructing the patient in the use of the product

#### INFORMATION

- ▶ Inform the patient about all the important points relevant to them that are contained in these instructions for use.
- ▶ Instruct the patient in the correct use of the product in accordance with the **Instructions for use (patient)** included in the scope of delivery.
- ▶ Hand over the Instructions for use (patient) to the patient when the briefing has been completed.

### 6.2 Cleaning

#### NOTICE

#### Non-observance of cleaning instructions

Damage to product and permanent soiling due to use of improper cleaning agents

- ▶ Only clean the product according to the cleaning instructions provided in the instructions for use.
- ▶ **Inform the patient.**

#### Light soiling

##### > **Materials required:**

warm water, soap, cloth, lint-free cloth

- 1) Clean the product with warm water, soap and a cloth.
- 2) Rinse off soap residues with water.
- 3) Dry the product with a lint-free cloth and allow it to air dry fully.

## Heavy soiling

### > **Materials required:**

640F12 Special Cleaner, 640F13 Pump Sprayer, clean, undyed cloth (e.g. cellulose, linen), water, lint-free cloth

### 1) **NOTICE! Clean the product promptly to avoid heavy dirt from permanently contaminating the material.**

Spray the special cleaner on to the soiled area of the product and leave to take effect for a short time.

- 2) Wipe off the dirt from the product with a clean, undyed cloth using light pressure.
- 3) Rinse off residual cleaner and dirt with water.
- 4) Dry the product with a lint-free cloth and allow it to air dry fully.

## Removal of nail varnish

### > **Materials required:**

Nail varnish remover (acetone-free), clean, undyed cloth (e.g. cellulose, linen), water, lint-free cloth

- 1) Remove the nail varnish from the product with nail varnish remover and a clean, undyed cloth, using light pressure.
- 2) Rinse off residual cleaner and dirt with water.
- 3) Dry the product with a lint-free cloth and allow it to air dry fully.

## 7 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

## 8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

## 9 Technical data

Reference Number Men's version Women's version Unisex version	AxonSkin Natural 8S501=* 8S502=*	AxonSkin Visual 8S500=*	AxonSkin Black 8S500=*
<b>Sides</b>	right (R), left (L)		
<b>Sizes</b>	M		
<b>Colours</b>	2, 4, 6, 8, 11, 14, 16	0	20

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