
ottobock.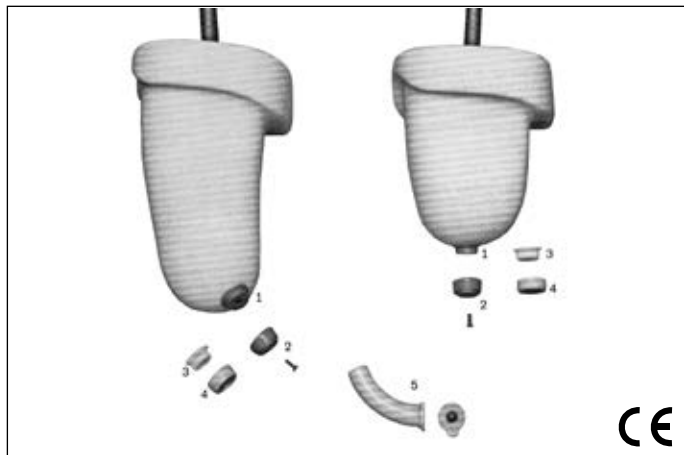

21Y45, 21Y95, 21Y97, 21Y105

DE Gebrauchsanweisung	2
EN Instructions for use	5
FR Instructions d'utilisation	8
IT Istruzioni per l'uso	11
ES Instrucciones de uso	14
PT Manual de utilização	17
NL Gebruiksaanwijzing	20
SV Bruksanvisning	23
DA Brugsanvisning	26
PL Instrukcja użytkowania	29
HU Használati utasítás	32
CS Návod k použití	35
EL Οδηγίες χρήσης	38
ZH 使用说明书	41

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NOTICE Warnings regarding possible technical damage.

INFORMATION Important information on the fitting/use.

INFORMATION

Last update: 2021-06-11

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- Please keep this document in a safe place.

The 21Y97 Valve provides the detachable connection between the flexible socket and the distal portion of the socket frame (for example in the ISNY Socket). The valve kit comprises all parts including patterns which are required for adequate installation. The 21Y45, 21Y95 and 21Y105 valve sets (without connection tube) are available for lateral installation.

1 Parts (Fig. 1)

- (1) Pattern for Vacuum Forming, with wood screw
- (2) Pattern for Laminating, with countersunk screws M5
- (3) Insert Ring
- (4) Lamination Collar with 21Y103 Seal Ring
- (5) Flat Rubber Valve with 21Y77 Tube and Seat Ring
- (6) Screw-in Aid (not illustrated)

2 Safety instructions

NOTICE

Risk of damage if using powder. Any kind of powder on the inner side of the socket or on the skin can restrict or disable the functionality of the valve during use. Avoid using powder in areas close to the valve.

3 Instruction for installation

The valve can be provided at the distal end, centrally or laterally, depending on the length of the residual limb. Detailed instruction for the fabrication of an ISNY Socket using the 21Y97 Valve is given in the Technical Information bulletins 2.3.5 and 2.3.6.

21Y97: Hold the 21Y101 Pattern for Vacuum Forming in position at the distal end of the plaster cast using a wood screw (Fig. 2).

21Y105: Use a wood screw to secure the 21Y101 vacuum forming template laterally to the residual limb end of the plaster model.

After vacuum forming clear the cylindrical part of the pattern. Remove the wood screw and attach the 21Y102 Pattern for Laminating by a countersunk screw (Fig. 3).

After lamination clear the area of the pattern by grinding. Remove the countersunk screw and pull the vacuum formed socket out of the lamination socket (Fig. 4).

Press the insert ring from the inside into the socket using the enclosed screw-in aid, and attach the lamination collar from the outside (Fig. 5).

When using a connection tube and seat ring: Fill Plastilin clay into the valve opening at the lamination socket, add PEDILEN foam to the socket attachment area and subsequently drill a hole through it, using a Forstner drill, diam. 28 mm (Fig. 6).

Apply sealing resin putty to the conical surface of the lamination collar. Press the flexible socket into the frame. Make sure it fits accurately in the valve area (Fig. 7).

After shaping the foam, fit in the connection tube of the valve (Fig. 8).

Apply sealing resin to the valve tube, insert it into the valve opening in such a way that tube and lamination collar are bonded to each other (Fig. 9).

Cross-cut of a socket with installed valve (Fig. 10):

- (1) Flexible Socket
- (2) Plastic Socket Frame
- (3) PEDILEN Rigid Foam H300
- (4) Outer Laminate
- (5) 5R1 Socket Attachment Block
- (6) Insert Ring*
- (7) Seal Ring
- (8) Lamination Collar
- (9) Connection Tube with Seat Ring

* An insert ring with longer thread (21Y99=14) which can be shortened for individual requirements is available.

21Y45, 21Y95: Insulate the valve with foil and connect it to the seat ring. Drill a hole corresponding to the valve at an appropriate place in the prosthetic socket. Glue the seat ring to the prosthetic socket and let the adhesive cure.

4 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

4.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

4.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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