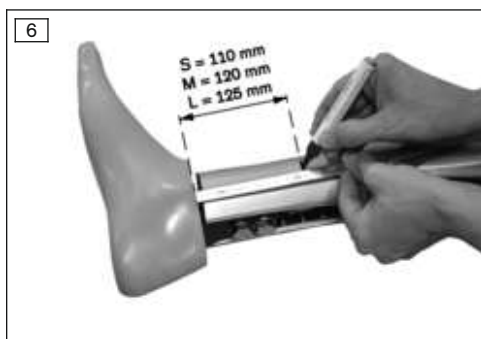
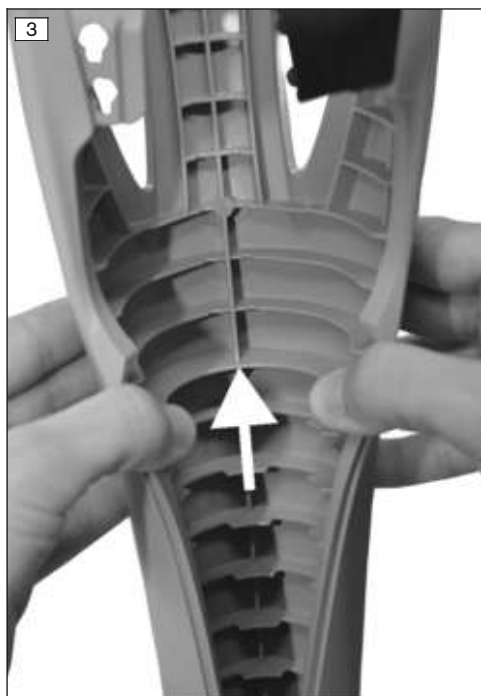
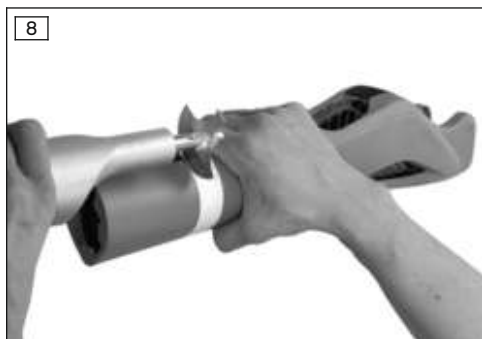
Genium Protector 4X880=*

DE	Gebrauchsanweisung (Fachpersonal)	4
EN	Instructions for use (qualified personnel)	10
FR	Instructions d'utilisation (Personnel spécialisé)	16
IT	Istruzioni per l'uso (Personale tecnico specializzato)	23
ES	Instrucciones de uso (Personal técnico especializado)	30
PT	Manual de utilização (Pessoal técnico)	36
NL	Gebruiksaanwijzing (Vakmensen)	43
SV	Bruksanvisning (Fackpersonal)	49
DA	Brugsanvisning (Faguddannet personale)	56
NO	Bruksanvisning (Fagpersonell)	62
FI	Käyttöohje (Ammattihenkilöstö)	68
PL	Instrukcja użytkowania (Personel fachowy)	74
CS	Návod k použití (Odborný personál)	81
HR	Upute za uporabu (Stručno osoblje)	87
TR	Kullanma talimatı (Uzman personel)	93
EL	Οδηγίες χρήσης (Τεχνικό προσωπικό)	99
RU	Руководство по применению (Квалифицированный персонал)	106
ZH	使用说明书 (专业人员)	113
KO	사용 설명서 (전문기술요원)	119

INFORMATION

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INFORMATION

Date of last update: 2021-03-03

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The "4X880=* Genium Protective Cover" is referred to simply as the Protective Cover/product below.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the Protective Cover into use in accordance with the information provided in the accompanying documents supplied.

2 Product description

2.1 Design

The product consists of the following components:

1. Protective Cover main component
2. Upper closure
3. Lengthwise closure
4. Protector foot cuff for Protective Cover

2.2 Function

The Protective Cover provides a means of enclosing the Genium electronic knee joint, including the tube adapter, to enhance its appearance and function. The housing of the knee joint is protected against external influences by the Protective Cover, e.g. scratching and surface contamination. The Protective Cover is not required for the proper and safe functioning of the joint.

2.3 Combination possibilities

Prosthetic components that are suited for combination with the product are listed below.

Knee joints

- 3B1, 3B1=ST Genium
- 3B1-2, 3B1-2=ST Genium
- Genium: 3B1-3, 3B1-3=ST

Prosthetic feet

- 1A30 Greissinger plus
- 1C60 Triton
- 1C61 Triton Vertical Shock
- 1C63 Triton Low Profile
- 1C64 Triton Heavy Duty
- 1C30 Trias
- 1C40 C-Walk
- 1D35 Dynamic Motion
- 1M10 Adjust
- 1E56 Axtion
- 1E57 Lo Rider
- Challenger: 1E95
- 1B1 Meridium
- Meridium: 1B1-2
- Empower: 1A1-1
- 1C10 Terion

Protector foot cuff according to foot size

Foot size	Cuff size
Up to 25	S
26 to 28	M
From 29	L

INFORMATION

The product "1B1=*/1B1-2=* Meridium" includes a connection plate that must be used for connecting the Protective Cover.

The connection plate is compatible with cuff sizes S and M. Cuff size M is equivalent to foot size 29 for the Meridium.

INFORMATION

The foot cuff is **not** compatible with the slim footshells.

3 Intended use

3.1 Indications for use

The product is to be used **solely** for lower limb exoprosthetic fittings.

3.2 Conditions of use

The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

In principal, the product can be used with a rotation axis-floor-measurement* of 430 mm - 560 mm.

An exception is fittings with the 1C61 Triton Vertical Shock prosthetic foot. Due to the height of the foot, the fitting area is restricted to 470 mm - 560 mm.

*Position of the alignment reference point of the Genium (rotation axis): 20 mm above the medial tibial plateau.

Rotation axis-floor-measurement measured while standing (barefoot)

INFORMATION

For smaller users that require the Protective Cover to be substantially shortened, the Protector foot cuff material can extend backwards into the charging area, with the result that it needs moving to the side when the charging unit is inserted. The rotation axis-floor-measurement at which this may occur depends on both the Protector foot cuff size and the foot being used.

3.3 Contraindications

- All conditions which contradict or go beyond the specifications listed in the section on "Safety" and "Indications for use".

3.4 Qualification

The fitting of patients with the product may only be carried out by qualified personnel (such as an O&P professional).

4 Safety

4.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

4.2 Structure of the safety instructions

CAUTION

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

> E.g.: Consequence 1 in case of failure to observe the hazard

> E.g.: Consequence 2 in case of failure to observe the hazard

▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General safety instructions

⚠ CAUTION

Non-observance of safety instructions

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety instructions and the stated precautions in this accompanying document.

⚠ CAUTION

Improper disassembly, assembly and handling of the product

- > Injury due to pinching of body parts.
- > Possible injury in case of parts with sharp edges.
- ▶ Use increased caution during disassembly/assembly and handling of the product.
- ▶ Instruct the patient in the proper disassembly, assembly and handling of the product.

NOTICE

Mechanical damage to the product

Change in or loss of functionality due to damage.

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

INFORMATION

When the Protective Cover is installed, the battery charger cable has to point to the upper closure. A correct knee joint charging process is only ensured with this alignment.

5 Scope of delivery

- 1 pc. 4X880=* Genium Protective Cover
- 1 pc. 4P880=U Upper closure
- 1 pc. 4P880=R Lengthwise closure
- 1 pc. 4P880=* Protector foot cuff
- 1 pc. Instructions for use (qualified personnel)
- 1 pc. Instructions for use (user)

6 Preparing the product for use

6.1 Required tools

- 718H5 Deburring knife
- 743B2 Tape measure
- 627B6=15 Crepe adhesive tape
- 636K13 adhesive (Loctite 241)
- Vibrating/band saw
- 710D20 Torque wrench

6.2 Shortening protective cover

6.2.1 Shortening the Protective Cover – Preliminary Shortening

- 1) Remove the prosthesis from the residual limb.
- 2) Remove the shoe from the prosthetic foot.
- 3) Loosen the foot connection cap of the prosthetic foot.

- 4) Check whether the closures on the Protective Cover are properly fastened and the Protective Cover is not twisted.
- 5) Lay the Protective Cover next to the prosthesis.
INFORMATION: Make sure that the knee plateaus of the knee joint and Protective Cover are at the same level (see Fig. 1).
- 6) Mark the Protective Cover distally at the level of the foot contact surface (see Fig. 2).
INFORMATION: For tall patients, shortening the Protective Cover may not be required.
- 7) Make another mark 1 cm above the existing mark.
- 8) Apply crepe adhesive tape proximally all around at this mark.
- 9) Use a vibrating or band saw to shorten the Protective Cover along the bottom edge of the crepe adhesive tape.

6.2.2 Shortening the Protective Cover – Final Shortening

- 1) Remove the crepe adhesive tape and lengthwise closure from the Protective Cover after preliminary shortening.
- 2) Open the upper closure on the Protective Cover.
- 3) Spread open the Protective Cover and apply it to the knee joint (see fig. 4).
INFORMATION: The two retainers of the Protective Cover main component (at the side in the upper closure area) have to engage on the frame element of the knee joint (see fig. 5).
- 4) Fasten the upper closure.
 - The retainer in the area of the upper closure has to engage on the frame element of the knee joint (see fig. 14).
 - Compress the Protective Cover slightly with both hands.
 - Engage the two retainers of the upper closure in the frame of the knee joint.
 - Push down the upper closure with your thumb after engaging it (see fig. 15).
- 5) Depending on the size of the chosen Protector foot cuff, mark the following distances in at least three places on the Protective Cover (see fig. 6):
INFORMATION: The distances have to be measured from the foot contact surface in each case, creating a cut edge parallel to the foot contact surface. This allows the Protector foot cuff to overlap over the Protective Cover main component to an ideal extent (see page 15).
 - Size S: 110 mm
 - Size M: 120 mm
 - Size L: 125 mm
- 6) Remove the Protective Cover.
- 7) Fasten the upper and lengthwise closures (see fig. 7).
- 8) Tape off the Protective Cover with crepe adhesive tape above the mark.
- 9) Using a vibrating or band saw, shorten the Protective Cover along the bottom edge of the crepe adhesive tape (see Fig. 8).
INFORMATION: Make sure that the closures are not shifted while sawing.
- 10) Deburr the cut edge.

NOTICE

Loss of stability due to shortening the Protective Cover too much

No hold of the Protective Cover on the knee joint.

- ▶ At the most, the Protective Cover may be shortened to just before the marked rib (see fig. 3).

Additional Instruction for Using a Torsion Adapter

- ▶ Using suitable tools, remove the profile ribs on the interior of the Protective Cover (see Fig. 9).

6.3 Installing the Protective Cover and Foot Cuff

6.3.1 Installing the Foot Cuff

- 1) Remove the prosthetic foot from the tube adapter.
In doing so, unscrew the two set screws that are screwed in the furthest but are not opposite each other. Mark the two set screws.
- 2) Pull the foot cuff over the tube adapter (see Fig. 10).
- 3) Install the prosthetic foot on the tube adapter (see Fig. 11).
Apply Loctite® 241 to the two set screws and screw them in.
- 4) Position the clips of the foot cuff on the footshell, aligning the seam to posterior.
- 5) Attach the clips in the footshell (see Fig. 12).

INFORMATION: When using the 1C40 C-Walk, the clips that do not engage have to be removed.

6.3.2 Protective Cover Installation

- 1) Remove the crepe adhesive tape from the Protective Cover after final shortening.
- 2) Open or remove the closures.
- 3) Spread open the Protective Cover.
- 4) Thread the Protective Cover into the Protective Cover foot cuff from anterior (see fig. 13).
- 5) Apply the Protective Cover to the knee joint.

INFORMATION: The retainers of the Protective Cover have to engage on the frame element.

- 6) Fasten the two closures on the back of the Protective Cover (see fig. 7 and see fig. 15).

NOTICE

Loss of stability due to inadequately installed retainers

No hold of the Protective Cover on the knee joint.

- ▶ Ensure that all retainers and closures are properly engaged and/or fastened.

6.4 Functional Test

After assembly, the following points have to be checked:

- Does the foot cuff overlap the Protective Cover by approx. 2 cm?
- Is the function of the rotation adapter impaired?

Material may be removed from the Protective Cover to improve the functionality.

INFORMATION: If material is removed, the stability/flexibility of the Protective Cover on the knee joint may be impaired.

7 Maintenance

- ▶ During the standard inspection of the prosthesis components, also inspect the product.

7.1 Cleaning and Care

NOTICE

Improper product care

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Only clean the product with a damp cloth and mild soap (e.g. 453H10=1-N Ottobock DermaClean).

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

8 Legal information

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

9 Technical data

Ambient conditions	
Storage and transport in original packaging	-25 °C/-13 °F to +70 °C/+158 °F
Storage and transport without packaging	-25 °C/-13 °F to +70 °C/+158 °F max. 93% relative humidity, non-condensing
Operation	-10 °C/+14 °F to +40 °C/+104 °F max. 93% relative humidity, non-condensing

General	
Reference number	4X880=*
Product service life	Wear part subject to normal wear and tear
Protective Cover weight (with closures)	450 g
Foot cuff weight	approx. 60 g

10 Appendices

10.1 Symbols Used

 Lot number (PPPP YYYY WW)
PPPP – plant
YYYY – year of manufacture
WW – week of manufacture

Manufacturer

Declaration of conformity according to the applicable European directives

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Otto Bock Healthcare Products GmbH