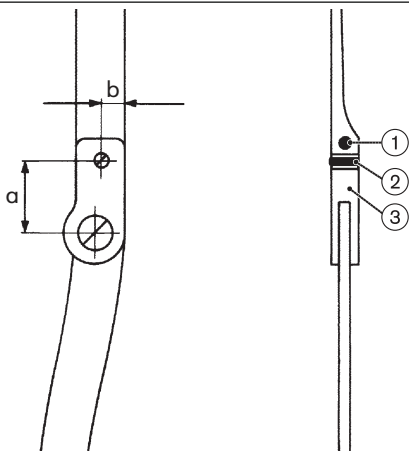
## 501D1

|   |    |
|---|----|
| <b>DE</b> Montageanleitung .....            | 3  |
| <b>EN</b> Assembly instructions .....       | 6  |
| <b>FR</b> Instructions de montage .....     | 9  |
| <b>IT</b> Istruzioni per il montaggio ..... | 12 |
| <b>ES</b> Instrucciones de montaje .....    | 15 |
| <b>PT</b> Instruções de montagem .....      | 18 |
| <b>NL</b> Montagehandleiding .....          | 22 |
| <b>SV</b> Monteringsanvisning .....         | 25 |
| <b>DA</b> Monteringsvejledning .....        | 28 |
| <b>NO</b> Monteringsanvisning .....         | 31 |
| <b>PL</b> Instrukcja montażu .....          | 34 |
| <b>CS</b> Montážní návod .....              | 37 |
| <b>TR</b> Montaj talimatları .....          | 40 |
| <b>EL</b> Οδηγίες συναρμολόγησης .....      | 43 |
| <b>RU</b> Инструкция по монтажу .....       | 46 |
| <b>JA</b> 組立方法 .....                        | 50 |

1

**Bohrungsmaße [Drill whole measurement]**

| Größe/Size   | a [mm] | b [mm] |
|--------------|--------|--------|
| 17K29=4      | 16     | 10     |
| 17K29=5, *=6 | 21     | 7      |

## INFORMATION

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### INFORMATION

Date of last update: 2021-12-08

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide important information on the processing of the 501D1 spring-loaded thrust piece.

## 2 Product description

| Components (see fig. 1) |                            |                |
|-------------------------|----------------------------|----------------|
| Item                    | Designation                | Article number |
| 1                       | Clamping sleeve            | 506S1=3x14     |
| 2                       | Spring-loaded thrust piece | 501D1          |
| 3                       | Orthotic knee joint        | 17K29*         |

### 2.1 Available sizes

The 501D1 spring-loaded thrust piece made of stainless steel is available in one size.

### 2.2 Components

| Scope of delivery |          |                            |                  |
|-------------------|----------|----------------------------|------------------|
| Item              | Piece(s) | Designation                | Reference number |
| not illustrated   | 1        | Spring-loaded thrust piece | 501D1            |

## 3 Intended use

### 3.1 Indications for use

The product is intended **exclusively** for orthotic fittings of the lower limbs.

### 3.2 Combination possibilities

The 501D1 spring-loaded thrust piece can be combined with the 17K29=4, \*=5 and \*=6 orthotic knee joints.

### 3.3 Lifetime

The product is designed for a lifetime of **3 years** when used as intended and assembled professionally.

### 3.4 Qualification

Patients may be fitted with the product only by trained qualified personnel. The qualified personnel must be familiar with the handling of the various techniques, materials, machines and tools.

## 4 Safety

### 4.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

### 4.2 Safety instructions

 **CAUTION**

#### **Incorrect alignment, assembly or adjustment**

Risk of injury due to change in or loss of functionality

- ▶ Assembly, adjustment and maintenance operations may only be completed by qualified personnel.
- ▶ Observe the alignment, assembly and adjustment instructions.

## 5 Preparing the product for use

### Preparing the threaded bore

- > **Required tools:** 724S6=3.2 drill, 726W8 counterbore, 731B21=M4 screw tap.
- 1) Remove the clamping sleeve from the upper joint section and take off the ring lock.
  - 2) Determine the position of the bore according to the table and mark it on the bar (see page 2).
  - 3) Drill the **Ø 3.2 mm** core hole.

- 4) Deburr the bore with the 726W8 counterbore.
- 5) Cut the M4 thread.

### **Installing the product**

- 1) Screw the spring-loaded thrust piece into the thread and secure it with Loctite 241. The thrust piece has to be flush with the surface of the bar.
- 2) Slide on the ring lock.
- 3) Check the function of the thrust piece and readjust if needed.
- 4) Insert the clamping sleeve.

## **6 Cleaning**

Promptly clean the product after contact with water containing salt, chlorine or soap, or if it gets dirty.

- 1) Rinse the product with clear fresh water.
- 2) Dry the product with a cloth or allow it to air dry. Avoid exposure to direct heat (e. g. from an oven or radiator).

## **7 Maintenance**

### **INFORMATION**

**The product may be exposed to increased loads by the patient.**

- ▶ Shorten the maintenance intervals according to the expected loads.

## **8 Disposal**

Dispose of the product in accordance with national regulations.

## **9 Legal information**

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### **9.1 Liability**

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### **9.2 CE conformity**

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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