
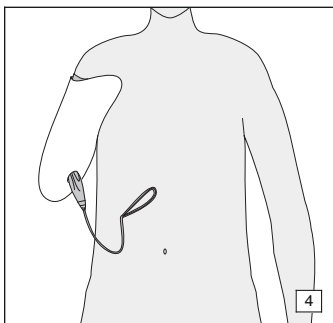
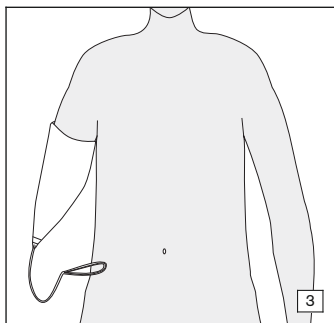
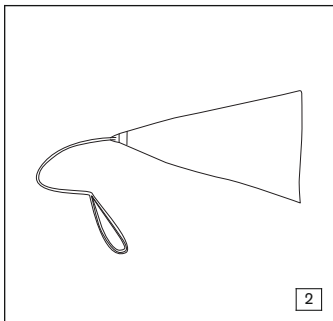
## OC1560 EasyFit Arm

<b>DE</b> Gebrauchsanleitung (Benutzer / Techniker) .....	3
<b>EN</b> Instructions for Use (User / Technician) .....	5
<b>FR</b> Mode d'emploi (Utilisateur / Technicien) .....	8
<b>IT</b> Istruzioni d'uso (utilizzatore / tecnici) .....	11
<b>ES</b> Instrucciones de uso (usuario / técnico) .....	14
<b>PT</b> Manual de instruções (Usuário / Técnico) .....	17
<b>NL</b> Gebruiksaanwijzing (gebruiker/technicus) .....	19
<b>SV</b> Bruksanvisning (användare/tekniker) .....	22
<b>DA</b> Brugsvejledning (bruger/tekniker) .....	25
<b>NO</b> Bruksanvisning (bruker/teknikker) .....	28
<b>FI</b> Käyttöohje (käyttjä/teknikko) .....	30
<b>PL</b> Instrukcja użytkowania (Użytkownik / Technik) .....	33
<b>HU</b> Használati utasítás (használóknak / műszerészeknek) .....	36
<b>CS</b> Návod k použití (pro uživatele / techniky) .....	39
<b>RO</b> Instrucțiuni de utilizare (utilizatori / tehnicieni) .....	41
<b>HR</b> Upute za uporabu (korisnik/tehničar) .....	44
<b>SL</b> Navodila za uporabo (uporabnik/tehnik) .....	47
<b>SK</b> Návod na použitie (používateľ / technik) .....	49
<b>BG</b> Инструкция за употреба (потребител / техник) .....	52
<b>TR</b> Kullanım kılavuzu (Kullanıcı / Teknisyen) .....	55
<b>EL</b> Οδηγίες χρήσης (χρήστης/τεχνικός) .....	58
<b>RU</b> Руководство по применению (для пользователей / технического персонала) .....	61
<b>JA</b> 取扱説明書（ユーザー用/技術者用） .....	64
<b>ZH</b> 使用说明书（用户/技术人员） .....	66
<b>KO</b> 사용 안내서 (사용자/기술자) .....	69

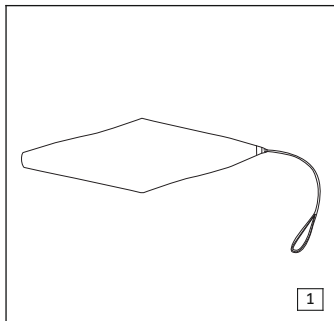# 1 Produktbeschreibung

## INFORMATION

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### INFORMATION

Last update: 2013-10-24

- ▶ Please read this document carefully.
- ▶ Follow the safety instructions.

## 1.1 Construction and Function

The OC1560 EasyFit Arm donning sheath is used for easier donning of arm prostheses with valve opening.

It is made of a lightweight material (silicone-coated nylon) with very good sliding qualities and reduces the pain prosthesis wearers may experience when donning their prosthesis.

## 2 Application

### 2.1 Intended Use

The product is to be used solely for upper limb prosthetic fittings.

The product can be used for prostheses with a valve opening.

## 3 Safety

### 3.1 Explanation of Warning Symbols

 **WARNING** Warning regarding possible serious risks of accident or injury.

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warnings regarding possible technical damage.

### 3.2 General Safety Instructions

 **WARNING**

#### **Product within reach of children**

Danger of suffocation due to lack of oxygen supply

- ▶ Store the product away from the reach of children.

 **CAUTION**

#### **Using the product without following the instructions for use**

Deteriorating health condition and damage to the product because of failure to follow the safety information

- ▶ Observe the safety information in these instructions for use.

 **CAUTION**

#### **Reuse on other users**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Use the product on only one user.

#### NOTICE

#### **Mechanical overload**

Impaired functionality due to mechanical damage

- ▶ Check the product for damage prior to each use.
- ▶ Do not use the product if its functionality has been impaired.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

## 4 Handling

### 4.1 Application

#### NOTICE

#### **Use of powder (e.g. baby powder, talcum)**

Loss of function of prosthetic components due to clogging with particles or deprivation of lubricant

- ▶ Avoid contact with powders.

- 1) Fully spread out the donning sheath (see Fig. 1).
- 2) Turn one half of the donning sheath into the other half, so that the strap is facing out (see Fig. 2).
- 3) Pull the donning sheath over the residual limb, avoiding any wrinkles (see Fig. 3).
- 4) Slide into the prosthetic socket and guide the pull-through aid through the valve opening on the prosthetic socket.
- 5) Pull the donning sheath out through the valve opening on the prosthetic socket (see Fig. 4).

#### INFORMATION

- ▶ Perform light pumping movements with the residual limb to aid donning the prosthetic socket.
- ▶ Pull on different parts of the material to ensure that the soft tissue is evenly distributed.

### 4.2 Cleaning

#### NOTICE

#### **Use of improper cleaning agents**

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

- 1) Wash the product in **40 °C** warm water with standard mild detergent.
- 2) Allow the product to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

## 5 Disposal

Dispose of the product in accordance with national regulations.

## 6 Legal Information

### 6.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document (Instructions for Use). The manufacturer will not assume liability for damage caused by disregard of this document.

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Otto Bock HealthCare GmbH