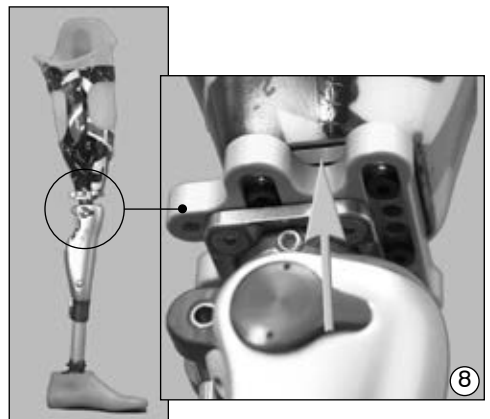
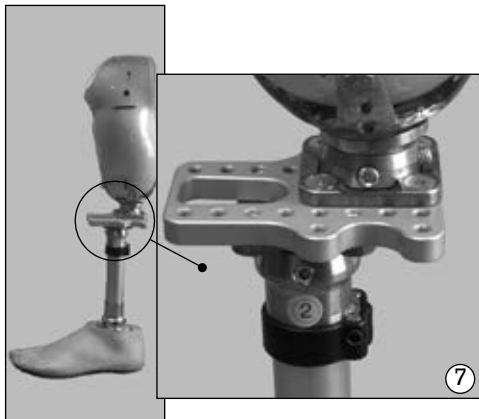
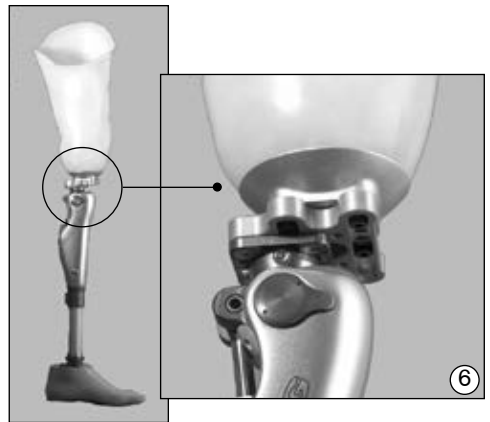
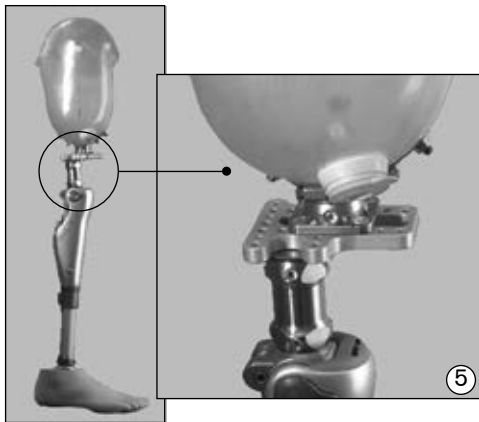
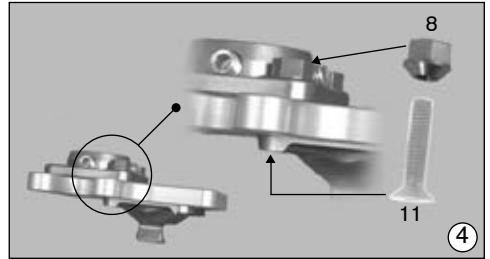
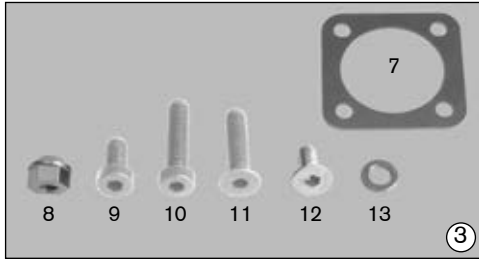
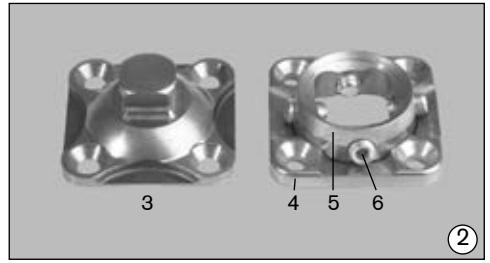
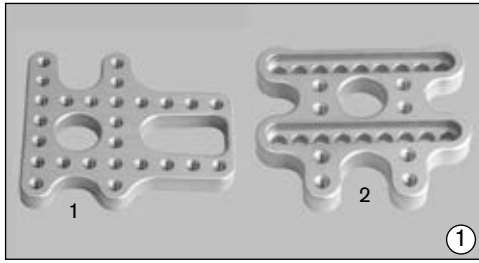
4R112

DE Gebrauchsanweisung (Fachpersonal)	3
EN Instructions for use (qualified personnel)	7
FR Instructions d'utilisation (Personnel spécialisé)	11
IT Istruzioni per l'uso (Personale tecnico specializzato)	15
ES Instrucciones de uso (Personal técnico especializado)	20
NL Gebruiksaanwijzing (Vakmensen)	24
SV Bruksanvisning (Fackpersonal)	28

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English

Last update: 2015-10-20

- Please read this document carefully before using the product.
- Follow the safety instructions to avoid injuries and damage to the product.
- Instruct the user in the proper and safe use of the product.
- Please contact the manufacturer if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). Contact information can be found on the back page.
- Please keep this document in a safe place.

1 Application

The Ottobock Universal Alignment Adapter-Set 4R112 is to be **used** solely for the temporary prosthetic fitting of lower limb amputees.

Suitable for patients weighing up to a maximum of 100 kg.

1.1 Qualification

Patients may be fitted with the product only by qualified personnel authorised by Ottobock after completing the corresponding training.

2 Safety recommendations

Attention!

- The Universal Alignment Adapter is to be employed only during trial fittings. The Universal Alignment adapter is not intended for permanent installation or longer use!
- Be sure that all screws are tightened to the previously mentioned torque values. Please use the Ottobock Torque Wrench (Ottobock 710D4) for this purpose.
- Before using the Universal Alignment Adapter, please check for any dents, nicks or hair-line cracks. If in doubt, please contact the Ottobock Professional Services department.
- Alteration of Universal Alignment Adapter components is strictly forbidden by the manufacturer.

**Attention!**

Generally port connections (double adapter w/pyramid or rotational 4-hole adapter) are not standardized. Therefore a combination of connecting elements from different (various) manufacturers could influence the stability with negative results.

3 Components

The Universal Alignment Adapter-Set 4R112 is delivered with:

(Fig. 1)	Pos. 1	1pc.	Mounting plate 1	4R112-1
	Pos. 2	1pc.	Mounting plate 2	4R112-2
(Fig. 2)	Pos. 3	1pc.	4-Hole adapter w/pyramid	
	Pos. 4	1pc.	Rotational 4-hole adapter/ base plate	
	Pos. 5	1pc.	Rotational 4-hole adapter/ adapter ring	
	Pos. 6	4pc.	Allen head set screws	M8x12
(Fig. 3)	Pos. 7	4pc.	Pressure plate	4Y19
	Pos. 8	4pc.	Hex nut with conical face	502Z22=M6
	Pos. 9	4pc.	Cap screw/Allen head/low profile	M6x12 501T61=M6x12
	Pos. 10	4pc.	Cap screw/Allen head/low profile	M6x25 501T48=M6x25
	Pos. 11	4pc.	Countersunk Allen head screw	M6x25 501S71=M6x25
	Pos. 12	8pc.	Countersunk Allen head screw	M6x12 501S70=M6x12
	Pos. 13	4pc.	Washer	M6,2 507U12=6,2x10,3
		1pc.	Instruction manual	647H457

**Attention!**

Do not use the adapter pieces (Fig. 2) positions 3,4 and 5 in place of the very similar Ottobock components 4R54 and 4R51 because the Universal Alignment Adapter lacks a shear ring. Use these components only with the Universal Alignment Adapter-Set 4R112.

4 Description and function

The Ottobock Universal Alignment Adapter-Set 4R112 can be used during trial fittings to alter both linear and rotational positions between modular prosthetic components. This adjustability can be used to optimize the prosthetic fitting.

5 Examples of typical use

(Fig. 5) Assembly between transfemoral test socket and knee joint using socket adapter 4R42 or 4R89 along with the following components:

	Mounting plate 1	
	4-Hole adapter w/pyramid	
	Rotational 4-hole adapter/ base plate	
	Rotational 4-hole adapter/ adapter ring	
	Allen head set screws	M8x12
8 pc.	Countersunk Allen head screws	M6x12

(Fig. 6) Assembly between transfemoral test socket and knee joint with using socket adapter 5R6 along with the following components:

	Mounting plate 2	
	Rotational 4-hole adapter/ base plate	
	Rotational 4-hole adapter/ adapter ring	
	Allen head set screws	M8x12
4 pc.	Countersunk Allen head screws	M6x12
4 pc.	Cap screw / Allen head / low profile	M6x25
4 pc.	Washer	M6,2

(Fig. 7) Assembly between Transtibial test socket and foot using socket attachment block 4R42 or 4R89 along with the following components:

	Mounting plate 1	
	4-Hole adapter w/pyramid	
	Rotational 4-hole adapter/ base plate	
	Rotational 4-hole adapter/ adapter ring	
	Allen head set screws	M8x12
8 pc.	Countersunk Allen head screws	M6x12

(Fig. 8) Assembly between transfemoral test socket and knee joint using socket attachment block 5R1 along with the following components:

	Mounting plate 2	
	Rotational 4-hole adapter/ base plate	
	Rotational 4-hole adapter/ adapter ring	
	Allen head set screws	M8x12
4 pc.	Countersunk Allen head screws	M6x12
4 pc.	Cap screw/Allen head/low profile	M6x25
	Pressure plate	
4 pc.	Washer	M6,2

6 Assembly

Attention!

The adapter ring of the rotational 4-hole adapter (Fig. 2, Pos. 5) must be mounted in full contact with the mounting plate. Tighten screws holding the base plate of the rotational 4-hole adapter (Fig. 2, Pos. 4) equally in a diagonal pattern.

Attention!

When mounting the Universal Alignment Adapter above the knee joint, make sure that the joint angle remains unrestricted and that no detrimental contact results. Please be aware that screw heads or bolts do not prevent orderly connection between the ball and the ball cup during assembly of the 4-hole adapter w/pyramid to the rotational 4-hole adapter/adapter ring.

The position of the adapter ring (Fig. 2, Pos. 5) can also be altered. Simply loosen the countersunk screws to rotate the adapter ring. The ring rotates with a ratcheting motion against the attachment plate.

Use all attachment points when connecting adapter to mounting plate. If, during assembly of the 4-hole adapter w/pyramid to the rotational 4-hole adapter, any of the attachment points lie over one another, use a longer countersunk Allen head screw (M6 x 25) along with a hex nut with conical face (Fig. 4).

In order to assemble mounting plate 2 to a socket that has a sunken receptacle (such as the Ottobock 5R1 socket attachment block), it may be necessary to use one or more pressure plates as filler so as not to damage the laminated edge (Fig. 8).

7 Symbols Used

Declaration of conformity according to the applicable European directives

LOT PPPP YYYY WW

Lot number

Legal manufacturer

8 Torque values

Set screw torque	M8	15 Nm
Countersunk screw torque	M6	10 Nm
Cap screw torque	M6	10 Nm

9 Technical data

A-P adjustment range: 48 mm in 12-mm-steps (Mounting plate 1)
 24 mm in 12-mm-steps (Mounting plate 2)

M-L adjustment range: 24 mm in 12-mm-steps (Mounting plate 1)
 18 mm in 9-mm-steps (Mounting plate 2)

Rotational adjustment range: +/- 18° in 3°-steps

Weight:	Mounting plate 1	Approx.	123 g
	Mounting plate 2	Approx.	113 g
	4-Hole adapter w/pyramid	Approx.	52 g
	Rotational 4-hole adapter	Approx.	68 g
	Pressure plate	Approx.	10 g

System height: Mounting plate 1 (with double-sided adapters) Approx. 32 mm

Mounting plate 2 (attached directly to socket) Approx. 12 mm

10 Liability

Otto Bock Healthcare Products GmbH, hereafter referred to as manufacturer, assumes liability only if the user complies with the processing, operating and maintenance instructions as well as the service intervals. The manufacturer explicitly states that this product may only be used in combination with components that were authorized by the manufacturer (see instructions for use

and catalogs). The manufacturer does not assume liability for damage caused by component combinations which it did not authorize.

The product may only be opened and repaired by authorized Ottobock technicians.

11 Trademarks

All denotations within this accompanying document are subject to the provisions of the respective-applicable trademark laws and the rights of the respective owners, with no restrictions.

All brands, trade names or company names may be registered trademarks and are subject to the rights of the respective owners.

Should trademarks in this accompanying document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

12 CE conformity

This product meets the requirements of the 93/42/EWG guidelines for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Ottobock with sole responsibility according to appendix VII of the guidelines.

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