
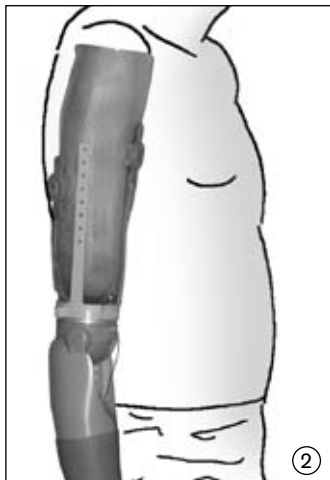
## 13E135

<b>DE</b>	Gebrauchsanweisung .....	3
<b>EN</b>	Instructions for Use .....	5
<b>FR</b>	Instructions d'utilisation .....	8
<b>IT</b>	Istruzioni per l'uso .....	10
<b>ES</b>	Instrucciones de uso .....	13
<b>PT</b>	Manual de utilização .....	16
<b>NL</b>	Gebruiksaanwijzing .....	18
<b>SV</b>	Bruksanvisning .....	21

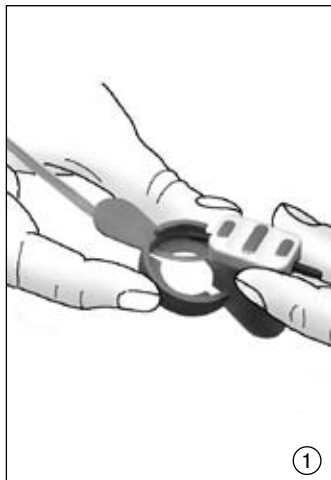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English

Date of the last update: 2013-09-23

- Please read this document carefully.
- Follow the safety instructions and the precautions specified in this accompanying document.
- If assistance is required during the start-up, use or maintenance of the product, as well as in the event of unexpected operating behaviour or circumstances, contact the manufacturer (see manufacturer's address on the back of this accompanying document).

## 1 Intended Use

The Electrode Mounting Set 13E135 is to be used **solely** for the mounting of Ottobock electrodes on upper extremity myoelectric fittings.

## 2 Description

The Electrode Mount 13E135 serves to positioning and mounting of Ottobock Electrodes. The electrode mount and the electrode are fixed to

the plaster interim socket or on the interim socket made of ThermoLyn (616T52, 616T53 or 646T69). This allows the patient to begin training his movements during the initial phase of rehab, creating a smoother transition to the final prosthesis.

### 3 Use

The 13E135 Electrode Mount can be used with the 13E200=\* Ottobock Electrode.

### 4 Safety Instructions

Follow the safety instructions and the precautions specified in this accompanying document.

#### **ATTENTION!**

Advise your patient that driving vehicles of any kind or operating machinery is strictly forbidden while using the interim socket.

#### **ATTENTION!**

Only use the Electrode Mount 13E135 in workshop or during a visit to a rehabilitation center.

### 5 Scope of Delivery

- 1 pc Electrode Mounting Set 13E135
- 1 pc Manual 647H472

### 6 Mounting Instructions

- 1.) Transfer the previously determined electrode position to the interim socket and drill a 35 mm diameter hole.
- 2.) Lay the electrode cable into the opening of the electrode mount and press into place (fig. 1).
- 3.) Use cloth tape to affix the electrode mount and the electrode cable to the interim socket of the patient.

#### **6.1 Example for mounting of the Ottobock Electrodes**

Fig. 1 Electrode 13E200=50, 13E200=60

## **7 Examples (Fig. 2, Fig. 3)**

Attach Electrode Mount 13E135 and electrodes, along with the mounting aid for ErgoArm elbow fitting parts 743A23, on upper part of interim socket (Fig. 2).

Making use of the equipment for interim socket as shown in Fig. 3 (with attached Electrode Mount 13E135 and electrodes, MyoBoy 757M11 and laptop) allows the patient to start training controlled movements during the very first phase of rehabilitation.

## **8 Liability**

Otto Bock Healthcare Products GmbH, hereafter referred to as manufacturer, assumes liability only if the user complies with the processing, operating and maintenance instructions as well as the service intervals. The manufacturer explicitly states that this product may only be used in combination with components that were authorized by the manufacturer (see instructions for use and catalogs). The manufacturer assumes no liability for damage caused by component combinations (use of third-party products) and applications which it did not authorise.

The product may only be opened and repaired by authorized Ottobock technicians.

## **9 Trademarks**

All denotations within this accompanying document are subject to the provisions of the respective applicable trademark laws and the rights of the respective owners, with no restrictions.

All brands, trade names or company names may be registered trademarks and are subject to the rights of the respective owners.

Should trademarks in this accompanying document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

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