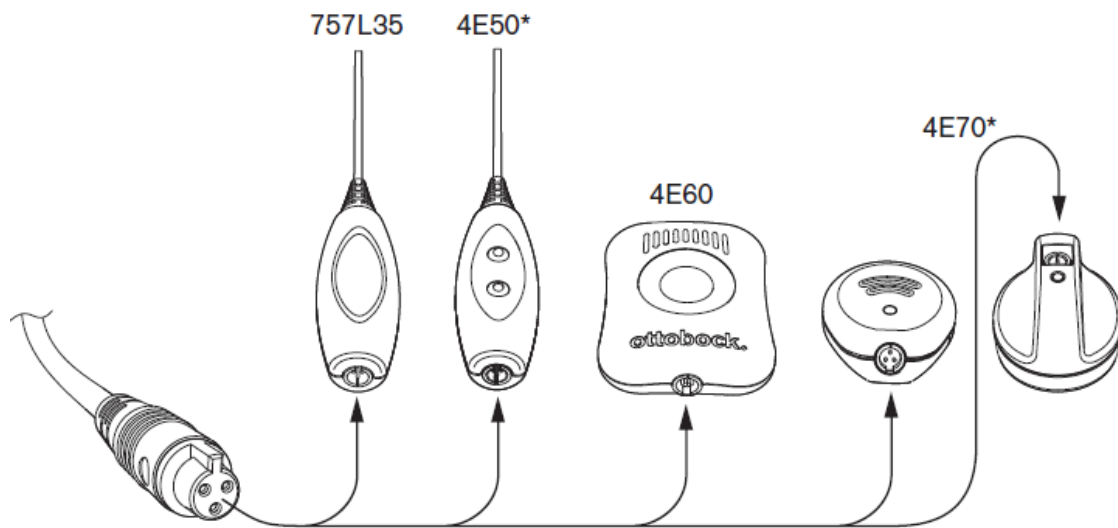
757L43

DE Gebrauchsanweisung (Benutzer)	3	NO Bruksanvisning (Bruker)	69
EN Instruction for use (user)	9	DA Brugsanvisning (Bruger)	75
FR Instructions d'utilisation (Utilisateur)	15	FI Käyttöohje (Käyttäjä)	81
IT Istruzioni per l'uso (Utilizzatore)	21	PL Instrukcja użytkownika (Użytkownik)	87
NL Gebruiksaanwijzing (Gebruiker)	27	SK Návod na používanie (Užívateľ)	93
ES Instrucciones de uso (Usuario)	33	SL Navodila za uporabo (Uporabnik)	99
SV Bruksanvisning (Användare)	39	CS Návod k použití (Uživatel)	105
RU Руководство по применению (Пользователь)	45	HR Upute za uporabu (Korisnik)	111
PT Manual de utilização (Usuário)	51	HU Használati utasítás (felhasználó)	117
ZH 使用说明书 (用户)	57	Electromagnetic environment	123
TR Kullanma talimatı (Kullanıcı)	63		

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Intended use

This USB charging adapter is intended to supply electricity to medical battery chargers for prostheses. Any other or additional use is considered improper and may cause damage and injuries. Claims of any kind due to damage resulting from other than intended use are excluded.

Warnings and safety notices

Warning

Warning regarding possible serious risks of accident or injury.

Caution

Warning regarding possible risks of accident or injury.

Notice

Warning regarding possible technical damage.

Warning

Risk of electric shock due to contact with live components

- Do not open the USB charging adapter.
- Do not immerse the USB charging adapter in liquids.
- Do not expose the USB charging adapter to extreme strain.
- In case of damage to the housing or cable, do not continue using the USB charging adapter. Replace it promptly.

Strangulation due to longer and flexible parts

- Keep the device and cables out of reach of children.

Allergic reaction due to skin contact

- The device is not suitable for permanent skin contact.

Caution

Interference of other electronic devices due to increased emissions

- Use the product only in combination with the accessories and cables listed in the sections "Compatibility" (inside the cover), "Scope of delivery" and "Suitable power sources with USB connection".

Notice

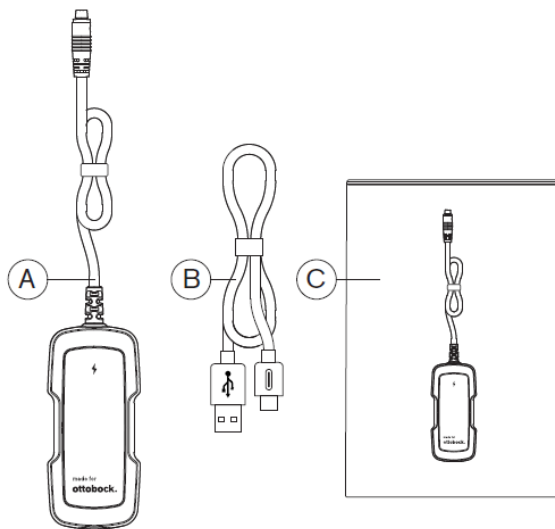
Lack of proper charging functionality due to malfunction

- Ensure that the USB charging adapter is not covered during operation and is not in the immediate vicinity of a heat source.
- Ensure that the USB charging adapter is not exposed to direct sunlight and resulting severe temperature fluctuations.
- Ensure that the USB charging adapter is not operated in environments with high humidity and/or condensation.
- Ensure that the USB charging adapter is not exposed to continuous vibrations.
- Do not operate the USB charging adapter in the immediate vicinity of other electronic devices.
- Do not stack the USB charging adapter with other electronic devices during operation.
- If simultaneous operation cannot be avoided, monitor the USB charging adapter and verify proper use in the existing setup.
- Changes and modifications are not permitted.

Information


- Heating of the housing during operation is normal and does not interfere with the proper functioning of the device.
- Use a damp cloth for cleaning.
- Use DESCOSEPT Pur disinfectant for surface disinfection.

Scope of delivery

- A... USB charging adapter
- B... USB cable (USB type C to USB type A)
- C... Instructions for use

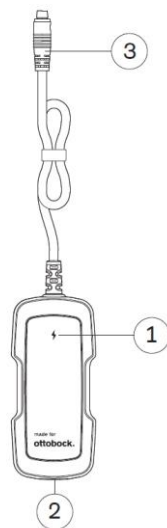Overview

- 1... Operating LED
- 2... USB type C port
- 3... Plug connection to Ottobock battery chargers

Suitable power sources with USB connection

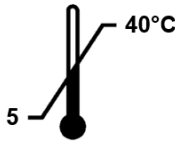
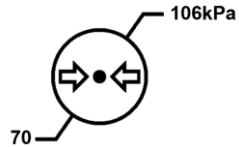
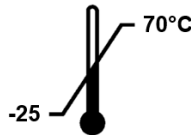
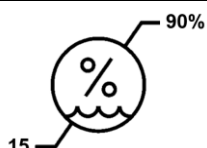
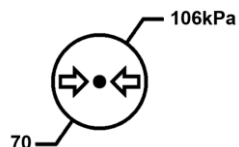
To ensure unrestricted charging in the intended charging times, the USB source (power supply, computer port etc.) that is used must provide a nominal voltage of 5.0 VDC and an output current of at least 2.5 A. Output currents less than 2.5 A can increase the charging time in some cases. Reliable charging is not possible with an output current less than 1.5 A.

When a power bank is used, the recommended minimum capacity is 10,000 mAh to ensure complete charging of the prosthesis.

A cable for power sources with a USB-A port is included in the scope of delivery. A suitable cable has to be used with power sources with a USB-C port. Be sure to only use cables with a sufficient current carrying capacity.

Technical data

Product	
Type	D2DFM 11.5 W 5 V 12 V
Protection level	2xMOPP working voltage 250 Vrms
Test standards	IEC/EN60601-1 Standard for the electrical safety of medical equipment
	IEC/EN60601-1-2 Standard for the electromagnetic compatibility of medical equipment
	IEC/EN60601-1-11 Standard for medical equipment in a household environment
Input voltage	5 VDC +/-10% USB C type
Max. input current	3 A
Output voltage	12 VDC +/-5%
Max. output current	0.96 A
Max. output power	11.5 W
Max. input cable length	2.0 m
Max. output cable length	0.5 m
Dimensions (L*W*H)	95.4 mm * 43.3 mm * 26 mm
Weight	85 g
Typical lifetime	8 years (17,500 active operating hours)

Ambient conditions			
Operation	Temperature	+5 °C to +40 °C	
	Relative humidity	Max. relative humidity 90%, non-condensing	
	Air pressure	70 to 106 kPa	
Storage	Temperature	-25 °C to +70 °C	
	Relative humidity	Max. relative humidity 90%, non-condensing	
	Air pressure	70 to 106 kPa	

Status of the operating LED

Before charging:

Operating LED	Meaning
LED shows green light	USB charging adapter is operational
LED shows red light or is not lit	USB charging adapter is defective

If the operating LED shows a green light, the individual Ottobock battery charger can be connected to the prosthesis.

During charging:

Operating LED	Meaning
LED shows green light	Unrestricted charging possible
LED shows yellow/orange light	Output power of the power source with USB connection is too low! Only limited charging possible. This results in longer charging times. Please use a different power source with USB connection.
LED shows red light	The output power of the power source with USB connection is insufficient, or the device is defective. Charging the prosthetic component is not possible. Please use a different power source with USB connection.

Troubleshooting

Operating LED not lit:

- The power source with USB connection is defective.
Disconnect the USB charging adapter from the power source with USB connection and replace it with a different one.
- The USB cable is defective.
Disconnect the USB cable from the USB charging adapter and from the power source with USB connection and replace it with a different one.
- The USB charging adapter is defective.
If changing the power source with USB connection and changing the USB cable does not cause the status of the operating LED to change, the USB charging adapter is defective.

Operating LED shows yellow/orange light:

- The output power of the power source with USB connection is insufficient.
Disconnect the USB charging adapter from the power source with USB connection and replace it with a power source with USB connection that can deliver a higher output current.

Operating LED shows red light:

- The prosthesis is defective.
Disconnect the individual battery charger from the prosthesis.
If disconnecting the battery charger from the prosthesis does not cause the status of the operating LED to change to green, please note the following point.
- The individual battery charger is defective.
Disconnect the individual battery charger from the USB charging adapter.
If disconnecting the battery charger from the prosthesis does not cause the status of the operating LED to change to green and disconnecting the individual battery charger from the USB charging adapter does not cause the status of the operating LED to change to green, the USB charging adapter is defective.

Disposal

The USB charging adapter has to be disposed of according to the applicable state laws if it is defective or at the end of its lifetime.

Explanation of symbols

Symbol	Description	Symbol	Description
	Symbol indicating conformity with the relevant EU directives		Use only in enclosed rooms
	NRTL C/US test mark for Canada and the USA		The power supply has to be disposed of according to the applicable state laws for electronics and electronic equipment.
	FCC – EMV test mark		Read the instructions for use
	Australia RCM test mark	IP22	Designation of the protection rating
	Protection class II		

Electromagnetic environment

For information on the electromagnetic environment, see page 123.

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EMC – Special requirements according medical standard	
Intended use and intended environment	Home healthcare and/or Professional environment
Basic safety and essential performance of the EUT	The DC/DC converter unit itself is not a medical end product, therefore no essential performance is defined by the manufacturer.
Basic safety regarding EMC	The DC/DC converter has to ensure proper output voltage according to its characteristics, without service within expected service life.
WARNINGS	Medical electrical equipment needs special precautions regarding EMC and needs to be installed according to EMC information.
	User shall not modify DC/DC converter.
	The DC/DC converter is designed to achieve the EMI behavior of the specified environment, it includes specific EMI filter to reduce the emissions which are specified in the IEC60601-1-2 standard.
	Please read the complete technical documentation to avoid adverse events to the patient and operator. Read also instructions for use.

EMC - Environment			
The DC/DC converter is intended for use in the electromagnetic environment specified below. The customer or the user of the DC/DC converter should assure that it is used in such an environment.			
Emissions test	Compliance	Electromagnetic environment - guidance	
RF emissions CISPR 11	Group 1	The DC/DC converter uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.	
RF emissions CISPR 11	Class B	The DC/DC converter is suitable for use in all establishments, including domestic establishments. This DC/DC converter shall be connected with an USB power supply. This USB power supply can be connected directly to the public low-voltage power supply network that supplies buildings used for domestic purposes.	
Harmonic emissions IEC 61000-3-2	N/A		
Voltage fluctuations / flicker emissions IEC 61000-3-3	N/A		
Immunity test	EN 60601-1-2:2015 test level	Achieved levels according EN 60601-1-2:2015 and achieved levels from additional standards.	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ±2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 8 kV contact ±2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 610004-4	N/A		
Surge IEC 61000-4-5	N/A		
Voltage dips, short interruptions and	N/A		

voltage variations on power supply input lines IEC 61000-4-11			
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	1, 3, 30 A/m	Power should be at levels characteristic of frequency magnetic fields a typical location in a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	6 Vrms 150 kHz to 80 MHz	6 Vrms	
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2,7 GHz	10 V/m	

Field strengths from fixed transmitters such as base stations for radio (cellular/cordless) telephones, land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast, cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters an electromagnetic site survey should be considered. If the measured field strength in the location in which the DC/DC converter is used, exceeds the applicable RF compliance level above, the DC/DC converter should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the DC/DC converter.

Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey should be less than the compliance level in each frequency range. Over the frequency range 150 kHz to 80 MHz, field strength should be less than 3 V/m.

Interference may occur in the vicinity of equipment marked with the following symbol: 

	Frequency range and Level: RF wireless communication equipment			
	Test Frequency (MHz)	Modulation	Immunity Level (V/m)	Supplementary information:
Proximity fields from RF wireless communications equipment IEC 61000-4-3	385	**Pulse Modulation: 18Hz	27	EUT powered at one of the nominal input voltages and frequencies.
	450	*FM ±5Hz deviation: 1kHz sine	28	
	710 745 780	**Pulse Modulation: 217Hz	9	Dwell time minimum 1s. Actual dwell time noted in results table.
	810 870 930	**Pulse Modulation: 18Hz	28	Note * - As an alternative to FM modulation, 50% pulse modulation at 18Hz may be used because while it does not represent actual modulation, it would be worst case.
	1720 1845 1970	**Pulse Modulation: 217Hz	28	
	2450	**Pulse Modulation: 217Hz	28	
	5240 5500 5785	**Pulse Modulation: 217Hz	9	Note ** - The carrier shall be modulated using 50% duty cycle square wave signal.

Warnings for medical EMC

The ME EQUIPMENT or ME SYSTEM is suitable for hospitals and other medical facilities (Professional healthcare / Home healthcare environment). Do not use the device near active HF SURGICAL EQUIPMENT and the RF shielded room of an ME SYSTEM, for example at magnetic resonance imaging, where the intensity of EM DISTURBANCES is high!

WARNING: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally!

WARNING: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation!

WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the [ME EQUIPMENT or ME SYSTEM), including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result!

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EGSTON System Electronics Eggenburg GmbH