## 4P862

<b>DE</b> Gebrauchsanweisung (Fachpersonal) .....	3
<b>EN</b> Instructions for use (qualified personnel) .....	7
<b>FR</b> Instructions d'utilisation (Personnel spécialisé) .....	12
<b>IT</b> Istruzioni per l'uso (Personale tecnico specializzato) .....	16
<b>ES</b> Instrucciones de uso (Personal técnico especializado) .....	21
<b>PT</b> Manual de utilização (Pessoal técnico) .....	26
<b>NL</b> Gebruiksaanwijzing (Vakmensen) .....	30
<b>SV</b> Bruksanvisning (Fackpersonal) .....	35
<b>DA</b> Brugsanvisning (Faguddannet personale) .....	40
<b>NO</b> Bruksanvisning (Fagpersonell) .....	44
<b>FI</b> Käyttöohje (Ammattihenkilöstö) .....	49
<b>PL</b> Instrukcja użytkowania (Personel fachowy) .....	53
<b>CS</b> Návod k použití (Odborný personál) .....	58
<b>HR</b> Upute za uporabu (Stručno osoblje) .....	62
<b>TR</b> Kullanma talimatı (Uzman personel) .....	67
<b>EL</b> Οδηγίες χρήσης (Τεχνικό προσωπικό) .....	71
<b>RU</b> Руководство по применению (Квалифицированный персонал) .....	76
<b>JA</b> 取扱説明書 (有資格担当者) .....	81
<b>ZH</b> 使用说明书 (专业人员) .....	86

**INFORMATION**

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### **INFORMATION**

Date of last update: 2021-09-30

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The "4P862 C-Leg Guard" will hereon be referred to as the guard/product. These instructions for use provide you with important information on the use, adaptation and handling of the product. Only put the product into use in accordance with the information contained in the accompanying documents supplied.

## 2 Product description

### 2.1 Function

The guard functionally and visually sheaths the C-Leg electronic knee joint. In the covered area, the guard protects the housing of the knee joint against external influences such as scratching and surface contamination. The guard is open on the front. The guard is not required for the proper and safe functioning of the joint. To fully sheathe the knee joint, the 4P863=\* shield insert has to be used at the front.

### 2.2 Combination possibilities

Prosthetic components that are suited for combination with the product are listed below.

#### Knee Joint Combination Possibilities

Designation	Reference number
C-Leg	3C98-3/3C88-3

#### INFORMATION

The product is **not** compatible with the charger extension cable – 4X156\* ankle, charger extension cable – 4X157\* knee head and charger extension cable – 4X158\* ankle, long.

## 3 Intended use

### 3.1 Indications for use

The product is to be used **solely** for lower limb exoprosthetic fittings.

### 3.2 Conditions of use

The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

### 3.3 Contraindications

- All conditions which contradict or go beyond the specifications listed in the section on "Safety" and "Indications for use".

## 4 Safety

### 4.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

### 4.2 Structure of the safety instructions

#### **CAUTION**

##### **The heading describes the source and/or the type of hazard**

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in case of failure to observe the hazard

- > E.g.: Consequence 2 in case of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

### 4.3 General safety instructions

#### CAUTION

##### **Non-observance of safety instructions**

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety instructions and the stated precautions in this accompanying document.

#### CAUTION

##### **Improper disassembly, assembly and handling of the product**

- > Injury due to pinching of body parts.
- > Possible injury in case of parts with sharp edges.
- ▶ Use increased caution during disassembly/assembly and handling of the product.

#### CAUTION

##### **Changes or modifications to the product made independently**

Injury due to faulty operation or malfunction of the product.

- ▶ Have any changes or modifications to the product carried out only by authorised, qualified Ottobock personnel.

#### NOTICE

##### **Mechanical damage to the product**

Change in or loss of functionality due to damage.

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

## 5 Scope of Delivery and Accessories

### 5.1 Scope of delivery

- 1 pc. 4P862 C-Leg Guard
- 1 pc. Instructions for use (qualified personnel)

### 5.2 Accessories

The following components can be ordered separately as needed:

- 1 pc. 4P863=\* shield insert

## 6 Use

- 1) Open the closures on the guard.
- 2) Spread open the guard.
- 3) Put the guard over the knee joint from the front (see fig. 1).
- 4) Squeeze the guard together with both hands (see fig. 2).

**INFORMATION: The retainers (detail, see fig. 2) of the guard have to engage on the frame element of the knee joint.**

- 5) Fasten the two closures (see fig. 3).

If the optional shield insert is used, the following steps must also be completed:

- ▶ Slide the shield insert into the recess of the guard from the front and let it engage (see fig. 4).

#### NOTICE

##### **Loss of stability due to inadequately installed retainers**

No hold of the guard on the knee joint.

- ▶ Ensure that all retainers and closures are properly engaged and/or fastened.

## 6.1 Customising the shield insert

#### ⚠ CAUTION

##### **Painting the shield insert**

Falling due to unexpected product behaviour caused by changed damping behaviour.

- ▶ Only use paints that do not contain any electrically conductive particles and cannot be magnetised. This includes paints that are free of metallic additives (pure synthetic paints).
- ▶ In case of doubt, ask your specialist dealer whether the paint contains additives.
- ▶ Ensure that the paints being used have no negative influence on people and their environment.
- ▶ Some paints contain solvents that release harmful vapours during processing. Observe the information provided by the manufacturer.
- ▶ The paint is safe after it has dried.

- 1) Clean and degrease the 4P863=4 customisable shield insert (e.g. with Hesse-Lignal NV 395).
- 2) Pre-treat the outer surface of the shield insert with a bonding agent for polycarbonates (e.g. Hesse-Lignal TG 5250 adhesive primer) according to the processing instructions for the bonding agent.

The bonding agent being used specifies which paint can be used for painting.

**INFORMATION: Follow the processing instructions for the chosen paint to obtain optimal painting results.**

## 6.2 Functional Test

#### INFORMATION

All of the points described in this section must be completed exclusively by qualified personnel.

After assembly, the following points have to be checked:

- Is the distance between the socket and guard at least 5 mm?
- Is the function of the rotation adapter impaired?

Material may be removed from the guard to improve the functionality.

**INFORMATION: If material is removed, the stability/flexibility of the guard on the knee joint may be impaired.**

## 7 Maintenance

- ▶ During the standard inspection of the prosthesis components, also inspect the product.

### 7.1 Cleaning and Care

#### NOTICE

##### **Improper product care**

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Only clean the product with a damp cloth and mild soap (e.g. 453H10=1-N Ottobock DermaClean).

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.

3) Allow to air dry in order to remove residual moisture.

## 8 Legal information

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 8.2 CE conformity

Otto Bock Healthcare Products GmbH hereby declares that the product is in compliance with applicable European requirements for medical devices.

The full text of the regulations and requirements is available at the following Internet address: <http://www.ottobock.com/conformity>

### 8.3 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

## 9 Technical data

Environmental conditions	
Storage and transport in original packaging	-25 °C/-13 °F to +70 °C/+158 °F
Storage and transport without packaging	-25 °C/-13 °F to +70 °C/+158 °F Max. 93 % relative humidity, non-condensing
Operation	-10 °C/+14 °F to +60 °C/+140 °F Max. 93 % relative humidity, non-condensing

General information	
Reference number	4P862
Product service life	Wear part subject to normal wear and tear
Guard weight	225 g

## 10 Appendices

### 10.1 Symbols Used

Manufacturer

Declaration of conformity according to the applicable European directives

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